# Safety Instructions

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- EN Safety Instructions for Ivoclar Vivadent denture teeth
- DE Sicherheitshinweise für Ivoclar Vivadent-  
Prothesenzähne
- FR Consignes de sécurité relatives aux dents  
prothétiques Ivoclar Vivadent
- IT Avvertenze di sicurezza per i denti protesici  
Ivoclar Vivadent
- ES Indicaciones de seguridad para los dientes  
artificiales Ivoclar Vivadent
- PT Instruções de Segurança para os dentes  
artificiais da Ivoclar Vivadent
- SV Säkerhetsinstruktioner för Ivoclar Vivadent  
proteständer
- DA Sikkerhedsanvisninger for Ivoclar Vivadent  
protesetænder
- FI Ivoclar Vivadentin hammasproteesihampaita  
koskevat turvallisuusohjeet
- NO Sikkerhedsanvisninger for protesetenner  
fra Ivoclar Vivadent
- NL Veiligheidsinstructies voor prothese-  
elementen van Ivoclar Vivadent
- EL Οδηγίες ασφαλείας για τεχνητά δόντια της  
Ivoclar Vivadent
- TR Ivoclar Vivadent protez dişler için güvenlik  
talimatları
- RU Инструкции по технике безопасности  
при работе с искусственными зубами  
Ivoclar Vivadent
- PL Instrukcje bezpieczeństwa dotyczące  
zębów do protez
- SL Varnostna navodila za umetne zobe  
Ivoclar Vivadent
- HR Sigurnosne upute za Ivoclar Vivadent  
zube za protezu
- CS Bezpečnostní pokyny pro zuby  
Ivoclar Vivadent do snímatelných náhrad
- SK Bezpečnostné pokyny pre zubné protézy  
Ivoclar Vivadent
- HU Biztonsági tudnivalók az Ivoclar Vivadent  
műfogsor használatához
- SR Безбедносно упутства за Ivoclar Vivadent  
протетску зубну надокнаду
- МК Безбедносни упатства за забните п  
ротези Ivoclar Vivadent
- BG Инструкции за безопасност при работа  
със зъбни гарнитури Ivoclar Vivadent
- SQ Udhëzimet e sigurisë për proteza  
dhëmbësh "Ivoclar Vivadent"
- RO Instrucțiuni referitoare la siguranță  
pentru dinții de proteză Ivoclar Vivadent
- UK Інструкції з техніки безпеки у процесі  
роботи зі штучними зубами  
Ivoclar Vivadent
- ET Ivoclar Vivadenti hambaproteeside  
ohutusjuhised
- LV Ar Ivoclar Vivadent zobu protēzēm  
saistītās drošības instrukcijas
- LT „Ivoclar Vivadent“ dantų protezų saugos  
instrukcijos

Date information prepared: 2019-10-30/WW/Rev. 0

Ivoclar Vivadent AG  
Bendererstrasse 2, 9494 Schaan/Liechtenstein  
[www.ivoclarvivadent.com](http://www.ivoclarvivadent.com)

  
ivoclar  
vivadent®  
passion vision innovation

## Safety instructions for Ivoclar Vivadent denture teeth

It must be ensured that the stability of the teeth and the correct shade effect are maintained. It is therefore recommended to observe the minimum layer thickness.

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To ensure that removable dentures remain functional over longer periods of time, the following prerequisites should be met:

- The dentures should be planned and fabricated in close cooperation with the dentist.
- Dentist and dental technician should ensure a balanced occlusion and articulation.
- All dental products should be used according to the instructions of the manufacturer. Denture teeth, should not be weakened by excess grinding.
- In order to ensure a reliable bond for PMMA/composite teeth with the denture base material, the teeth are cleaned with steam, roughened and wetted with monomer. When completing the denture with an auto-curing polymer, we also recommend using mechanical retentions.
- The application of a glaze varnish or denture cleaning spray is not recommended.
- The teeth should not come into contact with a flame or direct heat (for example alcohol torch).
- Heat development as a result of too much pressure during grinding must be prevented.
- Cleaning is performed with commercial cleaning agents. Contact with solvents must be prevented.
- If an allergy against components of the product is known, prefabricated teeth should not be used.
- When using C&B veneering materials, the corresponding Instructions for Use must be observed.
- We recommend wearing a face mask and using extraction equipment for protection against grinding dust.
- We recommend wearing protective goggles to protect your eyes from grinding chips.

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**Ivoclar Vivadent Limited**
**Ivoclar Vivadent, Inc.**