signum®

universal bond

(DE) Gebrauchsanweisung	3	(HU) Használati utasítás	82
(GB) Instructions for use	10	(LV) Lietošanas instrukcija	88
(FR) Mode d'emploi	16	(LT) Naudojimo instrukcija	94
(ES) Instrucciones de uso	22	(PL) Instrukcja obsługi	100
(IT) Istruzioni per l'uso	28	(HR) Upute za uporabu	106
(PT) Instruções de uso	34	(RO) Instrucțiuni de utilizare	112
(NL) Gebruiksaanwijzing	40	(TR) Kullanma talimatı	118
(SE) Bruksanvisning	46	(KR) 사용 설명서	124
(DK) Brugervejledning	52	(BG) Инструкции за употреба ..	130
(NO) Bruksanvisning	58	(EE) Kasutusjuhend	136
(FI) Käyttöohjeet	64	(SI) Navodila za uporabo	142
(GR) Οδηγίες χρήσης	70	(RS) Uputstva za upotrebu	148
(CZ) Návod k použití	76		

KULZER
MITSUI CHEMICALS GROUP

Flüssig
Liquid
Liquide
Líquido
Liquido
Líquido
Vloeibaar
Flytande
Flydende
Flytende
Nestemäinen
ρευστό
Kapalná

Folyékony
Škidrs
Skysta(s)
Płynny
Tekuč
Lichid
Sıvı şekilde
액체의
течно
Fluïd
Tekoč
Tečno

MD

Medizinprodukt
Medical device
Dispositif médical
Producto sanitario
Dispositivo medico
Equipamento para saúde
Medisch hulpmiddel
Medicinteknisk produkt
Medicinsk udstyr
Medisinsk utstyr
Lääkinnällinen laite
Ιατροτεχνολογικό προϊόν
Zdravotnický prostředek

Orvostechikai eszköz
Medicīniska ierīce
Medicinos priemonė
Wyrób medyczny
Medicinski proizvod
Dispozitiv medical
Tibbi cihaz
의료기기
Медицинско изделие
Meditsiiniseade
Medicinski pripomoček
Medicinsko sredstvo

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Stand: 2022-05

Medical Device – for use by dental health care professionals only.

Intended purpose

Signum universal bond is a bonding agent for crown, bridge and partial framework.

This dental material is suitable for patients requiring dental treatment for the following indications with consideration of the contraindications. For pregnant and nursing women, the treatment risks must be weighed carefully against the benefits taking into consideration the unborn child or infant.

Kulzer medical devices ensure the rehabilitation of oral functions such as chewing, speaking and aesthetics. They stabilize the remaining dentition and/or the alveolar ridge restoratively or prosthetically.

Indications

Metal

- Bonding agent between framework surfaces made of metal alloys (precious metal, non-precious metal or titanium) and (meth-) acrylate-based veneering materials in conjunction with macromechanical retentions.
- Bonding agent between partial framework surfaces made of metal alloys (precious metal, non-precious metal or titanium) and (meth-) acrylate-based veneering materials or denture base materials (hot or auto-polymerizates).

- Bonding agent for the extraoral repair of metal-supported composite veneers.
- Bonding agent for the extraoral repair of porcelain bonded restorations.

Zirconium dioxide

- Bonding agent between frameworks, implant abutments and single crowns made of ZrO₂ and light-curing veneering materials.
- Bonding agent for the extraoral repair of composite veneers on ZrO₂ frameworks.
- Bonding agent for the extraoral repair of ceramics on ZrO₂ frameworks.

PEEK High performance polymers

- Bonding agent between frameworks, implant abutments and individual crowns made of PEEK high-performance polymers and light-curing veneering materials.
- Bonding agent between partial framework surfaces made of PEEK high-performance polymers and light-curing veneering materials or denture base materials (hot or auto-polymerisable).
- Bonding agent for the extraoral repair of composite veneers on PEEK frameworks.

Contraindications

In case of known or suspected allergy to components of the product, its use is contraindicated. Do not use this product in case of known or suspected allergies against (meth)acrylate compounds.

Potential side effects

Hypersensitivities to the product or its components cannot be excluded in individual cases.

Composition

Signum universal bond I: Acetone; MDP; Acetic acid

Signum universal bond II: Methylmethacrylate; UDMA; Mixture of acrylicpolymer in methylmethacrylate; Diphenyl(2,4,6-trimethylbenzoyl)phosphine oxide

Product application

Veneering of using the crown, bridge and partial framework

Framework Modelling and Material Thickness

- See the information and requirements in the instructions for use of the framework material indicated for veneering.
- Frameworks made of PEEK need to be designed with a sufficiently stable cross-section ($\geq 20 \text{ mm}^2$). Due to unstable constructions, framework deformations often lead to loss of the free-end veneer under loading.
Free-end solutions in dental constructions shall be avoided (e.g. in all-on-four implant superstructures, especially in bimaxillary restored patients).

Conditioning zirconium dioxide

- Sandblast zirconium dioxide (Al_2O_3 110–125 μm , 2 bar). Jet angle approx. 45 °, distance between jet nozzle and object surface 2–4 cm, use disposable blasting media. Then clean with a steam jet or clean tap water and dry carefully with clean compressed air (oil-free). Avoid surface contamination.

Conditioning metal or PEEK

- Sandblast metal (Al_2O_3 110–125 μm , 3 bar) or PEEK (Al_2O_3 110–125 μm , 2 bar). Jet angle approx. 45 °, distance between jet nozzle and object surface 2–4 cm, use disposable blast-

ing media. Then clean with clean tap water (no steam jet) and dry carefully with clean compressed air (oil-free). Avoid surface contamination.

- Signum universal bond I is now placed drop by drop, per veneer unit, in a serving bowl and applied to the dry, dust- and grease-free surface with a disposable brush. Allow to air dry.
- Then Signum universal bond II is filled into a serving bowl and applied with another disposable brush - cure in a polymerisation device (90 s in Heraflash, HiLite power or HiLite power 3D).
- In order to achieve an optimal bond it is necessary to apply the opaque without contamination of the intermediate layer (dispersion layer) through contact or a longer break and then to apply the components of the veneering system or the denture base materials (hot or auto-polymerisate) from Kulzer.
- Close Signum universal bond I and II immediately after use.

Repair / restoration of a veneered metal, zirconium dioxide or PEEK framework

- Carefully sand down the damaged area of the veneer down to the framework.
- Sandblast the framework surface as described above.
- Clean with water and compressed air, then dry carefully.
- The exposed framework surface (metal, zirconium dioxide, high-performance polymer) is moistened with Signum universal bond I using a disposable brush. Allow to air dry. Signum universal bond II is then applied thinly with another disposable brush and cured with one of the named polymerization devices.
- Care must be taken to avoid contamination of the intermediate layers (dispersion layer) with impurities.
- Fracture surfaces of silicate ceramic veneers are conditioned with Signum ceramic bond in accordance with the instructions for use.

- Fracture surfaces of composite veneers are conditioned with Signum liquid in accordance with the instructions for use.
- The damaged or chipped area can then be repaired with light-curing veneering materials from Kulzer.
- Close Signum universal bond I and II immediately after use.

General information

Please pay attention to the following safety instructions and all processing instructions in other sections. This product must be used according to the instruction for use only. Any use not complying with these instructions is at the discretion and sole responsibility of the dental professional itself. Visually inspect for damage before use. Damaged products must not be used.

Warnings and safety instructions

Highly flammable liquid and vapour. Wear protective gloves/protective clothing/eye protection/face protection. Avoid contact with skin. May cause an allergic skin reaction. Causes skin irritation. Skin contact may cause sensitization. If on skin (or hair): Take off immediately all contaminated clothing. Rinse skin with water / shower. Avoid contact with eyes. Causes serious eye irritation. In case of contact with eyes, rinse with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If symptoms persist, consult an eye specialist. Avoid breathing dust/fume/gas/mist/vapours/spray. May cause respiratory irritation. May cause drowsiness or dizziness. Toxic to aquatic life with long lasting effects.

For SSCP please see EUDAMED database (<https://ec.europa.eu/tools/eudamed>) or contact sscp-service@kulzer-dental.com

To find the SSCP in the EUDAMED use the UDI code attached at the packaging of the product.

Storage conditions

Store in a well-ventilated place. Store at temperatures of 4 to 25°C (39 to 77°F). Avoid direct sunlight. Store in a dry place. When product has been removed, reseal carefully. Do not use the material after the expiration date.

Disposal information

Recommendation: Dispose of in accordance with official regulations. Do not dispose of contents or partially emptied packaging in the household waste or allow them to enter the sewage system. European Waste Catalogue: 180106 Chemicals consisting of or containing hazardous substances.

Complaint notice

Please quote batch number **LOT** and article number **REF** or UDI in all correspondence about the product. See note on bottle/packaging.

According to EU Medical Device Regulation users/ patients are obliged to report serious incidents with a medical device to the manufacturer and to the competent authority of the country, the incident occurred.

Safety data sheets and more information are available at our website **www.kulzer.com**

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® = registered trademark of Kulzer GmbH
® = registrovani zaštitni znak Kulzer GmbH
Kulzer GmbH
Kulzer, LLC