TECHNO TRAY-P

Nuevo color
ROSA

Descripción

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New colour
PINK

Description

Self curing acrylic resin for individual trays and base and registration plates.

Properties

High dimensional stability and low curing contraction.
Easy to handle, it doesn't stick to the fingers.
Easy to cut
Soft in the bur and free of powder

Applications

- For the production of custom trays
- For registration plates

With the new colour pink:

- For the setting up of teeth in complete or partial prostheses.
- for bite rims

More rigid structures than with wax are obtained without the need of metallic reinforcement and it doesn't deform with the heat of the mouth.

Besides the colour is similar to the modelling wax

Composition

Acrylic resin in liquid and powder form.
Powder: methylmethacrylate-copolymer
Liquid: methylmethacrylate-monomer

Application method

Measure out the powder and liquid in the ratio of

1 part of liquid for 4 parts of powder

Pour the measured liquid in a mixing receptacle, add the powder and mix with a spatula for 30 – 40 seconds. The mix will stick on the spatula. Then you can pick the dough with your hands and knead it. The mixed dough must have a great plasticity

The hardening time is about 8 minutes

Finishing

TECHNO TRAY-P can be finished with any kind of cut bur.

Presentation

Green	Pink	
9077-100	9077-105	Kit Standard: 1Kg+300ml+mixing kit
9077-M20	9077-M25	Kit INTRO: 80 gr+ 60 ml+dosers
9077-020	9077-025	1 Kg Powder: 2 x 500 gr
9077-030	9077-035	4 Kg Powder: 8 x 500 gr
9071-020		Liquid: 300 ml
9071-030		Liquid: 2000 ml

Shelf life and storage

TECHNO TRAY-P powder and liquid retain all those qualities over a period of at least 2 years. The expiry date is shown on the container. Store in a dry place at a temperature not exceeding 25 °C. Keep out of sunlight.

Safety cautions

Liquid TECHNO TRAY contains inflammable solvents and must be used in well ventilated no smoking areas and away from naked flames and heat-generating apparatus.

Avoid contact with skin and eyes, wearing protective clothing such as gloves and glasses. In case of skin or eye contact wash liberally with water.

Prolonged contact with the skin may produce hypersensitivity in susceptible persons. Stop use in case of sensitisation.

Liquid spills must be cleaned up immediately and dried.

In case of fire, extinguish using a fire extinguisher, sand, or earth. Keep in a cool dark place, away from naked flames and warm areas. Ignition point: 10 °C.

Additional information

Our recommendations referring to the techniques applied, whether verbally, in writing or in the form of practical instructions, are based on our own experiences and/or tests, and must therefore be considered only as instructions

For further information, see the safety data sheet for the products TECHNO TRAY-P powder and liquid, which can be requested from your usual supplier.

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