# Esthetic Colorant

For KATANA™ Zirconia

CE 1639

EN	ENGLISH	English	Dental Zirconia
FR	FRANÇAIS	French	Zircone dentaire
ES	ESPAÑOL	Spanish	Óxido de zirconio dental
IT	ITALIANO	Italian	Ossido di zirconio dentale
NL	NEDERLANDS	Dutch	Dentaaal zirkoniumoxide
DE	DEUTSCH	German	Dentales Zirkonoxid
SV	SVENSKA	Swedish	Dental zirkoniumdioxid
NO	NORSK	Norwegian	Dental zirkoniumdioksid
FI	SUOMI	Finnish	Hammaslääketieteellinen zirkoniumdioksidi
DA	DANSK	Danish	Dentalt zirconia
PT	PORTUGUÊS	Portuguese	Óxido de zircónio dentário
EL	ΕΛΛΗΝΙΚΑ	Greek	Οδοντιατρικό οξειδίο του ζirkονίου
TR	TÜRKÇE	Turkish	Dental zirkonyum oksit
PL	POLSKI	Polish	Tlenek cyrkonu do aplikacji dentystycznych
RO	ROMÂNĂ	Romanian	Oxid de zirconiu dentar
HR	HRVATSKI	Croatian	Dentalni cirkonij
HU	MAGYAR	Hungarian	Fogászati cirkonoxid
SL	SLOVENŠČINA	Slovenian	Dentalni cirkonij
CS	ČESKY	Czech	Dentální zirkonoxid
SK	SLOVENSKY	Slovak	Dentálny oxid zirkónu
BG	БЪЛГАРСКИ	Bulgarian	Стоматологичен цирконий
UK	УКРАЇНСЬКА	Ukrainian	Стоматологічний цирконій
ET	EESTI	Estonian	Dentaalne tsirkoonium
LV	LATVIEŠU	Latvian	Zobārstniecības cirkonijs
LT	LIETUVIŠKAI	Lithuanian	Odontologinis cirkonio oksidas

<b>MD</b>	EN	Medical Device	DA	Medicinsk udstyr	CS	Zdravotnický prostředek
	FR	Dispositif médical	PT	Dispositivo médico	SK	Zdravotnícka pomôcka
	ES	Producto sanitario	EL	Ιατροτεχνολογικό προϊόν	BG	Медицинско изделие
	IT	Dispositivo medico	TR	Tıbbi cihaz	UK	Виріб медичного призначення
	NL	Medisch hulpmiddel	PL	Wyrób medyczny	ET	Meditsiiniseade
	DE	Medizinprodukt	RO	Dispozitiv medical	LV	Medicīniska ierīce
	SV	Medicinteknisk produkt	HR	Medicinski proizvod	LT	Medicinos priemonė
	NO	Medisinsk utstyr	HU	Orvostechikai eszköz		
	FI	Lääkinnällinen laite	SL	Medicinski pripomoček		

## Kuraray Noritake Dental Inc.

300 Higashiyama, Miyoshi-cho, Miyoshi, Aichi 470-0293, Japan  
TEL +81-561-32-8953 FAX +81-561-32-8976

EC REP

## EMERGO EUROPE

Prinsessegracht 20  
2514 AP The Hague, The Netherlands

## Kuraray Europe GmbH

Philipp-Reis-Str. 4,  
65795 Hattersheim am Main, Germany  
Phone:+49 (0)69 305 35835 Fax:+49 (0)69 305 98 35835  
URL:https://www.kuraraynoritake.eu

# Esthetic Colorant

## For KATANA™ Zirconia

C €1639

### I. Introduction

This IFU is for Esthetic Colorant. Esthetic Colorant is used for coloring pre-sintered zirconia structures and has 12 shade variations: (A plus, B plus, C plus, D plus, BROWN, ORANGE, BLUE, GRAY, VIOLET, WHITE, OPAQUE, and PINK).

### II. Intended Use

Esthetic Colorant is used for coloring pre-sintered zirconia structures.

### III. Composition

Water, Propane-1,2-diol, inorganic pigments, etc.

### IV. Directions for Use

- 1) To achieve consistent coloring results, any dust or grease must be removed from the surface of the pre-sintered zirconia before applying the liquid.
- 2) Select the color shade.
- 3) Make sure the lid is closed and shake the bottle well prior to use.
- 4) Apply GRAY and BLUE to reproduce the enamel color of the occlusal surface and cusp.
- 5) Apply ORANGE to reproduce mamelon color.
- 6) Apply A plus, BROWN, etc to the main groove to reproduce the dentin color.
- 7) Dry at 80-200°C (176-392°F) for at least 30 minutes.
- 8) Sintering is performed according to the instruction manual of each zirconia.

#### [Precautions in use]

- 1) Esthetic Colorant is only available with brush painting technique. Apply using a commercially available brush or brush pen. The dipping technique is not recommended.
- 2) Do not mix each shade of Esthetic Colorant, but it is possible to apply each shade in layers to pre-sintered zirconia.

### V. Remarks on Handling

#### Contraindications:

- If the patient is hypersensitive to zirconia or any other components, this product must not be used.

#### Warning:

- If the patient or the dental professional demonstrates a hypersensitivity reaction, such as rash, dermatitis etc., discontinue use of the product and seek medical attention immediately.

#### Caution:

- 1) Do not touch the skin or eyes when using.
- 2) If the product touches human body tissues, take the following actions:
  - <If the product gets in the eye>  
Immediately wash the eye with plenty of water and consult a physician.
  - <If the product comes in contact with the skin>  
Immediately rinse with plenty of soapy water.
- 3) Do not mix this liquid with other products.
- 4) Precautions for CMR substance (in accordance with CLP Regulation: Regulation (EC) No 1272/2008)  
Read precautions in this instruction for use.
  - Do not handle until all safety precautions have been read and understood.
  - Use personal protective equipment as required.
  - If exposed or concerned: Get medical advice/attention.
  - Store locked up.
  - Dispose of contents/container in accordance with local/regional/national/international regulation.

#### Storage:

1. Store in a cool and dry place. Keep away from direct sunlight.
2. The product should be stored at 10-30°C (50-86°F).
3. After opening this product, be sure to keep the bottle lid tightly closed.

#### [WARRANTY]

Kuraray Noritake Dental Inc. will replace any product that is proven to be defective. Kuraray Noritake Dental Inc. does not accept liability for any loss or damage, direct, consequential or special, arising out of the application or use of

or the inability to use these products. Before using, the user shall determine the suitability of the products for the intended use and the user assumes all risk and liability whatsoever in connection therewith.

#### [NOTE]

If a serious accident attributable to this product occurs, report it to the manufacturer's authorized representative shown below and the regulatory authorities of the country in which the user patient resides.

Refills do not include instructions for use (IFU). We provide the IFU in electronic format on the dedicated eIFU website.

"KATANA" is a trademarks of NORITAKE CO., LIMITED.

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