Directions for Use

Mode d'emploi
Istruzioni per l'uso
Gebrauchsanweisung
Instrucciones de uso

Responsible Manufacturer: **DENTSPLY Prosthetics**
570 West College Avenue
York, PA 17401 U.S.A.
1-800-243-1942
prosthetics.dentsply.com

EC REP DeguDent GmbH
Rodenbacher Chaussee 4
63457 Hanau-Wolfgang
Germany
Tel. +49/6181/5950
degudent.com

Rx only CE
0086

DP-0000176 Rev. 1 (09/16)

THE DENTAL SOLUTIONS COMPANY™

GB

Technical specifications:

Description	Multilayer PMMA	Shaded PMMA	Burnout PMMA
Modulus of Elasticity	>2,200 MPa	>2,200 MPa	N/A
Bending/Flexural Strength	>80 MPa	>80 MPa	>80 MPa
Toughness	(Charpy): >8KJ/m ² (Izod): >1.0 KJ/m ²	(Charpy): >8KJ/m ² (Izod): >1.0 KJ/m ²	N/A
Impact Wear Resistance	>1 x PMMA	>1 x PMMA	N/A
Tensile Strength	>70 MPa	>70 MPa	N/A
Absorption of Water (according to ISO 10477)	<20µg/mm ³	<20µg/mm ³	N/A
Water Solubility	<0.8µg/mm ³	<0.8µg/mm ³	N/A
Residual Monomer Content (24h at 37°C and 30 min in air)	<0.7%	<0.7%	<0.7%
Burning residual (2h at 500°C)	N/A	N/A	<0.05%

Indications:

Dentsply Sirona PMMA is indicated for manufacturing temporary crowns and bridges by a CAD/CAM system. PMMA blocks can also be used as a burnout material for a subsequent casting or pressing process.

Contraindications:

- > Any permanent restoration
- > This medical product may not be used for patients with hypersensitivity to methacrylate or any of the other ingredients
- > Bruxism or recalcitrant parafunctional habits
- > Insufficient available space
- > Endodontic posts
- > Endosseous implants
- > Inlay bridges

Warnings:

Possible cross-reactions or interactions of this medical product with other medical products or material already present in the oral environment must be taken into consideration by the dentist when selecting this medical product.

Precautions:

- > Keep product dust away from eyes
- > After use, wash your hands
- > Do not smoke, eat or drink while handling the product
- > Do not swallow the product
- > Do not inhale dust particles during grinding
- > Use local vacuum suction and suitable mouth/face protection during manual machining at the workplace

The safety and warning notes listed here describe how to use our medical product in a safe and risk-free manner. Notify the dentist in charge of all factors described above if you use this medical product for a custom design and make sure to comply with the pertinent Safety Data Sheets (SDS)

Adverse effects:

If properly processed and used, adverse effects of this medical product are highly unlikely. However reactions of the immune system (such as allergies) to substances contained in the material or localized paraesthesia (such as taste disturbances or irritation of the oral mucosa) cannot be completely ruled out. Should you hear or be informed of any adverse effects - even when doubtful - we would like to request notification.

Symbols on product labels:

- Product code or reorder number
- Batch or Lot number
- Expiration date
- Date of manufacture
- Consult instructions for use

Use Instructions:

Milling provisional crowns and bridges:

Dentsply Sirona PMMA is used on milling technique to manufacture cemented or screwed crowns, bridges, casting patterns - milled by CAM Systems from different manufacturers.

1. Attach the block on the machine following the equipment manufacturer's instructions.
2. When milling multilayered PMMA, the color variation at the element may vary due to the drill tool positioning, see example on pictures 1, 2 and 3.
3. After milling, remove the disc or block from the equipment and remove the sprue.
4. According to the type of requirement of the manufactured work, for finishing, use mills, sanders and rubbers. If gloss is needed, use mechanical polishing with polishing tools.

Important Notes:

- > Avoid heat and light during storage
- > Single use only
- > Avoid overheating during milling and polishing
- > Medical device for intended use only
- > Residual dust during milling operation may cause irritation
- > UV light sterilization only - do not use autoclave

Limits:

- > Multilayer and shaded: Max. 2 fixed sprues plus a suspended sprue
- > Minimum thickness: 1mm occlusion and 0.6mm cervical
- > Minimum connection diameters: Anterior: 3.6mm; Posterior: 4.4mm
- > The provisional can be fixed with all cements for temporary fixation. Eugenol temporary cements may decrease the effect liveness of cementation

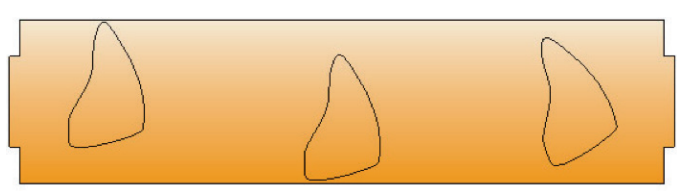
Figure 1: Positioning the piece at the top of the disc or block, more incisal.
Figure 2: Positioning the piece at the bottom of the disc or block, less incisal.
Figure 3: Inclining the piece is possible to obtain a different tone/shade gradient.

Milling patterns for burnout - red and clear PMMA only

Please use the suggested furnace cycle/program shown below:

Step heating	Heat rate	X1	X3	X6	X9
Room temperature to 260°C (500°F)	3°C (37.4°F)/min				
Holding time at 260°C (500°F)*		40 min.	60 min.	70 min.	90 min.
260°C to 580°C (500°F to 1076°F)	6°C (42.8°F)/min				
Holding time at 580°C (1076°F)*		20 min.	30 min.	40 min.	50 min.
580°C (1076°F) to end temperature**	9°C (48.2°F)/min				
Holding time at end temperature*		30 min.	40 min.	50 min.	60 min.

Important Notes:
* If more rings are inserted together in a furnace, each holding time should be extended by 10 minutes.
** Use the end temperature indicated by the investment and restoration material being used, as metal (casting), ceramic pressing, etc.

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