Prior to use, carefully read the instructions for use.

EN

COMPOSITE PRIMER

PRIMER FOR MICRO-FILLED COMPOSITE AND ACRYLIC RESIN

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

- Surface treatment of micro-filled resin (MFR) or micro-filled hybrid resin (GRADIA, GRADIA gum shades (GC)) prior to luting or repairing procedures.
- Surface treatment of acrylic resin prior to repairing procedure.

CONTRAINDICATIONS

Avoid use of this product in patients with known methacrylate monomer and methacrylate polymer allergies.

DIRECTIONS FOR USE

A. Surface treatment of the restoration prior to luting procedure

1. PREPARATION

Prepare the surface to be bonded to using a sandblaster (aluminum oxide) with a pressure of at least 3kg/cm² (2.94 bar). Refer to the manufacturer’s instructions for use. Clean with oil-free air.

Note: Apply masking tape to the surfaces where no treatment is needed.

2. APPLICATION Dispense one or two drops of COMPOSITE PRIMER into a dispensing dish. Apply a thin layer to the surfaces to be bonded to using a brush.

Note:

- Use immediately after dispensing. Replace the bottle cap immediately after use.
- Should the applied material be contaminated prior to light curing, wash and dry the surface and repeat the procedure by re-applying material.

3. LIGHT CURING

Light cure using a visible light curing unit (470 nm wavelength). Refer to the light manufacturer’s instructions for use. After light curing, do not remove the oxygen inhibited outer layer.

Halogen/LED : 20 seconds
G-Light : 10 seconds
Plasma arc : 3 seconds

4. CEMENTATION

Apply a suitable luting cement on the treated surfaces of the restoration and continue with the luting procedure. Refer to the manufacturer’s instructions for use.

B. Surface treatment of the restoration prior to repairing procedure

1.PREPARATION

Prepare the surface to be bonded to using a diamond bur or sandblaster (aluminum oxide) with a pressure of at least 3kg/cm² (2.94 bar). Take the necessary safety measures for the intraoral use of the sandblaster. Refer to the manufacturer’s instructions for use. Clean with oil-free air.

Note: Apply masking tape to the surfaces where no treatment is needed.

2. APPLICATION

Dispense one or two drops of COMPOSITE PRIMER into a dispensing dish. Apply a thin layer to the surface to be repaired using a brush.

Note:

- Use immediately after dispensing. Replace the bottle cap immediately after use.
- Should the applied material be contaminated prior to light curing, wash and dry the surface and repeat the procedure by re-applying material.

3. LIGHT CURING

Refer to A.3. above. When using Labolight DUO, LABOLIGHT LV-III (GC), light cure for 1 minute.

4. REPAIR

Apply composite material such as GRADIA (GC), GRADIA DIRECT (GC), GC KALORE or GRADIA gum shades (GC) on the treated surfaces of the restoration and start to finish. Refer to the manufacturer’s instructions for use.

STORAGE

Recommended for optimal performance, store in a cool, dark place (4-25°C) (39,2-77,0°F). (Shelf life : 2 years from date of manufacture)

PACKAGE

Bottle of 3mL

CAUTION

- COMPOSITE PRIMER is flammable. Do not use near open flame. Keep away from sources of ignition. Do not store large quantities in one area. Keep away from direct sunlight.
- In case of contact with oral tissues, immediately remove with a sponge or cotton pellet. Flush with water after treatment is completed.
- In case of contact with skin, immediately remove with a sponge or cotton pellet. Rinse with soap and water.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- Take care to avoid ingestion of the material.
- Do not mix with any other primers. Do not reuse a brush with other products.
- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.
- In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:

http://www.gceurope.com

or for The Americas

http://www.gcamerica.com

They can also be obtained from your supplier.


CLEANING AND DISINFECTING:

MULTI-USE DELIVERY SYSTEMS: to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect decolor and label for deterioration.

Discard device if damaged.

DO NOT IMMERGE. Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional/national guidelines.

Last revised : 09/2016

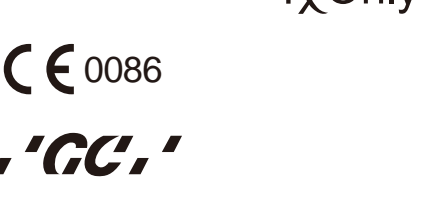MANUFACTURED by
GC DENTAL PRODUCTS CORP.
2-285 Torimatsu-cho, Kasugai, Aichi 486-0844, Japan

RESPONSIBLE MANUFACTURER IN CANADA:

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