## VITA YZ T COLORING LIQUID

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### Instructions for use

#### Area of application

The liquid is used for complete or partial coloration of milled VITA YZ T substructures before dense sintering. It is suitable only for coloring substructures made of VITA YZ T. YZ T COLORING LIQUID is available in 4 different shades, and matched in such a way that all VITA SYSTEM 3D-MASTER and VITA classical A1-D4 shades can be reproduced with the VITA VM 9 veneering ceramic. VITA does not accept any liability when coloring zirconium dioxide substructures of other manufacturers.

#### Use

The **basic shades light/pale** and **medium** are used for the actual shade reproduction (see classification table).

The **additional shade "neutral"** is used for reducing the intensity of the two basic shades (i.e. making them less chromatic). The **additional shade "intense"** serves the purpose of making the two basic shades more intensive (i.e. more chromatic). Both shades can also be used in their pure form: "intense" in order to achieve a very high degree of shade intensity, and "neutral", e.g. to decrease the lightness of zirconium oxide.

#### Classification table for shade reproduction with VITA VM 9

YZ T COLORING LIQUID	VITA 3D-MASTER	VITA classical A1-D4	
<b>Basic shades</b>	<b>light/pale</b>	1M1	A1
		1M2	
		2L1.5	
		2M1	
		2M2	
		2M3	
		2R1.5	
	<b>medium</b>	3M1	A2
		4M1	
		2L2.5	
		2R2.5	
		3L1.5	
		3L2.5	
		3M2	
<b>Additional shades</b>	<b>neutral</b>	3M3	A3.5
		4M2	A4
	<b>intense</b>	3R1.5	B2
		3R2.5	B3
		4L1.5	B4
		4L2.5	C2
		4M2	C3
		4M3	C4
		4R1.5	D2
		4R2.5	D3
5M1	D4		
5M2			
5M3			

Make sure that the surface of the restoration is not too smooth in order not to affect the penetration of the Coloring Liquid. To achieve consistent coloring results, any dust or grease must be removed from the substructure prior to the use. Then a cleaning cycle should be run in a dental ceramic furnace in order to remove coolants or lubricants from the porous structure. Place the substructure on a fibrous pad firing support.

#### Cleaning cycle in the VITA VACUMAT®

Predr. °C	→ min.	↗ min.	↗ °C/min.	Temp. approx. °C	→ min.	VAC min.
500	3.00	6.00	33	700	5.00	–

Shake the bottle or working container well prior to the use! The restoration is immersed in the coloring liquid according to the desired lightness

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GB

level of the shade to be reproduced. The substructure is placed in the liquid in a way to avoid bubbles and to ensure complete coverage with liquid. The recommended immersion time is 2 minutes. When immersing the restoration, vacuum or pressure (2 bars) can be used additionally.

Please note that only acrylic tweezers should be used for immersion. Subsequently remove all excess YZ T COLORING LIQUID with a paper tissue and leave to dry for a short period. Do not sinter in wet condition! Observe special information for high speed sintering. Alternatively, YZ T COLORING LIQUID can also be applied in a thin, homogeneous layer with a brush to the areas of the restoration to be colored. Then sintering and further processing are carried out in accordance with the Working Instructions No. 10166 VITA YZ T / VITA YZ HT. The substructure can be colored on the exterior and the interior at the margins in order to achieve complete penetration of the color.

Due to the higher proportion of material, pontics absorb more color pigments and they may appear to have a higher color saturation. In order to prevent excessive color saturation (chroma) at the margins, these can be moistened slightly using a brush and distilled water before they are immersed or colored. This prevents the penetration of too much color into the pontic.

#### Important:

The brush should only be used for the application of YZ T COLORING LIQUID. Do not use for layering the ceramic – risk of discoloration! Clean the brush only with distilled water. The brush should not contain any metal to avoid reactions.

Do not refill any used liquid into the bottle since this is possibly contaminated with ceramic dust. The liquid can be stored in the container up to one week; then it must be poured away and new liquid must be used. Used liquids can be thinned with a copious amount of water and disposed through the public sewage system.

#### Storage

Store the bottle and the container at room temperature between 10 and 25°C. In order not to change the concentration of the liquid, the bottle and the container must always be sealed well. Keep away from direct sunlight. The shade of VITA YZ T COLORING LIQUID can alter under the influence of daylight. This, however, does not affect its effectiveness.

#### Shelf life

In the original, sealed bottles, the product has a shelf life of 24 months.

#### Explanation of the markings on the packaging

-  Symbol for "Lot number"
-  Symbol for "Use by"
-  Symbol for "Please follow instructions for use"
-  Symbol for "lower and upper temperature limit"

#### Delivery form

VITA YZ T COLORING LIQUID is available as a Complete-Set in 4 shades and as One-Color-Sets in light/pale, medium, intense and neutral.

This product was developed specifically for dental use and must be processed according to the instructions for use. The information/illustrations contained herein do not constitute an assurance of properties and are not binding. Date of issue of these instructions for use: 02.2015.

VITA Zahnfabrik has been certified according to the Medical Device Directive and VITA YZ T COLORING LIQUID bears the designation **CE** 0124

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VITA Zahnfabrik H. Rauter GmbH & Co. KG