FLEXISTONE® Plus

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GB

Instructions for use

Insulating silicone material for use in dental technology, addition curing, low viscosity, high final hardness

Fig. 1

Fig. 2

Fig. 3

1. Dosage and mixing

Fill mixing bowl with base to the first marking, then with catalyst to the second one (Fig. 1). Spatulate **FLEXISTONE® Plus** homogeneously, until a uniform colour is achieved (Fig. 2).

2. Application

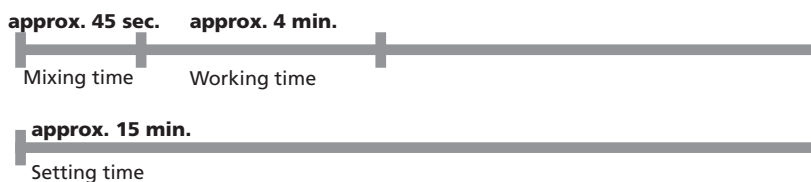
Beginning from the interdental spaces, apply mixed **FLEXISTONE® Plus** uniformly with a spatula reaching a thickness of approximately 3 mm. Prior to setting of **FLEXISTONE® Plus** provide undercuts by means of a wax-knife (Fig. 3). Thus an optimum bonding to the plaster is ensured. No retention crystals are necessary.

3. Deflasking

After polymerisation deflask as usual and remove **FLEXISTONE® Plus**.

Important working hints

- Avoid thin-tapered borders of **FLEXISTONE® Plus** – cutting of a small shoulder is recommended to prevent separation in the flask-counter.
- Increased temperatures accelerate, decreased temperatures retard the setting time.
- Containers should be tightly closed immediately after each use!
- Cured materials are chemically inert – spots on clothing should be avoided!
- **FLEXISTONE® Plus** is recommended for the IVOCLAR SR-Ivocap-System.
- Please follow the instructions of the safety data sheet!

Indications for use:

- Insulating material, A-silicone based, for the application with injecting technique and flask pressing technique. For the insulation and protection of acrylic or porcelain teeth during investing. Can be easily applied in one single procedure onto acrylic teeth and wax, forming a thin, very hard insulating layer for fixing of teeth during pressing.

Technical data:

- **Mixing ratio:** 1:1
- **Colour code:** **FLEXISTONE® Plus** base: blue **FLEXISTONE® Plus** catalyst: white
- **Mixing time:** approx. 45 sec.
- **Working time:** approx. 4 min.*
- **Setting time:** approx. 15 min.*
- **Final hardness:** 85 Shore A
- **Application:** At 23 °C ± 2 °C / 73 °F ± 4 °F, 50 ± 5 % rel. humidity
- **Storage:**

* from beginning of mixing at 23 °C ± 2 °C / 73 °F ± 4 °F, 50 ± 5% rel. humidity. Increased temperatures accelerate, decreased temperatures retard the indicated times.

Ordering information:

FLEXISTONE® Plus
Standard packing **02383**
2 Plastic bottles, ea. 160 ml
base + catalyst 1:1
2 Stands
1 Mixing bowl

Eco-packing **02329**
2 Plastic bottles, ea. 1000 ml
base + catalyst 1:1
4 Mixing bowls

DETAX

GmbH & Co. KG
Carl-Zeiss-Str. 4 · 76275 Ettlingen/Germany
Telefon: 072 43/510-0 · Fax: 072 43/510-100
www.detax.de · post@detax.de

Made in
Germany

05/2019

FLEXISTONE® Plus

Two chemical systems available

recommended by
IVOCLAR for the
SR-Ivoclar-System

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