VivaDent® Polishing Paste

[en] Instructions for Use

Fine - Professional polishing: to clean gently without scratching

Medium - Professional polishing: to clean effectively

Coarse - Professional polishing: to remove stains effectively

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Prophylaxis paste

Patient target group

- Patients with natural teeth
- Patients with restorations

Intended users

- Dentists
- Dental hygienists / Prophylaxis assistants

Special training

None

Use

For dental use only.

Description

VivaDent® Polishing Paste is a prophylaxis paste for the professional cleaning and polishing of teeth and restorations.

The polishing paste is available in the following abrasive grades:

- Fine (pink): Professional polishing: to clean gently without scratching
- Medium (light green) and Medium fluoride-free (dark green): Professional polishing: to clean effectively
- Coarse (light blue): Professional polishing: to remove stains effectively

Indications

None

Areas of application:

Professional cleaning and polishing of teeth and restorations

Contraindications

The use of this product is contraindicated if the patient is known to be allergic to any of the ingredients.

Limitations of use

Single dose must not be reused

Side effects

None known to date

Interactions

None known to date

Clinical benefit

- Removal of staining, plaque and calculus
- Polishing of teeth and restorations

Composition

- Fine: Glycerin, Hydrated silica, Water, Kaolin, Cetearyl alcohol, Sorbitol, Xylitol, Pigments, Flavour, Additives, Sodium fluoride (500 ppm F)
- Medium: Glycerin, Hydrated silica, Pumice, Water, Cetearyl alcohol, Sorbitol, Xylitol, Pigments, Flavour, Additives, Sodium fluoride (500 ppm F)
- Medium fluoride-free: Glycerin, Hydrated silica, Pumice, Water, Cetearyl alcohol, Sorbitol, Xylitol, Pigments, Additives
- Coarse: Pumice, Glycerin, Water, Cetearyl alcohol, Sorbitol, Xylitol, Pigments, Flavour, Additives, Sodium fluoride (500 ppm F)

2 Usage

Delivery form	Sufficient for
Single dose unit	1 full set of teeth
80-g tube	approx. 40 full sets of teeth

1. Apply polishing paste to polishing cup or brush.
2. Perform cleaning/polishing.
Recommended rotational speed for use with rotary instruments: 2000-3000 rpm
3. Rinse thoroughly.

3 Safety information

- The material has been developed solely for use in dentistry.
- In the case of serious incidents related to the product, please contact Ivoclar Vivadent AG, Bendererstrasse 2, 9494 Schaan/Liechtenstein, www.ivoclar.com and your responsible competent authority.
- Check to make sure that the packaging and the product are intact and undamaged before use. If in doubt, contact Ivoclar Vivadent AG or your local sales partner.
- Observe the Safety Data Sheet (SDS).

Supporting documents

Document	To be found at
Current version of Instructions for Use	www.ivoclar.com/eIFU
Structure of Instructions for Use and Warnings	www.ivoclar.com/eIFU
Explanation of symbols	www.ivoclar.com/eIFU
Safety Data Sheet (SDS)	www.ivoclar.com

Disposal information

Contaminated products and remaining stocks must be disposed of according to the corresponding national legal requirements.

4 Shelf life and storage

- Storage temperature: <30 °C (86 °F)
- Date of expiration: see note on Single Dose unit, tube/packaging

5 Legal information

- Keep material out of the reach of children.
- Prior to use, the product must be inspected under the user's responsibility for integrity and suitability.
- All procedures involving the patient must exclusively be performed by qualified dental professionals.
- The manufacturer assumes no liability for any damage caused by use in restoration types not intended by the manufacturer.
- Any liability and warranty claims become void in the event of damage resulting from improper use or use not in accordance with the intended purpose.

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