USER MANUAL CURINGPEN-X

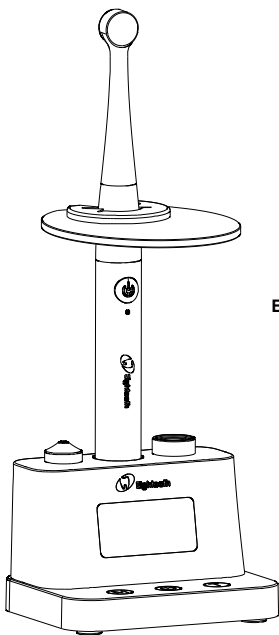
M3

Ultra
3000 mW
cm

3 s

CuringPen-X

English Instructions Page2

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FrançaisInstructions Page41

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Version: 01

IFU-6535051

Issued: 2025.12.30

Size: 176mm×88mm

Contents

1. Scope of CuringPen-X	3
1.1 Parts identification (Fig.1A)	3
1.2 Components and accessories (Fig.1)	3
2. Symbols Used	3
3. Before Use	4
3.1 Product description	4
3.2 Intended purpose	4
3.3 Contra-indications	4
3.4 Intended operate environment	4
3.5 Software versioning	5
3.6 Radiation Instructions	5
4. Setting up of the CuringPen-X	6
5. User interface	7
5.1 Handpiece buttons and indicators	7
5.2 Base buttons and the main screen of the display	7
6. Settings	8
6.1 Set the working mode and light intensity output time	8
6.2 Advanced Settings	8
7. Operations	9
7.1 Device operation	9
7.2 Description of working mode	10
7.3 Charge	10
8. Maintenance	10
8.1 Foreword	10
8.2 General recommendations	11
8.3 Disinfection components	11
8.4 Annually inspection	12
9. Error Tips	12
10. Troubleshooting	13
11. Technical Data	13
12. EMC Tables	14
13. Statement	19

1.Scope of CuringPen-X

1.1 Parts identification (Fig.1A)

① Disposable sleeve;② Light source head;③ Protective light shield;④ Adapter;⑤ Handpiece;⑥ Base;⑦Magnetic head

1.2 Components and accessories (Fig.1)

Handpiece(1pc)- Fig.1B 6551044	Light source head (1pc)- Fig.1C 6551045	Base (1pc)- Fig.1D 6551039
Disposable sleeves (100 pcs)- Fig.1E 6541013	Protective light shield (1pc)- Fig.1F 6551047	Magnetic head (1pc)- Fig.1G 6551048

For different regions, there are several different adapter options to be selected as follows.

Standard	Adapter	Power plug
European standard	Adapter (1pcs) -Fig.1H 6016021	/
American standard	Adapter (1pcs)- Fig.1I 6516003	American standard power plug (1pcs)- Fig.1J 6016011
Multi-standard	Adapter (1pcs)- Fig.1L 6516003	British standard power plug (1pcs)- Fig.1M 6016009
		Australian standard power plug (1pcs)- Fig.1K 6016010
		Argentina standard power plug (1pcs)- Fig.1N 6016014

2.Symbols Used

	If the instructions are not followed properly, operation may lead to hazards for the product or the user/patient.		Additional information, explanation of operation and performance.		Authorized Representative in the European Community
	Type B applied part		Direct current		Humidity limitation
	Serial number		Dispose of in accordance with the WEEE directive		Atmospheric pressure limitation

	Catalogue number		Keep dry		Manufacturer's LOGO
	Manufacturer		Do not reuse		Lot number
	Date and Country of manufacture		Consult instructions for use		Medical Device
	Class II equipment		Temperature limitation		CE marking
	Warning label				

3. Before Use

3.1 Product description

The CuringPen-X is designed to polymerize all photo-activated dental materials in the wavelength range of 380~515nm per ISO 10650:2018. It is a cordless pen-style, LED light polymerization device, and must only be used in hospital environments, clinics or dental offices by qualified dental personnel and not used in the oxygen-rich environment.

3.2 Intended purpose

The CuringPen-X is intended to polymerize resinous dental materials, restorative composite materials, and orthodontic brackets, bonding and sealing materials that are photo-polymerized in the 380~515nm waveband of visible light. This device must only be used in hospital environments, clinics or dental offices by qualified dental personnel and not used in the oxygen-rich environment.

3.3 Contra-indications

Do not use the device for non-dental procedure. Safety and effectiveness have not been established in pregnant women and children.

3.4 Intended operate environment

The operator shall have the following knowledge and skills:
 Understanding of the curing process for light-curable dental materials (e.g., composite resins), including appropriate curing times;
 Proficiency in basic operation of the device, including mode selection and exposure time settings;
 Awareness of safety precautions related to blue/violet light

exposure to prevent accidental irradiation of eyes or soft tissues;
Ability to recognize fault indicators and respond appropriately.

3.5 Software versioning

3.5.1 Handpiece software release version: CPX 1

3.5.2 Base software release version: CPXB 1

3.6 Radiation Instructions

The CuringPen-X emits non-ionizing radiation in the form of visible blue light intended for the polymerization of light-curable dental materials.

Type of radiation: Visible blue light

Wavelength range: 380nm - 515 nm

Intensity: Typical output irradiance of 1200 - 3000 mW/cm²

Distribution: The light beam is focused and approximately circular, with a beam diameter of approx. 8 - 10 mm. The irradiance decreases radially from the center of the beam.

Read the following warnings before use:

- The device must not be placed in humid surroundings or anywhere where it can come into contact with any type of liquids.
- Do not expose the device to direct or indirect heat sources. The device must be operated and stored in a safe environment.
- Do not use this equipment in an environment with oxygen-rich, anesthetic gas or flammable substances. This equipment must be operated, used and stored in a safe environment.
- The device requires special precautions with regard to electromagnetic compatibility (EMC) and must be installed and operated in strict compliance with the EMC information. In particular, do not use the device in the vicinity of fluorescent lamps, radio transmitters, remote controls, portable or mobile RF communication devices and do not use this system near the active HF Surgical Equipment in the hospital.
- Before the first use, please charge the base for at least 4 hours and the handpiece for at least 2 hours. When charging, please use the original power adapter.
- Protective light shield and a disposable sleeve are compulsory during treatment.
- If the light emission window cannot be optimally placed in relation to the composite restoration, the restoration must be polymerized using a conventional method. If soft tissue exposure to the curing light cannot be avoided, the M3 Ultra Mode and M4 Ortho Mode must not be used, as exposure may result in damage of the soft tissues.
- If irregularities occur in the device during treatment, switch it off. Contact your local distributor.
- No modification of this equipment is allowed. Never open or repair the device yourself, otherwise, void the warranty.
- Repair work must only be carried out by qualified SERVICE PERSONNEL. Sifary will provide circuit diagrams, component part lists, descriptions, calibration instructions to assist to SERVICE PERSONNEL in parts repair.
- Only the original light source head, protective light shield, and adapter can be used.
- This device is categorized as a Class 2 Risk Group (Moderate Risk). Avoid direct ocular exposure to the light source during operation. The emitted optical radiation may pose a risk to eyes within 0.5 meters (Ocular Hazard Distance); therefore, avoid direct exposure within this range and use appropriate eye protection.
- The surface temperature of the applied part may reach up

to 50.2 °C during operation. When the surface temperature exceeds 41 °C, the following safe contact conditions shall apply: Recommended maximum continuous contact time: 5 minutes; Use with caution in heat-sensitive patients (e.g., children, elderly, patients with sensitive skin or relevant conditions); If the patient experiences a burning sensation or discomfort, discontinue use immediately.

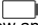
● Use of controls or adjustments or performance of procedures other than those specified herein may result in hazardous radiation exposure

4. Setting up of the CuringPen-X

4.1 Install the light source head

<p>Fig.4A</p>	<p>Make sure the light source head align to the slots of the handpiece. Push gently until there is a "click" sound which indicates that the light source head is securely installed into the handpiece. The light source head can be rotated 360 degrees without being removed, providing flexibility for easy adjustment during the treatment.</p>
----------------------	---

4.2 Plug the adapter

<p>Fig.4B Fig.4C</p>	<p>The base power indicator icon  blinks, indicating that the power is very low and should be charged in time. Insert the Type-C port of the power adapter into the charging port of the base, the charging symbol is displayed in the  upper right corner of the base screen, and the power indicator icon is displayed  dynamically. When the base is charged, the power indicator icon is stationary and the state of full charge is displayed.  Gently place the handpiece into the charging hole on the base. If the handpiece power indicator blinks and the handpiece power indicator icon blinks, the handpiece is being charged. When the charging is complete, the handpiece power indicator is steady green, and the handpiece power indicator icon  stops blinking. </p>
--	--

4.3 Install the disposable sleeve

<p>Fig.4D</p>	<p>Apply a disposable sleeve over the entirety of the light source head and handpiece before beginning a procedure.</p>
----------------------	---

4.4 Install the protective light shield

<p>Fig.4E</p>	<p>Align the gap in the shade plate with the thin position of the light source head, and push gently towards the end after clamping the light source head until the shade is fixed on the handpiece.</p>
----------------------	--

● Only the original light source head can be used. Check the light source head and handpiece before installation. Do not use damaged light source head and handpiece.

● Disposable sleeves must be discarded after each use.

● The light source head, protective light shield, base and handpiece should be cleaned and disinfected after each treatment.

- After installing the light source head, pull it gently to make sure the connection is good, otherwise, it may cause unexpected fault, even hurt the patients.

- Please use the original power adapter for charging. Using a non-original power adapter may damage the device.
- Please insert the power adapter plug correctly into the charging interface on the base.
- The adapter plug serves as the means of disconnection from the supply mains, and disconnection can be achieved by unplugging the adapter.

5. User interface

5.1 Handpiece buttons and indicators

Fig.5A	<p>① Switch button "⊕" ② Power indicator ③ Bluetooth status indicator</p> <p>Power on In the off state, press "⊕", the handpiece beeps, and vibrates, the power indicator lights up, complete the boot and enter the standby state. Output and turn off the light intensity In standby, press "⊕" to start the light intensity output. When the handpiece is outputting, press "⊕" to turn off the light intensity output.</p> <p>Power Off In the standby state, long press "⊕", handpiece beeps, and vibrates, the power indicator turns off, completing the shutdown process.</p> <p>Electric quantity Power indicator "②" green means power higher than 50%, yellow means power 15%~50%, red means power below 15%, charge the device in time. If used continuously, a low battery warning will appear after a period of time, and the device will automatically shut down.</p> <p>Bluetooth connection status Bluetooth status indicator "③" Blue: the handpiece is connected to the base; Blinking: The connection is ongoing; Off: The connection is disconnected.</p>
--------	---

5.2 Base buttons and the main screen of the display

Fig.5B	<p>① Screen ② "Ⓜ" Mode select button ③ "⊕" Switch key ④ "Ⓜ" Time select key</p> <p>Power On In the off state, long press "⊕", the base beeps, the screen will display the startup picture and enter the main interface, boot is complete.</p> <p>Mode selection Tap the "Ⓜ" mode selection button to select the corresponding working mode.</p> <p>Time selection Tap the "Ⓜ" time selection button to select different light intensity output time or number of cycles.</p> <p>Power Off In the startup or standby state, long press "⊕", a</p>
--------	---

	'beep' sound will be emitted from the base, and the screen will turn off, signaling a complete shutdown.
Fig.5C	①Base power ②Bluetooth connection status and handpiece power ③Mistouch prevention ④Working mode ⑤Light Intensity ⑥Light intensity output time

6.Settings

6.1 Set the working mode and light intensity output time

Dental Curing Light CuringPen-X has 8 built-in working modes. Tap the "M" mode selection button to select the corresponding working mode. Tap the "T" time selection button to select different light intensity output time, different working modes and optional time are as follows:

Table 1 Different working modes and optional time

Modes	Optional time
M1 Normal	5s, 10s, 15s, 20s, 40s
M2 High	3s, 5s, 10s
M3 Ultra	1s, 3s
M4 Ortho	2*5s, 3*5s, 4*5s, 5*5s, 10*5s
M5 Pulse	5s, 10s, 15s, 20s
M6 Pamp	5s, 10s, 15s, 20s
M7 Detect	30s, 60s
M8 Detect	60s, 180s, 300s

- The light intensity of M1-M8 working modes is built-in and cannot be modified by the user.
- The equipment should be protected against unauthorized use to prevent accidental radiation exposure.
- Eye and skin protection is recommended for both the operator and the patient/client when the equipment is in operation.

6.2 Advanced Settings

Press and hold down the "M" mode selection button to enter the setting screen. Tap the "M" button to switch to the function you want to set, tap the "C" button to select the setting parameters, and tap the "T" button to return to the home page. After no operation on the setting screen for 30 seconds, the system will automatically return to the home page.

Fig.6

- ① Base Volume (Low, Mid, High, Silence)
- ② Handpiece Volume (Low, Mid, High, Silence)
- ③ Handpiece Pairing (No, Yes)

Tap "T", select yes, and tap "C" to enter Handpiece Pairing. Place the host in the base, hold down the host switch button for 2s to start pairing. After the pairing is successful, the pairing success screen will be displayed, and automatically return to the standby screen after 2s.

- ④ Mistouch Prevention: Off (Mistouch Prevention Off)、Mode (Mode Mistouch Prevention on)、Time (Time Mistouch Prevention on)、Time&Mode(Time Mistouch Prevention on and Mode Mistouch Prevention on)
- ⑤ Language: English, Chinese

- ⑥ Backlight: Low, Mid, High
 - ⑦ Auto Power Off (Base): 5min, 10min, 30min
 - ⑧ Auto Power Off (Handpiece): 3min, 5min, 10min
 - ⑨ Restore Factory Settings: No, Yes
- Tap "ⓘ", select yes, and tap "⊕" to enter Restore Factory Settings.
- ⑩ Version

7. Operations

7.1 Device operation

Fig.7A	<p>7.1.1 Light intensity output</p> <p>After the base is powered on, select a working mode and light intensity output time. After the handpiece is powered on and Bluetooth is connected, press "⊕" start light intensity output to start the countdown. During the output of light intensity, press "⊕" to turn off the output of light intensity. When the light intensity output ends, the device will make a "beep" prompt sound and vibrate.</p> <p>If you do not need to change the working mode and light intensity output time, the handpiece can be used directly.</p>
Fig.7B	<p>7.1.2 Light intensity detection</p> <p>When the light intensity needs to be detected, long press the "ⓘ" button to enter the light intensity detection interface, place the light source head on the metering port, and keep it as parallel as possible. At this time, the current light intensity value will be displayed on the screen.</p>

- When the device is working, do not directly illuminate the eyes, otherwise it will cause injury.
- Do not directly irradiate the skin, otherwise high temperature burns may occur.
- Before using, please try it outside the oral cavity to ensure that there is no problem with the function of the device.
- Do not disassemble light source head during treatment.
- In the process of use, the maximum surface temperature of the light source head can reach 50.2℃, at this time do not touch the light source head, so as to avoid burns.

- During the use of the device at one time, after the cumulative use time reaches 60 seconds in M3 Ultra mode, the output light intensity will be prohibited to prevent patients from being burned by excessive temperature of the light source head. After 60 seconds, the device can continue to be used.
- When using, ensure that the surface of the light source head is clean and the light should be directly illuminated on the light curing material to avoid the effect of curing when there is dirt on the surface of the Light Source head or improper irradiation position.
- A disposable sleeves and a protective light shield must be used.
- Stop using this device when any abnormality occurs.

7.2 Description of working mode

M1	Output light intensity: 1200mW/cm ² (Fig.7C1) After pressing the handpiece "☺" button of handpiece, the set light intensity immediately outputs, and at the end of the treatment, the device beeps, and vibrates.
M2	Output light intensity: 2000mW/cm ² (Fig.7C2) After pressing the "☺" button of handpiece, the set light intensity immediately outputs, and at the end of the treatment, the device beeps, and vibrates.
M3	Output light intensity: 3000mW/cm ² (Fig.7C3) After pressing the "☺" button of handpiece, the set light intensity immediately outputs, and at the end of the treatment, the device beeps, and vibrates.
M4	Output light intensity: 3000mW/cm ² (Fig.7C4) After pressing the "☺" button of handpiece, the set light intensity immediately outputs for 3 seconds, and then extinguished for 2 seconds, and the cycle is repeated, and at the end of the treatment, the device beeps, and vibrates.
M5	Output light intensity: 1200mW/cm ² (Fig.7C5) After pressing the "☺" button of handpiece, the light intensity immediately outputs, flashing every 1 second, and at the end of the treatment, the device beeps, and vibrates.
M6	Output light intensity: 1200mW/cm ² (Fig.7C6) After pressing the "☺" button of handpiece, the light intensity will gradually increase to 1200mW/cm ² within the first 5 seconds, and then continue to output 1200mW/cm ² , and at the end of the treatment, the device beeps, and vibrates.
M7	Output light intensity: 600mW/cm ² (Fig.7C7) After pressing the "☺" button of handpiece, the set light intensity is immediately outputs, and at the end of the detection, the device beeps, and vibrates.
M8	Output light intensity: 600mW/cm ² (Fig.7C8) The white magnetic head is adsorbed on the light source head, and after pressing the "☺", button of handpiece, the set light intensity will be output immediately, and at the end of the detection, the device beeps, and vibrates.

7.3 Charge

- If the handpiece power indicator stays red, it indicates that the battery is less than 15% and must be recharged within 30 days, otherwise the battery may be irreparable damage due to a long period of low power.
- If the base battery indicator icon  blinks, the battery is approximately below 15% and must be recharged within 30 days, otherwise the battery may cause irreparable damage due to low power for a long time.
- If the product is not used for an extended period, please charge the handpiece and base at least once every month.

- Do not change the battery, only trained technician or distributor can change the battery, the electronic parts will be damaged if use a wrong battery or install with a wrong way.

8.Maintenance

8.1 Foreword

For hygiene and sanitary safety purpose, the handpiece, light source head and

protective light shield must be cleaned and disinfected even if the disposable sleeve is used. They should be cleaned and disinfected before each usage to prevent any contamination. This concerns the first use as well as the subsequent uses. Comply with your national guidelines, standards and requirements for cleaning and disinfection.

Reprocessing procedures have only limited implications to this dental instrument. The limitation of the numbers of reprocessing procedures is therefore determined by the function / wear of the device. From the processing side there is no maximum number of allowable reprocessing. The device should no longer be reused in case of signs of material degradation.

In case of damage, the device should be reprocessed before sending back to the manufacturer for repair.

8.2 General recommendations

The user is responsible for the sterility of the product for the first cycle and each further usage as well as for the usage of damaged or dirty instruments, where applicable after sterility.

For your own safety, please wear personal protective equipment (gloves, safety glasses, etc.).

The water quality has to be convenient to the local regulations especially for the last rinsing step or with a washer-disinfector.

Do not use bleach or chloride disinfectant materials.

8.3 Disinfection components

Disinfection components		
Handpiece(Fig.1B)	Light source head(Fig.1C)	Magnetic head(Fig.1G)
Protective light shield(Fig.1F)		
 <p>Before first use, after each use and whenever there is a risk of contamination, disinfect the above components.</p>		
Reprocessing Instructions		
<p>Preparation before cleaning: Immediately after using, remove the used disposable sleeve. Disconnect the components (light source head, protective light shield) from the handpiece. Put the handpiece, light source head and protective light shield in container for storage.</p>		
 <p>Make sure that used sleeves are disposed of as infected waste which is potentially biologically hazardous.</p>		
<p>Transportation: Safe storage and transportation to the reprocessing area to avoid any damage and contamination to the environment.</p>		

Cleaning

Wipe the handpiece, light source head, magnetic head and protective light shield surface thoroughly with a cloth moistened in Ethanol (70 to 80vol%) at least 2 mins, repeat for 5 times. (Visual inspection for cleanliness of the device and its accessories. If necessary, perform reprocessing process again until the component is visibly clean.)

Disinfection

Wipe the handpiece, light source head, magnetic head and protective light shield surface thoroughly with a cloth lightly moistened with Ethanol for Disinfection (Ethanol 70 to 80vol%) at least 2 mins, repeat for 5 times.

Drying

Use a lint free cloth to wipe the surfaces.

- Do not use disinfectants other than Ethanol (70 to 80vol%) for disinfection.
- Make sure no liquid penetrates the handpiece and light source head, otherwise, it will damage the internal parts.
- Adapter and base should be cleaned and disinfected with a cloth moistened in Ethanol (70 to 80vol%) before first use and after each use.

- Do not disinfect the handpiece and light source head in an autoclave or other sterilization container.
- Do not soak or immerse any part of the handpiece and light source head in liquid.

Storage:

Store the components in a clean and dry place for the next treatment.

Transport and storage conditions:

Ambient temperature: -20 °C~+55 °C

Relative humidity: 20%~80 %

Atmospheric pressure: 70kPa~106kPa

8.4 Annually inspection

Extensive checkup for the safe and normal function of the device. Inspect all the connection components on the handpiece.

When not in use, disconnect the power adapter from the charger.

9.Error Tips

Fig.9A	The device handpiece power indicator is red and constantly lit,the battery is about to run out, please charge it in time, if it still cannot be solved after charging, please contact your distributor.
Fig.9B	The base screen displays "Error: Please check the light head.", the LED light head of the device is faulty, please check the light head, if it still cannot be solved after connection, please contact the local dealer.
Fig.9C	The handpiece's prompt tone is prolonged. When the cumulative usage time of the device in M3 Ultra mode reaches its limit (within a single startup), please allow the device to rest for 60 seconds before resuming use.

10.Troubleshooting

When the trouble is found, check the following points before contacting your distributor. If none of these are applicable or the trouble is not remedied even after action has been taken, the product may have failed. Please contact your distributor.

Problem	Cause	Solution
The power is not turned on.	The battery is out of power.	Charge the battery.
	handpiece is broken.	Contact your local distributor.
The device flickers on the screen in standby mode.	The battery is out of power.	Charge the battery.
When the power adapter is plugged in, the base does not display a charging prompt.	There is no electricity in the outlet.	Check the connection.
	Use a wrong adapter.	Use the original adapter.
	The adapter is damaged.	Contact your local distributor.
	Handpiece is broken.	Contact your local distributor.
No sound.	The buzzer is damaged.	Contact your local distributor.
	Set the volume to mute.	Set the volume to non-mute.
The mainframe has no vibration.	The vibration motor is damaged.	Contact your local distributor.
Insufficient light intensity.	There is resin or other contaminants on the surface of the light source head.	Please clean the residue on the light source head.
	The light source head is damaged or worn out.	Replace light source head assembly.
Bluetooth cannot be connected.	The bluetooth module is damaged.	Contact your local distributor.
	The Handpiece and the base are not paired.	Please set it up according to 7.2.

11.Technical Data

Manufacturer	Changzhou Sifary Medical Technology Co.,Ltd.
Model	CuringPen-X
Dimensions	18.8cm×10cm×11.8cm±1cm(package)
Gross weight(including packaging)	730 g±10%
Handpiece power supply	Lithium ion battery: DC 3.7V, 650mAh, ±10%

Base power supply	power	Lithium ion battery*2: DC 3.7V, 1400mAh, $\pm 10\%$
Charger supply	power	AC 100-240 V
Charger output	power	5V  1A
Frequency		50/60Hz
Charger input	power	0.2A
Light intensity		M1:1200mW/cm ² M2:2000mW/cm ² M3:3000mW/cm ² M4:3000mW/cm ² M5:1200mW/cm ² M6:1200mW/cm ² M7:600mW/cm ² M8:1200mW/cm ²
Wavelength		380nm-515nm
Electrical safety class		Class II
Applied part		B(Light Source head and magnetic head)
Operating mode		Intermittent operation M1, M4, M5, M6, M7: 60s. ON / 60s. OFF M2, M3: 15s. ON / 40s. OFF M8: 5mins. ON / 1min. OFF
Ambient conditions		Use: in enclosed spaces Ambient temperature: 10°C~40 °C Relative humidity: 30%~75%; Atmospheric pressure: 70kPa~106kPa
Transport and storage conditions		Ambient temperature: -20 °C~+55 °C Relative humidity: 20%~80 % Atmospheric pressure: 70kPa~106kPa

12.EMC Tables

This product has no essential performance.

Guidance and manufacturer's declaration – electromagnetic emissions		
The CuringPen-X is intended for use in the electromagnetic environment specified below. The customer or the user of the CuringPen-X should assure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic environment-guidance
RF emissions CISPR 11	Group 1	The CuringPen-X uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The CuringPen-X is suitable for use in professional healthcare facility environment.
Harmonic emissions IEC61000-3-2	Class A	
Voltage fluctuations/flicker emissions IEC 61000-3-3	Complies	

Guidance and manufacturer's declaration – electromagnetic immunity			
The CuringPen-X is intended for use in the electromagnetic environment specified below. The customer or the user of the CuringPen-X should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	+/- 8 kV contact +/- 2 kV, +/- 4 kV, +/- 8 kV, +/- 15 kV air	+/- 8 kV contact +/- 2 kV, +/- 4 kV, +/- 8 kV, +/- 15 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrical fast Transients /bursts IEC 61000-4-4	±2kV 100kHz repetition frequency	±2kV 100kHz repetition frequency	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	Line to line: ±0.5kV, ±1kV	Line to line: ±0.5kV, ±1kV	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply lines IEC 61000-4-11	0% UT; 0.5 cycle at 0°, 45°, 90°, 135°, 180°, 225°, 270°, and 315° 0% UT; 1 cycle and 70% UT; 25/30 cycles sine phase at 0° 0% UT; 250/300 cycle	0% UT; 0.5 cycle at 0°, 45°, 90°, 135°, 180°, 225°, 270°, and 315° 0% UT; 1 cycle and 70% UT; 25/30 cycles sine phase at 0° 0% UT; 250/300 cycle	Mains power quality should be that of a typical commercial or hospital environment. If the user of devices require continued operation during power mains interruptions, it is recommended that devices be powered from an uninterruptible power supply or a battery

Power frequency magnetic field IEC 61000-4-8	30 A/m 50Hz or 60Hz	30 A/m 50Hz or 60Hz	Power frequency magnetic field should be at levels characteristic of a typical location in a typical commercial or hospital environment.
Note: UT: rated voltage(s); E.g. 25/30 cycles means 25 cycles at 50Hz or 30 cycles at 60Hz			

Guidance and manufacturer's declaration – electromagnetic immunity			
The CuringPen-X is intended for use in the electromagnetic environment specified below. The customer or the user of the CuringPen-X should assure that it is used in such an environment.			
Proximity magnetic fields	IEC 61000-4-39 test level	Compliance level	Electromagnetic environment – guidance
Proximity magnetic fields	8A/m 30KHz CW	8A/m	Power frequency magnetic field should be at levels characteristic of a typical location in a typical commercial or hospital environment.
Proximity magnetic fields	65A/m 134.2kHz Pulse modulation 2.1 kHz	65A/m	
Proximity magnetic fields	7.5A/m 13.56MHz Pulse modulation 50 kHz	7.5A/m	

Guidance and manufacturer's declaration – electromagnetic immunity			
The CuringPen-X is intended for use in the electromagnetic environment specified below. The customer or the user of the CuringPen-X should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance

Conducted disturbances induced by RF fields IEC 61000-4-6	3 V 0.15 MHz – 80 MHz, 6 V in ISM bands between 0.15 MHz and 80 MHz, 80 % AM at 1 kHz	3 V 0.15 MHz – 80 MHz, 6 V in ISM bands between 0.15 MHz and 80 MHz, 80 % AM at 1 kHz	Portable and mobile RF communications equipment should be used no closer to any part of the CuringPen-X, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.
Radiated RF EM fields IEC 61000-4-3	3 V/m, 80 MHz – 2,7 GHz, 80 % AM at 1 kHz	3V/m	Recommended minimum separation distances Refer to table "Recommended minimum separation distances"
Proximity fields from RF wireless communication equipment IEC 61000-4-3	Refer to table "Recommended minimum separation distances"	Complies	Recommended minimum separation distances Refer to table "Recommended minimum separation distances"

Recommended minimum separation distances

Nowadays, many RF wireless equipment have being used in various healthcare locations where medical equipment and/or systems are used. When they are used in close proximity to medical equipment and/or systems, the medical equipment and/or systems' basic safety and essential performance may be affected. The CuringPen-X has been tested with the immunity test level in the below table and meet the related requirements of IEC 60601-1-2:2014+A1:2020. The customer and/or user should help keep a minimum distance between RF wireless communications equipment and the CuringPen-X as recommended below.

Test frequency (MHz)	Band (MHz)	Service	Modulation	Maximum power (W)	Distance (m)	Immunity test level (V/m)
385	380-390	TETRA 400	Pulse modulation 18Hz	1.8	0.3	27
450	430-470	GMRS 460	FM ± 5 kHz	2	0.3	28

		FRS 460	deviation 1 kHz sine			
710	704-787	LTE Band 13, 17	Pulse modulation 217Hz	0.2	0.3	9
745						
780						
810	800-960	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE Band 5	Pulse modulation 18Hz	2	0.3	28
870						
930						
1720	1700-1990	GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 1, 3, 4, 25; UMTS	Pulse modulation 217Hz	2	0.3	28
1845						
1970						
2450	2400-2570	Bluetooth, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7	Pulse modulation 217Hz	2	0.3	28
5240	5100-5800	WLAN 802.11 a/n	Pulse modulation 217Hz	0.2	0.3	9
5500						
5785						

● Use of accessories and cables other than those specified or provided by the manufacturer of CuringPen-X could result in increased electromagnetic emissions or decreased electromagnetic immunity of CuringPen-X and result in improper operation.

Cable information:

Cable Name	Cable Length (m)	Shielded or not	Remark
Adapter Cable	1.2	No	/

● Use of CuringPen-X adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, CuringPen-X and the other equipment should be observed to verify that they are operating normally.

- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the CuringPen-X, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

Bluetooth RF Characteristics		
Equipment	Frequency range	Output power
Handpiece	2402 - 2480 MHz	6.6 dBm
Base	2402 - 2480 MHz	2.4 dBm

13.Statement

<p>Service Life</p> <p>The service life of CuringPen-X series products is 3 years. It is recommended that the equipment be checked and repaired at the dealer once a year.</p>
<p>Disposal</p> <p>The package should be recycled. Metal parts of the device are disposed as scrap metal. Synthetic materials, electrical components, and printed circuit boards are disposed as electrical scrap. The lithium batteries are disposed as special refuse. Please deal with them according to the local environmental protection laws and regulation.</p>
<p>Rights</p> <p>All rights of modifying the product are reserved to the manufacturer without further notice. The pictures are only for reference. The final interpretation rights belong to Changzhou Sifary Medical Technology Co., Ltd. The industrial design, inner structure, etc, have claimed for several patents by SIFARY, any copy or fake product must take legal responsibilities.</p>

Changzhou Sifary Medical Technology Co., Ltd.

Add: No. 26 Yandanghe Road, Xinbei District, 213000 Changzhou, Jiangsu, China

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- Use of accessories and cables other than those specified or provided by the manufacturer of CuringPen-X could result in increased electromagnetic emissions or decreased electromagnetic immunity of CuringPen-X and result in improper operation.

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- Use of CuringPen-X adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, CuringPen-X and the other equipment should be observed to verify that they are operating normally.

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Changzhou Sifary Medical Technology Co., Ltd.

Adresse: No. 26 Yandanghe Road, Xinbei District, 213000 Changzhou, Jiangsu, China

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All rights reserved.