BISCO

CE 0459

CORE-FLO™ DC Dual-Cured

Flowable Core Build-Up Composite

Instructions for Use

BISCO, Inc.
1100 W. Irving Park Road
Schaumburg, IL 60193
U.S.A.
1-800-247-3368
1-847-534-6000
www.bisco.com

ISO 9001

IN-176R10

Rev. 7/20

Caution: U.S. Federal law restricts this device to sale by or on the order of a licensed Dentist.

en CORE-FLO™ DC Dual-Cured Flowable Core Build-Up Composite

GENERAL INFORMATION

CORE-FLO DC is a flowable dual-cured, radiopaque core build-up composite. CORE-FLO DC is specifically designed to cement posts and build-up the core structure in one easy application. CORE-FLO DC is also an ideal dentin replacement material. CORE-FLO DC comes in an automix dual-syringe and is dispensed with the DUAL-SYRINGE DISPENSER. CORE-FLO DC is intended for use with a bonding agent that is compatible with dual-cured composites, such as BISCO adhesives.

Indications for use:

- The principle indications for use of CORE-FLO DC are:
- 1. Core build-up material
- 2. Cementation of posts or pins
- 3. Dentin replacement material

Intended user: licensed dentist

Warnings:

- Contamination by saliva, blood, or water should be avoided during these procedures. When using dental adhesives, contamination will compromise dentin bonding and may result in decreased longevity of the restore.
- A rubber dam is recommended for all indications where contamination may be possible.

Cautions:

- Cross-contamination:
 - Product may contain items that are designed for one-time use. Dispose of used or contaminated tips. Do not clean, disinfect or reuse.
 - Use of commonly available hygienic protective covering to avoid any contamination of syringes during treatment is recommended. If syringe becomes contaminated, discard. Do not clean or disinfect.

Precautions:

- Product may contain items that are designed for one-time use. Dispose of used or contaminated tips. Do not clean, disinfect or reuse.
- Use of commonly available hygienic protective covering to avoid any contamination of syringes during treatment is recommended. If syringe becomes contaminated, discard. Do not clean or disinfect.

Refer to the manufacturer's instructions. Unlike BISCO adhesives, not all one bottle total-etch or self-etch systems bond to self-cured or dual-cured materials. Prepare the core build-up for taking impressions or fabricating temporary crowns/bridges (it may be sufficient to moisten it thoroughly), in order to prevent any bonding with these materials.

- Refer to DUAL-SYRINGE DISPENSER INSTRUCTIONS FOR USE for proper use, handling, and care.
- If refrigerated, allow CORE-FLO DC to reach room temperature before use.
- Individual component labels for specific expiration dates.
- Safety data sheet available on request.
- Safety data sheet available at www.bisco.com.
- Instructions for Use available on request.
- Instructions for Use available at www.bisco.com.

TECHNICAL INFORMATION

At room temperature (20°C/68°F - 25°C/77°F):
Working Time: Minimum 2 minutes
Setting Time: Maximum 7 minutes
Working time and setting time may vary based on storage conditions, temperature, humidity, etc.

INSTRUCTIONS FOR USE

CORE MATERIAL MIXING PROCEDURES:

- Insert the syringe into a Dual-Syringe Dispenser and remove the syringe cap or mixing tip from the dual-syringe.
- Dispense a small amount of material onto a mixing pad to eliminate any voids or air bubbles.
- Attach a mixing tip to the dual-syringe by lining up the key and keyway. Then turn the brown locking mechanism clockwise.
- Dispense the desired amount of material by using gentle steady pressure. (See "TECHNICAL INFORMATION" section for Working and Setting Time information.)

1. CORE BUILD-UP USING CORE-FLO DC

- Isolate the tooth.
- Follow the DUAL-SYRINGE INSTRUCTIONS FOR USE for loading of the Dual-Syringe for dispensing CORE-FLO DC.

- a. If using a matrix band: place a matrix band around the prepared tooth. Place the end of the intra-oral tip directly into the preparation and dispense CORE-FLO DC. Light cure for 40 seconds.
- b. If using a core form: place the mixing tip directly into the core form and dispense CORE-FLO DC. Place the loaded core form onto the prepared tooth and hold in place. Light cure for 40 seconds.

Prepare and finish the core in the usual manner.

2. POST CEMENTATION AND CORE BUILD-UP TECHNIQUE USING CORE-FLO DC

- Isolate tooth and prepare the tooth according to the post manufacturer's instructions.
- Apply an adhesive according to the manufacturer's instructions; we recommend the use of any BISCO adhesive.

- Follow the DUAL-SYRINGE INSTRUCTIONS FOR USE for loading of the Dual-Syringe for dispensing CORE-FLO DC.
- Light cure the post for 40 seconds.

3. DENTIN REPLACEMENT MATERIAL

- Isolate tooth and prepare in a conservative manner. Rinse and dry.
- Apply an adhesive according to the manufacturer's instructions; we recommend the use of any BISCO adhesive.

- Follow the DUAL-SYRINGE INSTRUCTIONS FOR USE for loading of the Dual-Syringe for dispensing CORE-FLO DC.
- Excess CORE-FLO DC only to the level of the dentine-enamel junction.

- Light cure the CORE-FLO DC for 10 seconds. Continue to firmly pack a light-cured composite, building incrementally until cavity preparation is filled. Light cure according to manufacturer's instructions.
- Check the occlusion and continue with finishing and polishing procedures.

HYGIENE: To prevent polymerization or clogging, the dual-syringe mixing tip should be left in place until the next application. Use of commonly available hygienic protective covering to avoid any contamination of syringes during treatment is recommended. If used intraorally, wipe mixing tip with disinfectant prior to storage.

DISPOSAL: Refer to community provisions relating to waste. In their absence, refer to national or regional provisions relating to waste.

STORAGE: Store kit at room temperature (20°C/68°F - 25°C/77°F). See individual component labels for specific expiration dates.

WARRANTY: BISCO, Inc. recognizes its responsibility to replace products if proven to be defective. BISCO, Inc. does not accept liability for any damages or loss, either direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use, user assumes all risk and liability in connection therewith.

* CORE-FLO is a trademark of BISCO, Inc.

Dual-Syringe Dispenser is distributed by BISCO, Inc.

To Reader Accessories:

- X-81252P - Dual-Syringe Static Mixing Tips (30)
- X-81257P - Dual-Syringe Mixing and Intraoral Tips (30)
- X-81267P - Dual-Syringe Mixing and Root Canal Tips (30)

NOTICE: Report any serious incident to BISCO and the competent authority of the EU member state in which the user and/or patient is established.

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