GC Initial™ Ti Bonder

Special new developed Ti Bonder (Powder & Liquid) to secure optimal bonding between Titanium frame structures and Initial™ Ti porcelain.

INDICATIONS FOR USE

- Crown & Bridge structures made of pure Titanium or a Titanium alloy.
- For casted Titanium as well as for milled Titanium.

DIRECTIONS FOR USE

- Finishing the frame structure is carried out with cross cut burs. These burs should only be used for grinding Titanium.
- Grind with low pressure and slow speed (approx. 15000 rpm)
- Frame work should be grinded in one direction only
- Sharp edges and overlapping should be avoided
- Sandblast with noble corundum only
- Grain size 120 – 150 µm / 2 bar pressure
- Blast in an angle of 45° to the object
- Surface needs a passivating time of 5 minutes
- Steam clean the surface
- Mix the GC Initial™ Ti Bonder powder with the special liquid in a very thin consistency and apply the mixture in a very thin layer.
- Do not apply too thick, the coping should shine through.
- Pre-heating temperature: 450°C
- Drying time: 4 min
- Raise of temperature: 55°C/min
- Vacuum: Yes
- Final temperature: 810°C
- Holding time: 1 min, no vacuum
- The fired bonder surface is dark and has a slightly shiny appearance. Thicker areas are slightly gray, this should be avoided.

STORAGE

Recommended for optimal performance, store in a cool, dry place.

SHELF LIFE

GC Initial™ Ti Bonder Powder: 10 years
GC Initial™ Ti Bonder Liquid: 4 years

NOTES

- Clean the grinding burs using a steam cleaner.
- The Ti Bonder layer does not replace the wash layer.

PACKAGES

GC Initial™ Ti Bonder Powder, 4 g
GC Initial™ Ti Bonder Liquid, 25 mL
GC Initial™ Ti Bonder Intro Kit
(Powder 4 g, Liquid 25 mL)

CAUTION

In case of contact with eyes, flush immediately with water and seek medical attention. In rare cases the product may cause sensitivity to some persons. If such reactions are experienced, discontinue the use and refer to a physician. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:
<http://www.gceurope.com>
or for The Americas: <http://www.gcamerica.com>
They can also be obtained from your supplier.

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