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**EN** INSTRUCTIONS FOR USE

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## **DE** EINFÜGUNG

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## EN INSERT

Instructions for use:

Material: High-quality stainless steel.

Treatments: Prophylaxis, endodontics, periodontics, conservative and restorative dentistry.

### Use:

1. Prophylaxis: use the recommended tips and the corresponding potency for scaling in the ultrasonic scaler.
2. Perio: use the recommended tips and the corresponding potency for perio in the ultrasonic scaler.
3. Endo: use the recommended tips and the corresponding potency for the endo function in the ultrasonic scaler.
4. Preparation of the cavities: use the recommended tips and the corresponding potency for the restorative or conservative dentistry function in the ultrasonic scaler.

### Storage indications:

The tips must be stored in a clean, dry and well ventilated area, situated indoors and away from corrosive gases.

Contraindications:

1. Do not use on patients with haemophilia.
2. Use by doctors with pacemakers and on patients with pacemakers is strictly prohibited.
3. Use carefully on heart disease patients, pregnant women and children.

### Precautions:

1. The tip must be firmly screwed on to the hand-piece.
2. The tip must be tightened when working with sprays.
3. The tip must be cleaned and disinfected before use.
4. Replace the tips when the outer layers are damaged or worn.
5. Do not bend or fold the end of the tip.
6. The golden tips must be rinsed immediately with water after being soaked in H<sub>2</sub>O<sub>2</sub> NaClO, otherwise they will deteriorate.
7. Always check the compatibility of the tips with your ultrasonic scaler.

Connection: use the tightening wrench (torque wrench) to screw in the tips.

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**Guilin Yikeshi Instrument CO.,LTD.**
**Xpedent Austria GmbH.**