PHILIPS

ZOOM!

NiteWhite

DayWhite

Patient instructions

- DA Vejledning til klienten
- DE Patientenanweisungen
- ES Instrucciones para el cliente
- FR Mode d'emploi à l'intention du client
- IT Istruzioni per il cliente
- NL Cliëntinstructies
- NO Instruksjoner for kunde
- PT Instruções para o cliente
- FI Ohjeet asiakkaalle
- SV Klientanvisningar
- EL Οδηγίες για τον πελάτη
- RU Инструкции для пациента
- KK Емделушіре арналған нұсқаулар

English 4
Dansk 12
Deutsch 21
Español 30
Français 38
Italiano 47
Nederlands 56
Norsk 65
Português 73
Suomi 82
Svenska 90
Ελληνικά 99
Русский 108
Қазақша 117

Symbols

Consult instruction manual for use

Keep away from children

Avoid sunlight

Temperature limits

Opened shelf life

Manufacturer

Dentist instructions for Philips ZOOM! NiteWhite and DayWhite Take-Home Whitening products

To be only sold to dental practioners. For each cycle of use, the first use to be only done by dental practioners or under their direct supervision if an equivalent level of safety is ensured. Afterwards to be provided to the consumer to complete the cycle of use.

IMPORTANT

READ ALL INSTRUCTIONS PRIOR TO USING THIS PRODUCT.

Warnings

- This product contains hydrogen peroxide. Avoid contact of the product with eyes as irritation or serious damage to eyes may occur. Rinse with water immediately if this occurs.
- Syringes may be under slight pressure. Remove the cap carefully.
- Harmful when swallowed. If nausea or vomiting occurs, notify your dentist or medical professional right away.
- Do not eat, drink or smoke while wearing custom-fitted trays.
- Do not overload whitening trays with gel as gum irritation may occur.
- Avoid exposure of known hypersensitive teeth and teeth with large pulps and cavities to gel.
- Do not use any other household products for teeth whitening.
- Not for direct sale to the public. For supply only through qualified dental practitioners.
- Read the instructions and use the product accordingly.
- Do not use the product within two weeks prior to or immediately after dental restoration.
- Not for use by pregnant women or habitual tobacco and/or alcohol users.
- Stop using this product immediately if you experience any tooth sensitivity, gum irritation, toothache, defective restorations, gingivitis, nausea, etc.
- Store any remaining whitening gel in a cool dry place for later use.
- Do not freeze.
- Keep out of reach of children.
- Irritant and risk of serious damage to the eyes.
- Not to be used on a person under 18 years of age.

Kit contents

- Custom fitted whitening trays (made by your dentist).
- Whitening gel syringes (2.4 ml)
- Reusable mixing tips
- Tray storage case
- Shade guide
- Patient instructions

Indications for use

Zoom DayWhite (ZDW) and Zoom NiteWhite (ZNW) are tooth whitening gels for use in lightening the color of natural teeth. ZDW is for shorter day-wear usage, while ZNW is for longer wear (overnight). Before any whitening procedure, a consultation with a qualified dentist is recommended to ensure no underlying oral or health issues.

Contraindications

Current research has not evaluated the possible effects of whitening procedures on all patient types. Philips recommends that the following whitening candidates consult their medical professional before use:

- Pregnant and lactating women.
- Habitual tobacco and/or alcohol users.
- Those currently treated for a serious illness e.g. immune compromised, AIDS, etc.

Not to be used on person under 18 years of age.

First aid instructions

- Contact a Poisons Information Centre or a doctor at once if a large quantity is swallowed. Provide an ingredient label if possible.
- If swallowed, do not induce vomiting.
- If gel gets into the eyes, hold the eyelids apart and flush the eyes continuously with running water for at least 15 minutes.
- If clothing, skin or hair contact occurs, remove contaminated clothing and flush the skin or hair with running water.

Safety directions

Keep out of reach of children.


Precautions

- Avoid exposure of hypersensitive teeth to whitening gel.
- Syringes may be under slight pressure. Remove the cap carefully.
- Avoid contact with eyes. Rinse immediately if product comes into contact with eyes.
- If irritation (such as redness, swelling, soreness) of the gums or the mouth occurs, discontinue use and consult a dentist.
- Use for periods of longer than 14 days is to be only under the supervision of a dentist.
- Avoid swallowing.
- Harmful when swallowed.
- Do not use household products to whiten teeth.
- Avoid direct contact of the active surface of the tooth whitening product with the gums and/or salivary flow.

Application instructions

- Floss and brush teeth.
- Twist and pull off clear plastic cap in counterclockwise motion.
- Twist the mixing tip clockwise to lock on the end of the syringe.

1

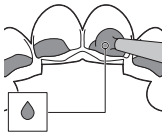- Place a small amount of gel in each tooth compartment in the tray. The gel drop shown represents the average size drop for each tooth compartment, depending on the size of your teeth. (Fig. 1)

TIP! Please take care to not overfill your custom whitening tray; it has been designed to fit snugly against your teeth and if too much gel is added to your tray it may be pushed out and cause gum irritation. Each 2.4 ml syringe contains enough gel for 3 whitening applications (when using both top and bottom trays).

Important: Overloading trays with gel may cause temporary gum irritation.

2

- Place tray with gel in your mouth (Fig. 2). As you insert tray, be careful not to push gel out of the tray. Bubbling within trays is normal.
- Use extra care to avoid getting gel on your gums. Remove excess gel with cotton swab or dry toothbrush.
- Once finished placing gel in the trays, remove the mixing tip from syringe and replace with clear plastic cap. This ensures ingredients in each side of the barrel stay separate for maximum effectiveness.

- After whitening, rinse trays with cold water. Hot water may damage trays. If necessary, use toothbrush to remove residual gel. Place trays in storage case in a cool, dry place.
- Rinse and brush teeth to remove excess gel.

Note: Do not eat, drink or smoke while wearing trays.

Note: Area of tooth closest to gums may take longer to lighten than biting edge.

Note: It is normal to see dark colors in trays where you have amalgam (silver) fillings. Gel oxidizes surface stains on amalgam fillings.

Treatment time

Formula*	Recommended wear time
NiteWhite 10%	Overnight
DayWhite 6%	60–90 minutes once a day
NiteWhite 16%	4–6 hours once daily or overnight

Trays can be worn for up to 2 weeks for optimal results. To maintain your desired smile, repeat your treatment every few months.

*Contains sodium fluoride and hydrogen peroxide. Not all formulas are available for distribution in all countries.

Expectations and optimum results

Results vary by patient. Anatomy, age, tooth condition, hygiene and starting shade impact results.

- You should have realistic expectations about the final results. Use the shade guide provided to record your starting and ending shades.
- Note that crowns, veneers, fillings, dentures and restorations will not whiten and may require replacement following the whitening treatment.
- Although most clients will achieve optimal results within 1–2 weeks, consult your dentist about continued use if not satisfied with the results. Do not use longer than 2 weeks without first contacting your dentist.

Post-Treatment

- Avoid staining foods, drink and tobacco for 30 minutes after each treatment.
- Continue good oral hygiene, use of Philips Sonicare electric tooth brush will help prolong the benefits of whitening treatment.
- Store any remaining whitening gel in a cool dry place for later use. Do not freeze.
- At your regular 6 month check-up, ask your dentist to evaluate your shade.

Potential adverse effects

Adverse effects that may be associated with the use of a whitening product containing peroxide include, but are not limited to:

Sensitivity

- Patients more susceptible to sensitivity are those with exposed root surfaces, defective restorations, exposed dentin, oral tissue injury, periodontal disease, recession, untreated caries, cracked teeth, abfractions, open cavities or leaking fillings.
- Some patients may experience increased tooth sensitivity to cold during the treatment, while others may have non-specific sensitivity in their teeth, gums, tongue, lips or throat. Symptoms should subside within 1-3 days.
- If the sensitivity is in the teeth, try whitening every other or every third night/day or ask your dentist for a lower concentration gel.
- Ask your dentist for a 5% potassium nitrate toothpaste.
- Ask your dentist for Relief ACP Oral Care Gel, to be worn in your custom trays for 10-30 minutes before and after each treatment.
- Aspirin or Ibuprofen may be taken for pain.

Gingival Irritation

- Trays that extend onto the gingiva (gums) may cause soft tissue irritation. If the trays extend over your gums, have your dentist trim them.
- Reduce amount of gel in your trays and wipe off any gel which gets on soft tissue. Teeth whiten when HP is absorbed into the tooth and whitens from the inside, so it is not important to cover the entire tooth.

If all of the above options have been exhausted and you still have gum irritation or sensitivity, treatment should be ceased until the irritation subsides. Resume treatment with shorter, less frequent applications until the desired whitening effect has been achieved. If the discomfort continues, discontinue the treatment and consult your dental professional.

Storage instructions

Store between 15°C and 25°C in a dark, dry place. Storage temperature above 25°C will result in reduced shelf life.

Disposal instructions

Dispose of this product in accordance with local waste regulations.

Warranty

Philips warrants that reasonable care has been used in the design and manufacture of this product. This warranty is in lieu of and excludes all other warranties not expressly set forth herein, whether expressed or implied by operation of law or otherwise, including, but not limited to, any implied warranties of merchantability or fitness for a particular purpose.

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