- |   |  |
|---|--|
| <b>de</b> BOWIE & DICK-TESTPAKET        | <b>cs</b> TESTOVACÍ BALÍČEK BOWIE-DICK   |
| <b>es</b> PAQUETE DE PRUEBA BOWIE-DICK  | <b>ro</b> PACHETUL DE TESTARE BOWIE-DICK |
| <b>fr</b> PACK D'ESSAI BOWIE-DICK       | <b>sk</b> BOWIE-DICK TEST PACK           |
| <b>it</b> PACCHETTO DI PROVA BOWIE-DICK | <b>da</b> BOWIE-DICK TESTPAKKE           |
| <b>pt</b> PACOTE DE TESTE BOWIE-DICK    | <b>sl</b> TESTNI PAKET BOWIE-DICK        |
| <b>nl</b> BOWIE-DICK TESTPAKKET         | <b>et</b> BOWIE-DICKI TESTIPAKETT        |
| <b>pl</b> PAKIET TESTOWY BOWIEGO-DICKA  | <b>hu</b> BOWIE-DICK TESZTCSOMAG         |

**de BOWIE & DICK-TESTPAKET**

**1- BESCHREIBUNG**

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**1- DESCRIPTION**

The Bowie-Dick Test Pack is a critical tool for validating the performance of pre-vacuum steam sterilizers (autoclaves). It ensures the sterilizer effectively removes air and allows steam to penetrate during the pre-vacuum phase, which is essential for proper sterilization.

**2- PURPOSE**

1. Verify the air removal efficiency and steam penetration of pre-vacuum sterilizers.
2. Detect malfunctions such as air leaks, inadequate vacuum pulses, or poor steam quality.

**3- MATERIAL & ASSEMBLY**

1 impermeable barrier - 1 foam - 1 permeable barrier -1 B&D sheet-6 permeable - wrapped with crepe paper.  
 Permeable and impermeable barriers: 350gsm  
 B&D test sheet and warning sheet: 120gsm

**4- COLOR CHANGE**

The indicator on the Bowie Dick Test Sheet will change from blue to black when exposed to a steam sterilization cycle with acceptable vacuum cycles.

**5- PREPARATION**

1. Select a Valid Test Pack:  
Use a commercially prepared, single-use Bowie Dick test pack (check expiration date).
2. Inspect the Sterilizer:  
-Ensure the sterilizer is clean and properly maintained.  
-Confirm the sterilizer is set to operate in pre-vacuum mode (not gravity displacement).
3. Placement:  
-Place the test pack horizontally in the coldest area of the sterilizer chamber, typically near the drain or as specified by the manufacturer.  
-Ensure the test pack is the only item in the chamber during the test (empty chamber test).

**6- RUNNING THE TEST CYCLE**

1. Select the Correct Cycle:  
a) Choose the sterilizer's "Bowie Dick Test" cycle (specific to pre-vacuum sterilizers).  
b) If no preset cycle is available, manually set parameters:  
-Temperature: 134–135°C (273–275°F)  
-Exposure Time: 3.5–4 minutes (after achieving temperature and vacuum).  
-Pre-vacuum Pulses: At least 3 vacuum pulses to remove air.  
c) A short dry time of 2-5 minutes will not affect the test.
2. Start the Cycle:  
-Close the sterilizer door and initiate the cycle.  
-Do not interrupt the cycle once started.

**7- INTERPRETATION**

1. Pass Criteria:  
-Remove the test pack from the sterilizer and allow the pack to cool. Do not place on a cold surface. Peel off outer label and open the outer wrapping crepe paper.  
-Remove the indicator from the pack and dispose of or recycle the remaining pack materials.  
-If the sterilizer is functioning properly, the indicator on the Bowie Dick Test Sheet will show a uniform colour change (black) throughout the entire pattern.  
-No visible unreacted (light-coloured) areas.
2. Fail Criteria:  
-Incomplete or uneven colour change (e.g., streaks, spots, or light areas, a blue area in the centre).  
-No colour change, indicating severe air removal failure.  
-On rare occasions, bubbling of the laminate may occur. If this happens, analyse the card for appropriate colour change as instructed above.

**8- POST-TEST ACTIONS**

1. Pass:  
-Document the test result (date, sterilizer ID, cycle parameters, and operator name).  
-Proceed with routine sterilization operations.
2. Fail:  
a. Immediately take the sterilizer out of service.  
b. Investigate potential causes:  
-Air leaks in the chamber door or seals.  
-Malfunctioning vacuum pump.  
-Blocked steam lines or insufficient steam supply.  
-Repair the sterilizer and repeat the test before reuse.

**9- TESTING FREQUENCY**

1. Daily: Before the first sterilization cycle of the day.
2. After Maintenance: Following repairs, part replacements, or autoclave relocation.
3. After Sterilization Failure: If a previous load failed biological or chemical indicators.

**10- KEY PRECAUTIONS**

1. Do NOT use expired test packs.
2. Never reuse a test pack (single use only).
3. Do not store the product near sterilizing agents
4. Do not expose this product to EO, Dry heat or any sterilization process other than steam.
5. Do not use in a gravity displacement steam sterilization cycle (where there is no pre-vacuum phase to check) or any kind of sterilization process other than vacuum assisted steam.
6. Do not reuse the sterilizer until the chemical indicator has changed homogeneously into the colour indicated in the products
7. Avoid contamination: Handle the test pack with clean gloves.
8. Combine with biological monitoring: Bowie Dick tests validate physical performance but do not replace monthly biological monitoring (e.g., Geobacillus stearothermophilus spores).

**11- TROUBLESHOOTING COMMON ISSUES**

Issue	Possible Cause	Solution
No color change	Inadequate steam exposure	Check temperature sensors and steam quality
Uneven color	Air pockets in the chamber	Test vacuum pump efficiency and seals
Test pack damaged	Improper storage or handling	Replace with a new, undamaged test pack

**12- PACKING**

1 piece in a PE plastic bag, 100psc in a reinforced shipping carton  
 Packing information: product name, manufacture date, expiry date, batch number, manufacturer information will be printed on the label.

**13- SAFETY PRECAUTIONS**

CAUTION: the test pack will be hot and should be opened carefully to avoid thermal injury.

**14- STORAGE**

1. Store at cool, dry place at room temperature 15° – 30°C and 30-80% R.H.
2. Store exposed indicator sheet away from light sources.

**15- EXPIRY DATE**

Expiry date is printed on the product packaging. Do not use after expiration date listed on packs.

**16- LOT NUMBER**

A unique identification code, LOT, is printed on each indicator and packaging labels.

**17- INTERFERING SUBSTANCES OR CONDITIONS**

There are no known interfering substances or conditions that could affect the intended use of the indicator or adversely affect the indicator performance.

**18- RELEASE OF TOXIC SUBSTANCES**

The indicator releases no known toxic substances in sufficient quantities to cause either a health hazard, or a hazard to the intended properties of the product being sterilized before, during or after the sterilization process.

**19- PRODUCT CLASSIFICATION**

Class 1, according to risk.

**20- AUTHORIZATION**

Designed under Quality Systems EN ISO 9001:2015 and EN ISO 13485:2016.

**21- DECLARATION OF CONFORMITY**

Anqing Yipak Packaging Material Co., Ltd declares that the YIPAK Bowie-Dick Test Pack indicator conforms to all applicable portions of ISO 11140-1 Class 2 indicators for use in specific tests and ISO 11140-4.

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**PROCLINIC S.A.U.**
**Anqing Yipak Packaging Material Co.,Ltd**