IPS e.max[®] Ceram Art

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1 Intended use

Intended purpose

Characterization and glazing of ceramic restorations in the anterior and posterior region in the CTE range of $9.4\text{--}17.5 \times 10^{-6}/\text{K}$ ($25\text{--}500\text{ }^{\circ}\text{C}$).

Patient target group

- Patients with permanent teeth

Intended users

- Dentists (clinical procedure)
- Dental laboratory technicians (fabrication of restorations in the dental laboratory)

Special training

None

Use

For dental use only.

Description

IPS e.max Ceram Art are universal staining, glazing and structuring pastes for the characterization and glazing of ceramic materials in the CTE range of $9.4\text{--}17.5 \times 10^{-6}/\text{K}$ ($25\text{--}500\text{ }^{\circ}\text{C}$).

These products are coordinated with the IPS layering, press and CAD ceramics from Ivoclar.

Product name	Product description
IPS e.max Ceram Art Magic Glaze	– Glazing of ceramic anterior and posterior restorations – Diluting of the following pastes: IPS e.max Ceram Art Illusion Gingiva
IPS e.max Ceram Art Magic Glaze FLUO	Glazing of full-contour ceramic restorations in the anterior and posterior region
IPS e.max Ceram Art Shade IPS e.max Ceram Art Essence IPS e.max Ceram Art Illusion	Staining and characterization of ceramic restorations in the anterior and posterior region
IPS e.max Ceram Art Illusion Gingiva	Staining and characterization of the gingival area
IPS e.max Ceram Art Structure	Design and individualization of the morphology and surface structure of ceramic restorations in the anterior and posterior region, as well as the gingival area
IPS e.max Ceram Art Universal Liquid	Diluting of the following pastes: IPS e.max Ceram Art Shade, IPS e.max Ceram Art Essence, IPS e.max Ceram Art Illusion, IPS e.max Ceram Art Illusion Gingiva, IPS e.max Ceram Art Magic Glaze, IPS e.max Ceram Art Magic Glaze FLUO

Technical specifications

Characteristics	Unit	Specifications*	
Flexural strength	MPa	≥ 50	
Chemical solubility	$\mu\text{g}/\text{cm}^2$	< 100	
Coefficient of thermal expansion ($25\text{--}400^{\circ}\text{C}$)	$10^{-6}/\text{K}^{-1}$	IPS e.max Ceram Art Magic Glaze	9.3 ± 0.5
		IPS e.max Ceram Art Magic Glaze FLUO	
		IPS e.max Ceram Art Shade	9.1 ± 0.5
		IPS e.max Ceram Art Essence	
		IPS e.max Ceram Art Illusion	
		IPS e.max Ceram Art Illusion Gingiva	
		IPS e.max Ceram Art Structure	9.2 ± 0.5
Glass transition temperature T_g	$^{\circ}\text{C}$	460 ± 20	
Radioactivity U^{238}	Bq/g	≤ 1.0	

* The performance criteria defined by ISO 6872:2015 have been met. Classification: Dental ceramic type 1 / Class 1

Firing table

These firing parameters are guidance values and they are valid for the Programat® furnaces from Ivoclar Vivadent AG.

Deviations (approx. $\pm 10\text{ }^{\circ}\text{C}/50\text{ }^{\circ}\text{F}$) may occur:

- Depending on the furnace generation
- Ceramic furnaces of other manufacturers
- Regional differences in the supply voltage
- Operating several electrical devices in the same electric circuit

Firing parameters

Stain and glaze firing when using the staining technique (monolithic) IPS e.max Ceram Art Magic Glaze FLUO, Shade, Essence, Illusion, Illusion Gingiva

IPS e.max Ceram Art	Stand-by temperature B [°C/°F]	Closing time S* [min]	Heating rate t [°C/°F/min]	Firing temperature T [°C/°F]	Holding time H [min]	Vacuum on V1 [°C/°F]	Vacuum off V2 [°C/°F]	Long-term cooling L [°C/°F]
IPS e.max® ZirCAD 1-4 units	403/757	IRT/6	45/81	710/1310	2:00	-	-	-
IPS e.max® ZirCAD 5-7 units	403/757	IRT/6	40/72	720/1328	2:00	-	-	**
IPS e.max® ZirCAD 8-14 units	403/757	IRT/6	30/54	730/1346	2:00	-	-	**
IPS e.max® CAD	403/757	IRT/6	45/81	710/1310	2:00	-	-	***
IPS e.max® Press	403/757	IRT/6	45/81	710/1310	2:00	-	-	-
IPS Empress® CAD	403/757	IRT/6	45/81	710/1310	2:00	-	-	-

*IRT standard mode

** In case of significant differences in the cross-sections of individual units within a restoration, long-term cooling L down to 500 °C/932 °F is recommended for each firing cycle.

*** For layer thicknesses exceeding 2 mm, long-term cooling L down to 500 °C/932 °F is required.

Structure firing in the structuring technique (monolithic) IPS e.max Ceram Art Structure / IPS e.max Ceram Art Structure Gingiva

IPS e.max Ceram Art	Stand-by temperature B [°C/°F]	Closing time S* [min]	Heating rate t [°C/°F/min]	Firing temperature T [°C/°F]	Holding time H [min]	Vacuum on V1 [°C/°F]	Vacuum off V2 [°C/°F]	Long-term cooling L [°C/°F]
IPS e.max® ZirCAD 1 unit	403/757	IRT/6	45/81	710/1310	2:00	550/1022	710/1310	-
IPS e.max® ZirCAD 2-4 units	403/757	IRT/6	40/72	710/1310	2:00	550/1022	710/1310	-
IPS e.max® ZirCAD 5-7 units	403/757	IRT/6	45/81	720/1328	2:00	550/1022	720/1328	500/932
IPS e.max® ZirCAD 8-14 units	403/757	IRT/6	30/54	720/1328	2:00	550/1022	720/1328	500/932
IPS e.max® CAD	403/757	IRT/6	45/81	710/1310	2:00	550/1022	710/1310	***
IPS e.max® Press	403/757	IRT/6	45/81	710/1310	2:00	550/1022	710/1310	-

*IRT standard mode

*** For layer thicknesses exceeding 2 mm, long-term cooling L down to 500 °C/932 °F is required.

Stain and Glaze firing in the layering technique (full veneering / partial veneering) IPS e.max Ceram Art Magic Glaze Shade, Essence, Illusion, Illusion Gingiva

IPS e.max Ceram Art	Stand-by temperature B [°C/°F]	Closing time S* [min]	Heating rate t [°C/°F/min]	Firing temperature T [°C/°F]	Holding time H [min]	Vacuum on V1 [°C/°F]	Vacuum off V2 [°C/°F]	Long-term cooling L [°C/°F]
IPS e.max® Ceram / IPS e.max® ZirCAD 1-4 units	403/757	IRT/6	45/81	710/1310	1:00	-	-	450/842
IPS e.max® Ceram / IPS e.max® ZirCAD 5-7 units	403/757	IRT/6	40/72	720/1328	1:00	-	-	450/842
IPS e.max® Ceram / IPS e.max® ZirCAD 8-14 units	403/757	IRT/6	30/54	720/1328	1:00	-	-	450/842
IPS e.max® Ceram / IPS e.max® CAD	403/757	IRT/6	60/108	710/1310	1:00	-	-	-
IPS e.max® Ceram / IPS e.max® Press	403/757	IRT/6	60/108	710/1310	1:00	-	-	-

IPS e.max Ceram Art	Stand-by temperature B [°C/°F]	Closing time S* [min]	Heating rate t [°C/°F/min]	Firing temperature T [°C/°F]	Holding time H [min]	Vacuum on V1 [°C/°F]	Vacuum off V2 [°C/°F]	Long-term cooling L [°C/°F]
IPS Style® Ceram	403/757	IRT/6	60/108	750/1382	1:00	-	-	-
IPS InLine®	403/757	IRT/6	60/108	830/1526	1:00	-	-	-

*IRT standard mode

Indications

None

Areas of application:

Characterization and glazing of ceramic materials:

- Zirconium oxide
- Milling ceramics
- Press ceramics
- Veneering ceramics (all-ceramics & metal-ceramics)

Contraindications

- The use of this product is contraindicated if the patient is known to be allergic to any of its ingredients.

Restrictions of use

The product must not be used in the following cases:

- Staining and glazing of partially crystallized lithium disilicate restorations (in their blue state)
- Reuse of the final restoration

Side effects

None known to date

Interactions

None known to date

Clinical benefit

- Restoration of esthetics

Composition

- IPS e.max Ceram Art Magic Glaze
Components: Alkali aluminosilicate glass, 1,3-butanediol, glycerine
- IPS e.max Ceram Art Magic Glaze FLUO
Components: Alkali aluminosilicate glass, 1,3-butanediol, glycerine
- IPS e.max Ceram Art Shade
Components: Alkali aluminosilicate glass, pigments, 1,3-butanediol, glycerine, water
- IPS e.max Ceram Art Essence
Components: Alkali aluminosilicate glass, pigments, 1,3-butanediol, glycerine, water
- IPS e.max Ceram Art Illusion
Components: Alkali aluminosilicate glass, pigments, 1,3-butanediol, glycerine, water
- IPS e.max Ceram Art Illusion Gingiva
Components: Alkali aluminosilicate glass, pigments, 1,3-butanediol, glycerine, water
- IPS e.max Ceram Art Structure
Components: Alkali aluminosilicate glass, pigments, 1,3-butanediol, water
- IPS e.max Ceram Art Universal Liquid
Components: 1,3-butanediol

2 Application

2.1 Preparing the ceramic surface

NOTICE! Depending on the material, the restoration surface must be prepared in varying ways. Observe the Instructions for Use of the respective material.

NOTICE! Smooth or prepolysh the areas for which a higher gloss is desired after glaze firing using silicone polishers.

Monolithic restorations made of zirconium oxide (ZrO₂)

NOTICE! Make changes to sintered restorations only if absolutely necessary.

1. If absolutely necessary: Prepare the restoration with suitable grinding instruments (see Instructions for Use) at low pressure and low speed.
2. Optional: Blast the restoration with aluminium oxide (Al₂O₃) 25–70 µm at 1 bar/15 psi pressure or 70–110 µm at 1.5 bar/22 psi pressure.
3. Smooth the incisal and occlusal contact points and the basal side of the bridge connectors using rubber polishers.
4. Thoroughly clean the restoration with running water or the steam jet.
5. Dry the restoration.

Monolithic restorations made of lithium disilicate (LS₂) CAD

NOTICE! Do not blast lithium disilicate restorations with Al₂O₃ or polishing beads.

- ✓ The restoration is in its precrystallized (blue) state.
- 1. Prepare restoration using diamond-bonded grinding instruments at low pressure and low speed. Observe the Instructions for Use.
- 2. Clean the restoration with ultrasound in a water bath and / or with a steam jet.
- 3. Dry the restoration.
- 4. Crystallize the restoration.

Monolithic restorations made of lithium disilicate (LS₂) Press

- 1. Prepare the restoration using ceramic- and / or diamond-bonded grinding instruments at low pressure and low speed. Observe the Instructions for Use.
- 2. Blast restoration with Al₂O₃, 100 µm at 1 bar/15 psi pressure.
- 3. Thoroughly clean the restoration with running water or the steam jet.
- 4. Dry the restoration.

Monolithic restorations made of leucite-reinforced glass-ceramic CAD

- 1. Prepare the restoration using ceramic- and / or diamond-bonded grinding instruments at low pressure and low speed. Observe the Instructions for Use.
- 2. Carefully prepare the restoration margins using rubber and silicone polishers. Observe the Instructions for Use.
- 3. Thoroughly clean the restoration with running water or the steam jet.
- 4. Dry the restoration.

Veneering ceramics

- 1. Prepare the restoration using ceramic- and / or diamond-bonded grinding instruments at low pressure and low speed. Observe the Instructions for Use.
- 2. Thoroughly clean the restoration with running water or the steam jet.
- 3. Dry the restoration.

2.2 Characterizing and glazing the restoration

Shade correspondence with A–D and BL shades					
IPS e.max Ceram Art	Shade Dentin 1	Shade Dentin 2	Shade Dentin 3	Shade Dentin 4	Shade Dentin 5
VITA® classical shades	BL	A	B	C	D

* No registered trademark of Ivoclar

Shade correspondence with gingival phenotypes				
Gingival phenotype	pink	red-blue	red-yellow	purple
IPS e.max Ceram Art Illusion Gingiva*	Base rose	Base pink	Base orange-pink	Base pink

*IPS e.max Ceram Art Illusion Gingiva materials can be used for mixing purposes.

The red-shaded IPS e.max Ceram Art Illusion Gingiva intensive red is not intended for mixing purposes with other shades. Use IPS e.max Ceram Art Illusion Gingiva for characterization.

- ✓ The restoration is free of dirt and grease residues.
- ✓ The restoration demonstrates a smooth surface without sharp edges / ridges.
- ✓ IPS e.max Ceram Art was thoroughly mixed with a metal-free spatula.

NOTICE! Avoid pooling and too thick stain layers. More intensive shades are achieved by repeating the staining procedure and firing, not by applying thicker layers.

White / red esthetics for monolithic restorations

Wet-in-wet staining / one-shot

- 1. Apply undiluted IPS e.max Ceram Art Magic Glaze FLUO on the teeth in an even layer using a brush.
- 2. Stain the teeth with IPS e.max Ceram Art Shade Dentin/Incisal and adjust shade in accordance with the shade guide.
- 3. Characterize the teeth with IPS e.max Ceram Art Illusion and IPS e.max Ceram Art Essence.
- 4. Optional: Apply undiluted IPS e.max Ceram Art Illusion Gingiva Base in the gingival area (self glazing).
- 5. Optional: Characterize the gingival area with IPS e.max Ceram Art Illusion Gingiva (self glazing).
- 6. Conduct Stain and Glaze firing. See firing parameters.
- 7. Repeat Steps 1–6, if required.

Conventional staining technique

1. Apply IPS e.max Ceram Art Magic Glaze FLUO alone or slightly diluted with IPS e.max Ceram Art Universal Liquid on the teeth in a thin layer using a brush.
2. Stain the teeth with IPS e.max Ceram Art Shade Dentin/Incisal and adjust shade in accordance with the shade guide.
3. Characterize the teeth with IPS e.max Ceram Art Illusion and IPS e.max Ceram Art Essence.
4. Optional: Apply undiluted IPS e.max Ceram Art Illusion Gingiva Base in the gingival area (self glazing).
5. Conduct Stain firing. See firing parameters.
6. Repeat Steps 1–5, if required.
7. Apply undiluted IPS e.max Ceram Art Magic Glaze FLUO on the teeth in an even layer using a brush.
8. Optional: Characterize the gingival area with IPS e.max Ceram Art Illusion Gingiva (self glazing).
9. Conduct Stain and Glaze firing. See firing parameters.
10. Repeat Steps 7–9, if required.

Structuring technique

NOTICE! Smooth IPS e.max Ceram Art Structure on a mixing pad using a spatula. For better sculptability, moisten the brush / instrument with IPS e.max Ceram Art Universal Liquid. IPS e.max Ceram Art Structure must not be diluted with IPS e.max Ceram Art Universal Liquid.

NOTICE! After application of IPS e.max Ceram Art Structure check the contact area to the antagonist, as well as the contact points.

- ✓ Application in the wet-in-wet technique or the conventional technique has been performed.
- 1. Apply undiluted IPS e.max Ceram Art Structure to the glazed surface of the restoration (self glazing).
- 2. Sculpt the morphology of the restoration.
- 3. Incorporate the microtexture in the surface using instruments and brushes.
- 4. Conduct structure firing. See firing parameters.

White / red esthetics for veneering materials

Layering technique / one-shot

1. Apply IPS e.max Ceram Art Magic Glaze in a thin even layer to the teeth using a brush.
2. Stain the teeth with IPS e.max Ceram Art Shade Dentin/Incisal and adjust shade in accordance with the shade guide.
3. Characterize the teeth with IPS e.max Ceram Art Illusion and IPS e.max Ceram Art Essence.
4. Optional: Apply undiluted IPS e.max Ceram Art Illusion Gingiva Base in the gingival area (self glazing).
5. Optional: Characterize the gingival area with IPS e.max Ceram Art Illusion Gingiva and IPS e.max Ceram Art Essence.
6. Conduct Stain and Glaze firing. See firing parameters.
7. Repeat Steps 1–6, if required.

2.3 Cementation

NOTICE! All working steps in the oral cavity of the patient must exclusively be performed by a qualified professional (e.g. dentist).

- ✓ The cementation surface of the restoration is free of ceramic material.
- 1. Cement the restoration. Observe the Instructions for Use.

3 Safety information

- The product has been developed solely for use in dentistry. Process according to the Instructions for Use.
- In the case of serious incidents related to the product, please contact Ivoclar Vivadent AG, Bendererstrasse 2, 9494 Schaan/Liechtenstein, www.ivoclar.com and your responsible competent authority.
- Do not inhale ceramic dust during finishing. Use suction equipment, a face mask and protective goggles.
- If applied frequently or for a prolonged period of time, peroxide-based professional bleaching agents (carbamide peroxide; hydrogen peroxide) as well as acidulated phosphate fluorides used for caries prevention can make the surface of restorations rough and matte.
- Check to make sure that the packaging and the product are intact and undamaged before use. If in doubt, contact Ivoclar Vivadent AG or your local sales partner.

Supporting documents

Document	To be found:
Current version of Instructions for Use	www.ivoclar.com/elFU
Structure of Instructions for Use and Warnings	www.ivoclar.com/elFU
Explanation of symbols	www.ivoclar.com/elFU
Safety Data Sheet (SDS)	www.ivoclar.com
Summary of Safety and Clinical Performance – SSCP	https://ec.europa.eu/tools/eudamed
Basic UDI-DI: 76152082ACERA014EZ	

Disposal information

Dispose contaminated products, remaining stocks or removed restorations according to the corresponding national legal requirements.

Residual risks

Users should be aware that any dental intervention in the oral cavity involves certain risks.

The following known residual clinical risks exist:

- Chipping / fracture of restorative material may lead to swallowing or inhalation of material and to repeated dental treatment.

4 Shelf life and storage

- Storage temperature 2–28 °C (36-82 °F)
- Store the product free of vibration.
- Store the product in a dry place.
- Protect the product from sunlight.
- Do not use the product after the indicated expiry date.
- Date of expiration: See note on packaging

5 Additional information

Keep material out of the reach of children!

The manufacturer accepts no liability for damages resulting from improper use of the product. Furthermore, the user is responsible for determining the product's suitability for the user's own purposes before use.

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