DESIGN 4DISKS

DIGITAL PRECISION SOLUTIONS PMMA Base

de Gebrauchsanweisung
en Instructions for use
es Instrucciones de uso
fr Mode d'emploi

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en Polymer blank for denture bases

Product description

Dental polymer block for milling removable or fixed denture bases using CAD/CAM technology. 4DESIGN PMMA Base blocks stand out for their unalterable colours, hardness and abrasion resistance and are available in different gingival pink colours that provide an extremely natural aesthetic effect.

Composition

PMMA-based acrylic copolymer > 99% / Inert mineral pigments and other non-toxic substances <1%.

Intended use and purpose

Intermediate product for the milling of removable or fixed denture bases to remedy the patient's lack of natural teeth, alleviating the functional and aesthetic problems caused by this deficiency.

Intended users

Exclusively dental health professionals: dental technicians and dentists.

Indications

Removable or fixed dental prostheses.

Kontraindications

Patients with known allergy/sensitivity to any minor component of the polymer blocks (PMMA, acrylic material).

Side effects

In individual cases, sensitivity or allergies may occur when using acrylic resin dentures. In this case, do not use them and consult a doctor.

Instructions/method of use

Can be used with all open CAD/CAM milling systems (98.5 mm discs) and with some special systems (other discs and blocks depending on measurements and contours). Secure the block in the milling machine in the usual way. Hard metal (tungsten) milling cutters can be used for machining. The milled structures can be separated using milling cutters or cutting discs for resin.

Finishing and polishing

Polish the milled framework using a hand tool without applying excessive pressure and at low speed to avoid overheating. Good hair, cotton or leather brushes and universal polishing paste or fine-grained pumice stone can be used for this purpose. Polishing the restoration is necessary to achieve perfect aesthetic results and prevent plaque accumulation.

Make-up (optional): After milling, make-up can be applied using conventional coating materials. MMA-based materials are preferred. When coating with a composite, adhesion with 4DESIGN PMMA Base should be done according to the instructions for use of that composite.

Design specifications: Dental structures are designed in CAD software.

A 5-axis milling machine is required to reach the undercut areas.

Minimum wall thickness: 2.0 mm. If the thickness is less, breakage may occur.

4DESIGN PMMA Base blocks are indicated for milling denture bases, whether total or partial.

Fixing the teeth to the base: Only adhesives compatible with acrylic materials should be used. Always follow the instructions for use and the recommendations of the manufacturers of the fixing materials. The use of products not suitable for PMMA resins may result in adhesion failure.

Cleaning and disinfection: Clean and/or disinfect cold, as PMMA may deform or crack at high temperatures. Use a maximum 10% alkaline solution. Avoid using highly alkaline solutions, strong oxidising agents or products not recommended for PMMA. Do not clean, degrease or condition the PMMA surface with acetone, chloroform or strong organic solvents, as these agents can damage the structure of the material.

Precautions

The product is not resistant to oxidising chemicals and strong acids.

During finishing, use a vacuum cleaner and/or mask to avoid breathing in dust.

The material is not visible with X-rays.

Do not reuse the product and do not use expired materials.

Do not vary the working parameters established by the manufacturer (mechanical modifications to the material could occur, with consequences for the final manufactured product).

Do not contaminate the product during the processing phase.

Warnings

Intended use

The product is indicated for milling removable or fixed dental prostheses. Regular supervision by a dentist at least once a year is recommended to ensure that the restoration continues to fulfil its intended purpose. The prescriber should review the suitability of PMMA restorations in patients suffering from parafunctional habits such as bruxism.

About interactions

Potential interactions with other products used in treatment are highly unlikely because the product is a fully polymerised acrylic resin-based material. Before using the product, the dentist should consider known interactions and cross-reactions of the product with other materials already present in the patient's oral environment to ensure that the restoration can be placed safely and durably.

Storage conditions

The product does not require special storage conditions.

Risks of reuse

The product can be reused as long as there is space available to mill the structures. Clean the product after each use and store it properly labelled in its box. Caution: reusing dentures and other milled dental structures used on other people can cause infections and transmit diseases.

About product disposal

Machining residues from the blocks do not require specific disposal and can therefore be disposed of in household waste. Accompanying packaging items such as boxes should be disposed of in the appropriate containers (cardboard). Contaminated products that pose biological and/or infection risks must be disposed of in accordance with applicable regulations.

Information for the patient

The professional must inform the end user of the following instructions:

a) Maintain normal oral hygiene and adopt a daily hygiene protocol*, avoid excessive brushing that can cause abrasive wear, and do not immerse the prosthesis in alkalis, acids or other substances that may damage acrylic materials.

b) The material is not visible on X-rays.

c) The material is relatively tough, but strong bites should be avoided.

d) In case of allergic reactions to the material, consult a doctor.

e) The following residual risks: Allergies and irritations caused by the material. Breakage and possible ingestion of the product. Deterioration of the product. Incorrect use of the product. Inadequate hygiene and maintenance of the product.

**Hygiene protocol:* Patients should brush their dentures at least once a day. Brush them with a soft-bristled brush and a non-abrasive denture cleaner to remove food, plaque and other deposits.

Manufacturer's responsibility

Material for dental use: The manufacturer is not liable for any damage that may result from use other than as instructed. Class IIa medical device: Any serious incident related to the product must be reported to the manufacturer and the competent authority of the Member State in which the incident occurred.

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