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ORTHODONTIC FACE MASK **CE**
INSTRUCTIONS FOR USE

GENERAL INFORMATION

The Orthodontic Face Mask is a medical device used in orthodontic therapies for the correction of dental malocclusions.

The device is used as external anchorage. After being adapted and shaped to the dimensions of the patient's face, the mask is placed on the forehead and on the chin by means of the forehead and chin support and connected to various devices already in place in the patient's mouth by means of elastic bands that are hooked to the grill supports. In this way, anterior-posterior traction forces are obtained which allow the altering of the position of the teeth and the maxilla.

The pack contains:

- 1 orthodontic face mask
- 1 hex key for adjusting

The device is available in two sizes, MINI and MAXI, and in different colours. Information on the purchased device is shown on the label affixed to the packaging.

The Orthodontic Face Mask is a multipurpose single-patient medical device.

The device can be used on children from the age of 6 and adults.

The device is subject to medical prescription. It is the responsibility of the dentist to evaluate the most suitable treatment based on the patient's clinical conditions.

The device contains tissues of animal origin.

The useful life of the device is set at five years.

CONTRAINDICATIONS

The device contains nickel and chromium. Patients allergic to these metals should not be treated with the device.

SAFETY WARNINGS

It is the responsibility of the dentist to inform the patient of the contents of these instructions for use or to ensure they are provided.

Use the device exclusively in accordance with the instructions given by the dentist in charge of treatment and only use any accessories and elastic bands supplied.

The device must only be used on healthy skin; in the event of redness of the parts in contact with the device, temporarily suspend its use.

The device must not be used during sporting activities, complex manual work and any dangerous situations resulting from moving objects in general. Do not drive while wearing the device.

In case of breakage, damage or other fault, do not use the device and contact your dentist.

If the leather pads in contact with the skin show wear or deterioration, contact your dentist.

In the event of a serious incident occurring in connection with the

device, immediately inform the manufacturer and the competent authority of the area in which the patient is based.

SETTING UP

The device is a product for orthodontic therapies that can only be used and set up by qualified personnel (dentist).

Indications on how to set up the device correctly:

- Place the chin rest (A) on the patient's chin, making sure that it does not press on the lower teeth.
- Move the forehead rest (C), positioning it over the patient's eyebrows, secure it in the desired position with the fixing screws on the sides by means of the hex key supplied.
- Adjust the height of the grill (E) with the adjustment screws (D), located on the back, using the hex key supplied.
- Connect the elastic bands to the brackets or wire hooks and pull them forward to hook them to the corresponding supports (F) positioned on the grill (E).

Before giving the device to the patient, put it in a specific container or case.

Do not force the tightening of the screws to prevent them from stripping.

The dentist is responsible for using suitable brackets or wire hooks. These devices must bear the CE marking.

Once being used on the patient, the device requires periodic checks carried out by the dentist who will evaluate the safety of the device installed on the patient according to the tensile force of the elastic bands applied.

STORAGE, CLEANING AND MAINTENANCE

When not in use, place the device in a dry place, away from heat sources and protect it from accidental impacts. It is recommended to store it in a suitable container or case.

Do not stack objects on the device.

Before each use, clean the device using a damp cloth, paying particular attention to the parts in contact with the skin.

DISPOSAL

At the end of the useful life, dispose of the non-disassembled device according to the regulations in force in the respective area.

KEY

SYMBOL	DESCRIPTION	SYMBOL	DESCRIPTION
	Manufacturer		Please read the instructions for use
	Item code		Read the warnings
	Batch number		Device compliant with MDD 93/42/EEC and subsequent amendments
	Date of manufacture		Caution: Federal law restricts this device to sale by or on the order of a dentist

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