RELIANCE[®] METAL PRIMER

INSTRUCTIONS FOR USE

RELIANCE[®]

ORTHODONTIC
PRODUCTS

RELIANCEORTHODONTICS.COM

V12.3.20 
2797

RELIANCE® METAL PRIMER

EN ENGLISH TRANSLATION

INDICATIONS FOR USE: Reliance Metal Primer is intended for use as a conditioner of metal surfaces for bonding with an adhesive.

INSTRUCTIONS FOR USE:

1. Prophy and sandblast the metal surface. Rinse and dry thoroughly.
2. Apply 1 coat of Reliance Metal Primer on the metal surface area to be bonded. Wait 30 seconds.
3. Paint a thin coat of sealant on the conditioned metal surface. Apply the paste to the bracket base and position on the tooth. Light cure

Special Safety Notice: This device Brush applicators are single use devices. It is required that a new brush applicator is used for each application.

 **Flammable**

 **PRECAUTIONS:** Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes.

 **TEMPERATURE LIMITATION:** Store at room temperature (15-30°C).

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontics Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from use of or suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

RX ONLY: U.S. Federal law restricts this device to sale by or on the order of a dental professional.

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