

# Instructions for Sunset Strip

- For the safety of user and patient, it is highly recommended that only qualified dentists should work with the Sunset Strip.
- Sunset Strips are designed for several uses and are autoclavable.
- The strips are designed to work manually in the interproximal gap.
- Avoid touching soft tissue with the strips.
- Sterilize the Sunset strip before use.

## Sterilization instruction

Disinfect and sterilize the strips before every use.

### **Cleaning**

- Clean the strips by dipping them in disinfection solution that contains a corrosion inhibitor for a minimum of 10 minutes.
- If needed - use brush to clean thoroughly.
- Rinse several times to remove all the disinfection solution.

### **Cleaning with Ultrasound**

- Strips may be ultrasonically cleaned.
- A cycle of 5 minutes is recommended, using a general purpose cleaner.
- Rinse several times.

### **Sterilizing Sunset Strip - autoclave**

- Place the strip in sterilization bags.
- Autoclave the strip for a minimum of 10minutes at a temperature of 135°C .

Following sterilizing or cleaning methods, dry the strips and store in a clean and moisture free environment.

### **Note:**

- Use sterilizing and cleaning devices and materials according to the manufacturers recommended procedure.
- It is the responsibility of the user to ensure that sterilization is effective.