

## 1 One step system

No need to apply primer or adhesive on the tooth, as it is already incorporated into the paste.

A simple etching of the enamel followed by drying is enough to prepare the bracket or buccal tube for bonding.



**1**  
Clean tooth surfaces with pumice and water. Rinse thoroughly with water and dry.



**2**  
Etch the tooth surfaces with GC Ortho Etching Gel for 30 sec., rinse and dry. Porcelain-, metal-, composite-restored teeth should be treated with the appropriate conditioner.



**3**  
Coat the bonding surface of the bracket with GC Ortho Connect paste.



**4**  
Remove the excess adhesive using an explorer or scaler.



**5**  
Cure each bracket for 10 seconds mesially and distally with an LED curing light.