

ENGLISH Instructions For Use
SmartClip™ Appliance Wire Disengagement Instrument Instructions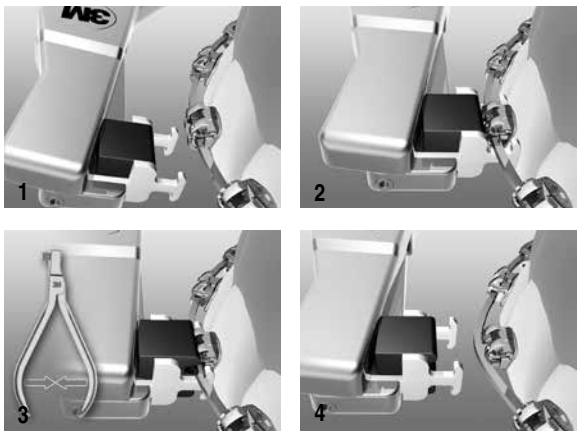

Disengagement


1. Approach archwire with disengagement instrument from occlusal or gingival side. (Figure 1)
2. Place instrument hooks under archwire keeping the bracket between the instrument hooks. (Figure 2) Make sure the instrument hooks are not under the bracket extensions holding the clips to avoid debonding or breakage. The instrument hooks are to be mesial and distal of the bracket only.
3. Squeeze handles. (Figure 3)

Note: Repeat procedure for each bracket.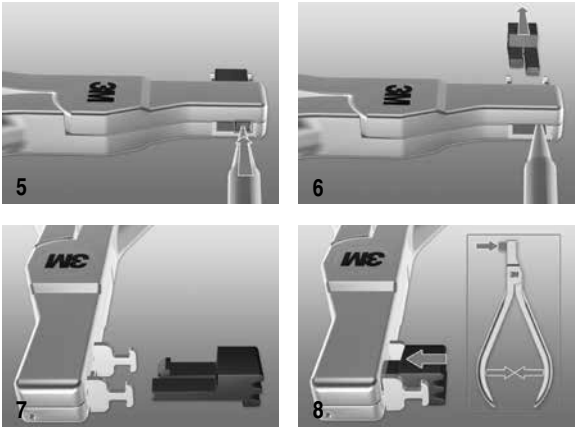

Slide Replacement

1. Using a pointed object, approach tab.
2. Push tab. (Figure 5)
3. Remove slide. (Figure 6)
4. Obtain new slide. Note: Ensure proper orientation of slide. (Figure 7)
5. Push slide while holding handles in closed position. (Figure 8)

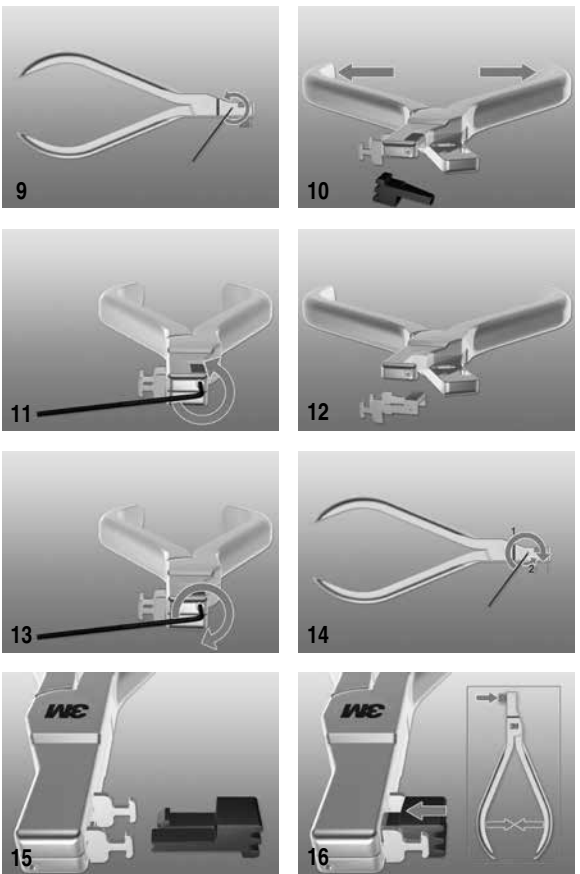Hook Replacement

1. Insert allen wrench* on first setscrew (Figure 9) and rotate counter-clockwise until handles open easily. (Figures 9 & 10)
2. Insert allen wrench on top setscrew and rotate counter-clockwise until hooks are free. (Figures 11 & 12)
3. Obtain new hooks. Note: Ensure proper orientation of hooks. (Figure 12)
4. Tighten top setscrew until first resistance is felt. (Figure 13)
5. Close plier handles. Insert allen wrench and tighten first setscrew clockwise until first resistance is felt and then rotate counter-clockwise 1/4 turn. (Figure 14)
6. Obtain new slide. Note: Ensure proper orientation of slide. (Figure 15)
7. Push slide while holding handles in closed position. (Figure 16)

*Included with Hook Insert Replacement Kit (P/N 804-162)

Intended Use:

This instrument is intended for use by an orthodontic professional to remove the archwire from the bracket clips.

WARNINGS:

Use instruments for indicated use only. Failure to do so may result in the breakage of the instrument and injury to patient or orthodontic professional. Safety glasses recommended for patient and orthodontic staff when using these instruments.

Stainless steel contains nickel and chromium. If an allergic reaction occurs, direct patient to consult a physician.

Disinfection Information: Please refer to literature 011-650, "Reprocessing Instructions for Reusable Non-Sterile Devices".

Warranty Information

3M Unitek warrants that this instrument will be free from defects in materials, or workmanship for a period of 1 year from the date of purchase. Failure to follow recommended use, maintenance and sterilization procedures will void all warranty claims.

Recommended maintenance includes cutter tip/blade re-sharpening (if applicable) and reconditioning (cleaning, polishing, adjusting screw as necessary) every six months. 3M Unitek's sole obligation and customer's sole remedy in the event of a claimed defect shall be limited to, at 3M Unitek's option, repair, replacement of merchandise, or refund of the

purchase price. ALL IMPLIED WARRANTIES, INCLUDING IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR ANY PARTICULAR PURPOSE OR USE ARE DISCLAIMED.

Limitation of Liability

Except where prohibited by law, 3M Unitek will not be liable for any loss or damages arising from this Product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

All instruments should be sent to the 3M Unitek Authorized Service Center at the address below for evaluation to determine credit, replacement, or repair. The Authorized Service Center will determine, upon receipt of instrument, if the instrument is to be covered by warranty. Materials, workmanship, and corrosion resistance are the only items covered by this warranty. Misuse or abuse will void warranty. Customer will be contacted regarding repair charges for out of warranty instruments.

3M Unitek – Authorized Service Center

Precision Plier Service
14175 Telephone Avenue, Suite D
Chino Hills, CA 91710-5762
Phone Number: 1-800-229-9060 USA, Canada, 1-909-590-2085.

References

ISO 17664:2004(E). Sterilization of medical devices—Information to be provided by the manufacturer for the processing of resterilizable medical devices.

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