TOOTH FOAM

CREATING A TEMPORARY TOOTH ON A STONE MODEL
USING PERFECT A SMILE PONTIC PUTTY

INSTRUCTIONS FOR USE

RELIANCE®

ORTHODONTIC
PRODUCTS

RELIANCEORTHODONTICS.COM

V1.11.20

TOOTH FOAM

CREATING A TEMPORARY TOOTH ON A STONE MODEL
USING PERFECT A SMILE PONTIC PUTTY


ENGLISH TRANSLATION

Perfect A Smile® Pontic Putty is indicated for use as a light cure bracket adhesive. It can also be used for the below procedure.

INSTRUCTIONS FOR USE:

Dispense approximately ½ gram of Perfect A Smile Pontic Putty.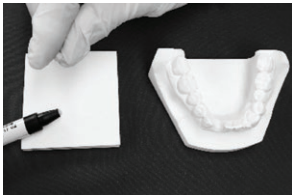

Place roll of composite on gingival surface of the missing tooth socket and lightly depress ball of composite between thumb and index. Precise shaping of the incisal, mesial and distal can be done with an instrument.

Light Cure for 30 seconds from 2 angles.

IMPORTANT!

- For a single pontic tooth on the model, create the incisal edge slightly shorter (approx. 0.5mm) than the adjacent teeth. This compensates for the thickness of the aligner material on the facial surface of the adjacent teeth once the aligner socket is painted.
- Fractured or worn teeth can be repaired or altered in shape with Perfect A Smile Pontic Putty.
- For multiple or adjacent teeth, (for an exact match), form the pontic tooth on the model to the exact incisal height of adjacent teeth.

Please Note: Fabricate a shade guide by placing 3 coats of each shade on a wax covered mixing pad, light curing each for 10 seconds between coats. The final polymerized template can then be lifted off the mixing pad and held up to the tooth for matching purposes.

⚠ CAUTION: Use of UV, halogen and LED curing lights can be associated with eye damage. The patient, doctor and the assistant should wear properly designed protective eyewear when using these light curing devices.

⚠ PRECAUTIONS: Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. If accidentally instilled into eye, flush with copious amounts of water and seek medical attention immediately. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes.

15°C 30°C **TEMPERATURE LIMITATION:** Store at room temperature (15-30°C).

TO REORDER: REF# LBOPS

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

RX ONLY: U.S. Federal law restricts this device to sale by or on the order of a dental professional.

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