GC MI Paste Plus®

TOPICAL CREME WITH CALCIUM, PHOSPHATE AND FLUORIDE
For use by or under the supervision of a dental professional in the recommended indications.

DESCRIPCIÓN
GC MI Paste Plus is a water based creme containing RECALDENT™ * with incorporated fluoride (CPP-ACPF: Casenin Phosphopéptide-Amorphous Calcium Phosphate fluoride). The level of fluoride is 0.2% w/w (900ppm), which approximates that in adult strength toothpastes. When CPP-ACPF is applied in the oral environment, it will bind to biofilms, plaques in hard tissue, hydroxyapatite and soft tissue localising bio-available calcium, phosphate and fluoride.

IMPORTANT:
Saliva will enhance the effectiveness of CPP-ACPF and the flavor will help stimulate saliva flow. The longer CPP-ACPF and saliva are maintained in the mouth, the more effective the result.

A topical paste containing bio-available calcium, phosphate and fluoride

- Provides extra protection for teeth.
- Helps neutralize acid challenges from adogenic bacteria in plaque.
- Helps neutralize acid challenges from other internal and external acid sources.

CONTRAINDICATIONS
Casenin phosphopéptides are derived from milk casain. Do not use this material on patients with a proven or suspected milk protein allergy and/or with a sensitivity or allergy to benzatezo preservatives.

DIRECTIONS FOR USE INDICATIONS

- Following in-office bleaching procedures.
- After ultrasonic, hand scaling or root planing.
- Following professional tooth cleaning (P.T.C.).
- Hypersensitivity prevention and control.
- As an alternative means of applying fluoride topically in children aged 6 and above.
- During orthodontic treatment.
- For high risk caries patients.
- To provide a topical coating for patients suffering from erosion, xerostomia or Sjögrens syndrome.

I. For special needs adult patients.

In-office application:

- Custom tray application**
 - Before use rinse the custom tray thoroughly under running water.
 - Extrude a generous layer of GC MI Paste Plus into the tray and apply to the upper and / or lower teeth.
 - Remove the tray.
 - Leave the tray undisturbed in the mouth for a minimum of 3 minutes.
- Remove the tray.
- Instruct the patient to use the tongue to spread the remaining GC MI Paste Plus throughout the mouth. Instruct the patient to retain for as long as possible (another 1-2 minutes) avoiding expectoration and delaying swallowing. The longer GC MI Paste Plus and saliva are maintained in the mouth, the more effective the result.
- Ask the patient to expectorate thoroughly and if possible avoid rinsing. Any GC MI Paste Plus remaining on the surface can be left to gradually dissipate. Advise the patient not to eat or drink for 30 minutes following application.
- Any residual GC MI Paste Plus in the tray should be rinsed or brushed off under running water immediately after use.

- For special needs adult patients.**
 - If necessary, remove any excess saliva on the tooth surface with a cotton roll or paper. However, it is NOT necessary to dry the teeth with compressed air.
 - Apply GC MI Paste Plus to the tooth surfaces using an application swab, gloved finger and/or Interproximal areas using an Interproximal Tooth Cleaning Brush.
 - Leave GC MI Paste Plus undisturbed for a minimum of 3 minutes.
 - Then instruct the patient to use the tongue to spread the remaining GC MI Paste Plus throughout the mouth. Hold in the mouth for as long as possible (a further 1-2 minutes) avoiding expectoration (spitting out) and delaying swallowing. The longer GC MI Paste Plus and saliva are maintained in the mouth, the more effective the result.
 - Ask the patient to expectorate thoroughly and if possible avoid rinsing. Any GC MI Paste Plus remaining on the surface can be left to gradually dissipate. Advise the patient not to eat or drink for 30 minutes following application.

II. Day time application after tooth brushing as recommended by a dental professional
Apply a sufficient amount of GC MI Paste Plus to upper and lower teeth. A pea size amount for each arch is the minimum required. The material should be applied to the tooth surfaces using a clean dry finger or cotton tip. For difficult areas (between the teeth) use an Interproximal Tooth Cleaning Brush or dental floss coated with GC MI Paste Plus.

- Apply a sufficient amount of GC MI Paste Plus to upper and lower teeth. A pea size amount for each arch is the minimum required. The material should be applied to the tooth surfaces using a clean dry finger or cotton tip. For difficult areas (between the teeth) use an Interproximal Tooth Cleaning Brush or dental floss coated with GC MI Paste Plus.
- Leave GC MI Paste Plus on the teeth undisturbed for a minimum of 3 minutes.
- Then use your tongue to spread the remaining GC MI Paste Plus throughout the mouth. Hold in the mouth for as long as possible (a further 1-2 minutes) avoiding expectoration (spitting out) and delaying swallowing. The longer GC MI Paste Plus and saliva are maintained in the mouth, the more effective the result.
- Expectorate thoroughly and if possible avoid rinsing. Any GC MI Paste Plus remaining on the surface can be left to gradually dissipate. Do not eat or drink for 30 minutes following application.

III. Night time application after tooth brushing as recommended by a dental professional

The overnight use is especially recommended for high-risk adult patients, but not recommended for children under the age of 6 years.

- Apply a sufficient amount of GC MI Paste Plus to upper and lower teeth. A pea size amount for each arch is the minimum required. The material should be applied to the tooth surfaces using a clean dry finger or cotton tip. For difficult areas (between the teeth) use an Interproximal Tooth Cleaning Brush or dental floss coated with GC MI Paste Plus.
- Leave GC MI Paste Plus on the teeth undisturbed for a minimum of 3 minutes.
- Then use your tongue to spread the remaining GC MI Paste Plus throughout the mouth. Hold in the mouth for as long as possible (a further 1-2 minutes) avoiding expectoration (spitting out) and delaying swallowing. The longer GC MI Paste Plus and saliva are maintained in the mouth, the more effective the result.
- Expectorate thoroughly and if possible avoid rinsing. Any GC MI Paste Plus remaining on the surface can be left to gradually dissipate. Do not eat or drink for 30 minutes following application.

PACKAGES
Tube containing 40g (35mL).
1 Assorted Package
10 x 40g (2 each of Strawberry, Melon, Vanilla, Mint, Tutti-Frutti)
2 Single Flavor Package
10 x 40g in each flavor (Vanilla, Strawberry, Melon, Tutti-Frutti, Mint)

NOTE
GC MI Paste Plus with its unique CPP-ACP formulation was developed at The School of Dental Science, The University of Melbourne, Victoria, Australia.
*RECALDENT is a trademark used under licence. CPP-ACP technology has related patents or patents pending in Australia, NZ, Europe, Canada and USA.

- CAUTION**
- Always replace the cap after use and remove any residual paste or moisture around the tube neck or inside the cap.
 - It is not recommended to use GC MI Paste Plus on patients under the age of 6.
 - The overnight at home application is not recommended for children under 12.
 - In case of contact with eyes, flush immediately with water and seek medical attention.
 - In case of spillage on clothes, wash off with water.
 - If any angioedema symptoms are experienced, this may indicate sensitivity or allergy to the benzatezo preservatives, or to some other component of the product such as a flavouring agent. In this event, discontinue the use of the product and refer to a physician. Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the Safety Data Sheets available at: http://www.gceurope.com/

It can also be obtained from your supplier.

Last revised : 11/2016

3731116

NOTE
GC MI Paste Plus with its unique CPP-ACP formulation was developed at The School of Dental Science, The University of Melbourne, Victoria, Australia.
*RECALDENT is a trademark used under licence. CPP-ACP technology has related patents or patents pending in Australia, NZ, Europe, Canada and USA.

- Always replace the cap after use and remove any residual paste or moisture around the tube neck or inside the cap.
- It is not recommended to use GC MI Paste Plus on patients under the age of 6.
- The overnight at home application is not recommended for children under 12.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- In case of spillage on clothes, wash off with water.
- If any angioedema symptoms are experienced, this may indicate sensitivity or allergy to the benzatezo preservatives, or to some other component of the product such as a flavouring agent. In this event, discontinue the use of the product and refer to a physician. Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the Safety Data Sheets available at: http://www.gceurope.com/

It can also be obtained from your supplier.

Last revised : 11/2016

3731116

NOTE
GC MI Paste Plus with its unique CPP-ACP formulation was developed at The School of Dental Science, The University of Melbourne, Victoria, Australia.
*RECALDENT is a trademark used under licence. CPP-ACP technology has related patents or patents pending in Australia, NZ, Europe, Canada and USA.

CAUTION

- Always replace the cap after use and remove any residual paste or moisture around the tube neck or inside the cap.
- It is not recommended to use GC MI Paste Plus on patients under the age of 6.
- The overnight at home application is not recommended for children under 12.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- In case of spillage on clothes, wash off with water.
- If any angioedema symptoms are experienced, this may indicate sensitivity or allergy to the benzatezo preservatives, or to some other component of the product such as a flavouring agent. In this event, discontinue the use of the product and refer to a physician. Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the Safety Data Sheets available at: http://www.gceurope.com/

It can also be obtained from your supplier.

3731116

NOTE
GC MI Paste Plus with its unique CPP-ACP formulation was developed at The School of Dental Science, The University of Melbourne, Victoria, Australia.
*RECALDENT is a trademark used under licence. CPP-ACP technology has related patents or patents pending in Australia, NZ, Europe, Canada and USA.

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