# OptiBond™

## eXTRa Universal

Two-Component Universal Adhesive

### Instructions For Use

**Kerr Corporation**  
1717 W. Collins Avenue Orange, CA 92867 USA  
1-800-KERR-123 | kerrdental.com

**EC REP** Kerr Italia S.r.l. Via Passanti, 332  
I-84018 Scafati (SA) Italy  
+39-081-850-8311 | kerrdental.com

Distribution Facility:  
Kerr Australia Pty. Ltd.  
Unit 10, 112-118 Talavera Road  
North Ryde, NSW, 2113 Australia

DMC3235\_Rev.01

### SYMBOLS DESCRIPTION

	Warning or Caution
	Consult Instructions for Use
	Temperature Storage Limitations: Low 2°C/36°F and High 8°C/46°F
	Manufacturer
	Use-by-Date
	Do not re-use
	Manufacturing lot number
	European Authorized Representative
	Part Number

en - ENGLISH

### OptiBond™ eXTRa Universal

Two-Component Universal Adhesive

### DEVICE DESCRIPTION

OptiBond eXTRa Universal is a two-component universal adhesive system including a PRIMER and an ADHESIVE. The OptiBond eXTRa Universal PRIMER provides effective etching to enamel and dentin without the need for a separate phosphoric acid etch, thus simplifying the bonding procedure. The OptiBond eXTRa Universal ADHESIVE is 15% filled with 0.4 micron barium glass to help reinforce bond strength. The material's chemistry allows for compatibility with all self-cure or dual-cure resin cements and core build-up materials. Dentists can therefore utilize OptiBond eXTRa Universal for direct and indirect procedures without the need for a secondary bonding system or auxiliary product.

### INDICATIONS FOR USE

**Note: OptiBond eXTRa Universal is compatible with all composite, self-cure, dual-cure and light-cure resin cement and core buildup materials, and does not require the use of any Dual Cure Activator.**

### Direct Bonding Applications

- Light-cured composite and compomer restorations.
- Composite/ceramic/metal repairs.
- Cavity sealing for amalgam restorations.
- Sealing of hypersensitive and/or exposed root surfaces.
- Core build-ups (self-cured, light-cured, or dual-cured).

### Indirect Bonding Applications

- Veneers.
- Self-cure, dual-cure, light-cure resin cements and core buildup materials.
- Porcelain, ceramic (including zirconia-based, lithium disilicate-based and alumina-based), composite, and metal-based (including precious and non-precious metal inlays, onlays, crowns, bridges).
- Endodontic posts.
- Cavity sealing as a pretreatment for indirect restorations.

### CONTRAINDICATIONS

1. Patients who have a history of severe allergic reactions to methacrylate resins.
2. Direct pulp capping.

### WARNINGS

Do not use on patients with known methacrylate allergies. Uncured methacrylate resin may cause contact dermatitis and damage to the pulp. OptiBond eXTRa Universal is a corrosive liquid as it contains acetone and ethanol and is flammable. Keep away from sources of ignition. Avoid contact with skin, eyes, and soft tissue. In case of contact with skin or eyes, flush immediately with water. Get medical attention for eyes. Do not take internally.

**Unidose™ Delivery** is for single patient use only, to prevent cross contamination between patients.

### PRECAUTIONS

Refrigerate upon receipt, between 2° - 8°C (36° - 46°F). OptiBond eXTRa Universal should be tightly closed immediately after use. OptiBond eXTRa Universal adhesive is a light-cured material. Once the material is placed in the mixing well, use immediately or protect from ambient light. Use one applicator tip for OptiBond eXTRa Universal Primer and a separate applicator for OptiBond eXTRa Universal Adhesive. Product contains sodium hexafluorosilicate.

### DELIVERY OPTIONS

OptiBond eXTRa Universal is available in both bottle and Unidose™ Delivery systems:

- **Bottle Delivery:** Shake OptiBond eXTRa Universal ADHESIVE bottle briefly (5 seconds) prior to dispensing. Dispense 1 - 2 drops of OptiBond eXTRa Universal PRIMER and OptiBond eXTRa Universal ADHESIVE into two separate clean wells. Replace cap immediately after dispensing. Wet applicator tip completely.
- **Unidose™ Delivery:** Shake OptiBond eXTRa Universal ADHESIVE Unidose briefly (5 seconds) prior to dispensing. Open the Unidose device and insert the applicator brush into the container to saturate the tip. Discard Unidose device after use.

### INSTRUCTONS FOR USE

#### Step-by-Step Instructions

#### Direct Restorations

#### Bonding Composites/Compomers to Dentin/Enamel

1. Prepare cavity. Use a rubber dam to isolate the oral cavity (Kerr recommends Kerr OptiDam™). Pumice clean unprepared tooth structure with a fluoride-free cleaning paste; rinse thoroughly with water spray and air dry.
2. Apply OptiBond eXTRa Universal PRIMER to the enamel/dentin surface using the

disposable applicator brush. Scrub the dentin and enamel surface with a brushing motion for 20 seconds.

- Air thin for 5 seconds with medium air pressure.
- Shake OptiBond eXTRa Universal ADHESIVE bottle briefly. Apply OptiBond eXTRa Universal ADHESIVE to the enamel/dentin surface with light brushing motion for 15 seconds.
- Air thin for 5 seconds.
- Light-cure for 10 seconds\*.
- Apply composite per manufacturer's instructions for use.

Note: OptiBond eXTRa Universal produces high enamel bond strengths without the use of phosphoric acid. However, phosphoric acid can be used on enamel and/or dentin if an etch and rinse (total etch) or selective enamel etch technique is desired. After step 1, etch Enamel or Dentin cavosurfaces with a phosphoric acid Gel Etchant (e.g. Kerr's 37.5% Gel Etchant) for 15 seconds, then rinse thoroughly ensuring that all etchant has been removed. Proceed to step 2.

## INDIRECT RESTORATIONS

### I. Restoration Surface Preparation

Prepare the internal surface of the restoration per manufacturer's instructions. Here are some general recommendations:

**Lithium Disilicate, Ceramic/Porcelain Restorations:** Air-particle-abrade the surface with 50µm alumina with a pressure of about 30 psi (0.2 MPa) and etch the restoration with hydrofluoric acid for 1 minute. Apply a coat of OptiBond eXTRa Universal ADHESIVE† on the internal surface of the restoration, air thin with gentle air first and then strong air to avoid pooling of OptiBond eXTRa Universal ADHESIVE, and light-cure for 10 seconds\* (light-curing of OptiBond eXTRa Universal ADHESIVE is optional if NX3 resin cement is used for cementation).

† OptiBond eXTRa ADHESIVE can be used as silane replacement.

**Metal-based, Zirconia-based, Alumina-based and Composite Restorations:** Air-particle-abrade the internal surface with 50µm alumina with a pressure of about 60 psi (0.4 MPa) for metal-based, zirconia-based and alumina-based restorations, or a pressure of about 15 psi (0.1 MPa) for composite-based restorations. Apply a coat of OptiBond eXTRa Universal ADHESIVE on the internal surface of the restoration, air thin with gentle air first and then strong air to avoid pooling of OptiBond eXTRa Universal ADHESIVE, and light-cure for 10 seconds\* (light-curing of OptiBond eXTRa Universal ADHESIVE is optional if NX3 resin cement is used for cementation).

### II. Tooth Surface Preparation

- Clean the preparation. Wash thoroughly with water spray and air dry.
- Apply OptiBond eXTRa Universal PRIMER to the enamel/dentin surface using the disposable applicator brush. Scrub the surface with a brushing motion for 20 seconds.
- Air thin for 5 seconds with medium air pressure.
- Shake OptiBond eXTRa Universal ADHESIVE bottle briefly. Apply OptiBond eXTRa Universal ADHESIVE to the enamel/dentin surface with light brushing motion for 15 seconds.
- Air thin with gentle air first and then strong air for at least 5 seconds to avoid pooling of adhesive before light curing. For post cementation, excess OptiBond eXTRa Universal ADHESIVE can be removed with a dry applicator brush or absorbent paper point.
- Light cure for 10 seconds\* (Light-curing of OptiBond eXTRa Universal ADHESIVE is optional if NX3 resin cement is used for cementation. When NX3 resin cement is used for cementation and light-curing of OptiBond eXTRa Universal ADHESIVE is omitted, OptiBond eXTRa Universal ADHESIVE needs to be air thinned with maximum air pressure for 15 seconds prior to NX3 application).

Note: OptiBond eXTRa Universal produces high enamel bond strengths without the use of phosphoric acid. However, phosphoric acid can be used on enamel and/or dentin if an etch and rinse (total etch) or selective enamel etch technique is desired. After step 1, etch Enamel or Dentin cavosurfaces with a phosphoric acid gel etchant (e.g. Kerr's 37.5% Gel Etchant) for 15 seconds, then rinse thoroughly ensuring that all etchant has been removed. Proceed with step 2.

### III. Cement Application and Seating of Restoration

See indication-specific information in sections A - C below. Veneers, crowns, bridges,

inlays, onlays and metal-based restorations (used in conjunction with a luting cement).

**Note: OptiBond eXTRa Universal is compatible with all composite, self-cure, dual-cure and light-cure resin cement and core buildup materials, and does not require the use of any Dual Cure Activator.**

**A. Veneers** - Apply NX3 (or a resin cement of your choice) to the veneer, seat the veneer, and cure the cement per manufacturer's instructions for use.

**B. Crowns, bridges, inlays, onlays and metal-based restorations** - Apply NX3 resin cement (or a resin cement of your choice) to the restoration or preparation, seat the restoration, and cure the cement per manufacturer's instructions for use.

**C. Post Bonding and Core Buildup** - Prepare the surface of the post per manufacturer's instructions, e.g. mechanical roughening or air-abrasion. Using an applicator brush, apply OptiBond eXTRa Universal Primer in the root canal preparation and on all the cavity walls involved in the bonding procedure, then air thin. Apply OptiBond eXTRa Universal Adhesive in the root canal preparation, then air thin and light-cure for 10 seconds\* (light-curing of OptiBond eXTRa Universal ADHESIVE is optional if NX3 resin cement is used for cementation). Apply a coat of OptiBond eXTRa Universal ADHESIVE on all post surfaces, air thin and light-cure for 10 seconds\*. Using an applicator brush, apply NX3 resin cement (or a resin cement of your choice) onto the post, seat the post, vibrate the post slightly avoiding the possibility of trapped air, and then cure NX3 per the manufacturer's instructions. Proceed with core buildup with NX3 as core material (or a core material of your choice). All self-cure and dual-cure core materials can be built in bulk as light-cured OptiBond eXTRa Universal is compatible with all self-cure and dual-cure core buildup material.

### \*Recommended Cure Times when using Kerr Demetron Lights:

Optilux 501 - 10 seconds

L.E.Demetron I - 10 seconds

L.E.Demetron II - 5 seconds

Demi/Demi Plus/Demi Ultra - 5 seconds

For all other lights see manufacturer's recommendation.

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