GRADIA® PLUS MODULAR COMPOSITE SYSTEM FOR INDIRECT RESTORATIONS

For use only by a dental professional in the recommended indications.

INDICATIONS

1. Crown and Bridge work (with metal backing)
2. Metal free restorations: Jacket Crowns, Inlays, Onlays and Laminated Veneers
3. Implant superstructures
4. Reproduction of gum tissue for crown restorations

CONTRAINDICATIONS

Avoid use of this product in patients with known methacrylate monomer and methacrylate polymer allergies.

DIRECTIONS FOR USE**1. BUILDING UP PROCEDURE****A. Anterior / Posterior Veneer Crown (with metal backing)**

- a. Wax-up framework conventionally, occlusal surface should have a minimum of 1 mm thickness. Apply retention beads to the facing surface. Cast in the usual manner.
- b. Sandblast the metal frame with 50 micron aluminum oxide. Clean and dry with air gun.
- c. Immediately apply METAL PRIMER II / METAL PRIMER Z (GC) to retention areas. Allow to dry.

2. Build up Veneer

- a. Immediately apply a layer of OPAQUE over retention area and light cure for 1 minute. Repeat this process until the metal color is masked out.
- b. Apply Dentin shades*, building up to the desired thickness.

*A processing aid like MODELING LIQUID can be used.

c. Progressively thinly apply Lustre Paint (A) in 2-3 mm width around the cervical area as required and light cure for 1 minute.

d. Apply Enamel shades progressively from the incisal edge. Light cure for 1 minute.

e. Adjust contour and characterize surface with diamond and carbide burs. (At this time, scrape off all air inhibition layers on the surface.)

3. Glazing (Coating and Polishing)**3-1 Coating method**

- a. Roughen the resin surface using a carbide bur or sandblast (1.5 kg/cm²) for obtaining the mechanical adhesion. Dry and clean with air gun.
- b. Immediately apply a silane coupling agent (e.g. CERAMIC PRIMER II (GC)) to the resin surface and dry.

To prevent contamination, wash the brush with Lustre Paint Diluting Liquid before use.

3-2 Polishing method

- a. Use appropriate instruments and burs to polish and finish the prosthesis, and to confirm, remove any agent or paste for polish and finish in proper way and see the surface has proper shine.

B. Anterior or Posterior Jacket Crown (metal free)**1. Preparation of the working model**

- a. Prepare a stone master model.
- b. Apply DIE-HARDNER to the stone surface.
- c. Block out undercut with wax.
- d. Apply a layer of paraffin wax as die spacer as required.

2. Special pre-treatment

- a. Apply a metal core, apply first a thin layer of OPAQUE. Light cure for 1 minute.
- b. In case of a vital abutment tooth, apply first a thin layer of (LB)-Base OD and Pre-cure.

3. Build up of Crown

For building up the surface, see instructions BUILDING UP PROCEDURE A. (Section 2. b-e) above.

4. Glazing

See instructions BUILDINGS UP PROCEDURE A. (Section 3) above.

5. Cleaning, treating restoration for bondingRoughen the inner surface of Jacket crown using alumina sandblast (1.5 kg/cm²) for obtaining the mechanical adhesion. Then clean with air gun.**6. Cementation (Chair side)**

- a. Apply a thin layer of a silane coupling agent (e.g. CERAMIC PRIMER II (GC)) to bonding surface.
- b. Use a dental adhesive resin cement (e.g. G-CEM LinkForce (GC)).

C. Posterior Inlay**1. Preparation of the working model**

- a. Follow the instructions Preparation of the working model (B. Section 1. a-e) above.

2. Build up of Inlay

- a. Build up (LB)-Inlay TD as a dentin color in order to retain the natural tooth color. Pre-cure.

Note: In the case of a stained tooth, it is recommended to first apply OPAQUE to the cavity floor. Light cure for 1 minute.

- b. Build up (LB)-Inlay E, shape and contour. Final-cure.

3. Glazing

See instructions BUILDING UP PROCEDURE A. (Section 3) above.

II. ADDITIONAL APPLICATION AND REPAIR**A. LASBIDE**

1. Roughen the resin surface using a carbide bur or sandblast (1.5 kg/cm²). Immediately apply a silane coupling agent (e.g. CERAMIC PRIMER II (GC)) to the resin surface and dry.

B. CHAIRSIDE MASKING OF EXPOSED METAL SURFACES WHEN REPAIRING INDIRECT RESTORATIONS INTRAOРАЛЬ**1. As for PREPARATION OF RESTORATION, refer to G-Premio BOND IFU-II.**

Note: Use a rubber dam for isolation.

2. Applying GRADIA PLUS OPAQUE (GC) to the prepared metal surface only

The layer applied should have a thickness of maximum 0.2 mm.

3. If the surface is still not sufficiently masked, apply a second layer of OPAQUE and light-cure.**Irradiation time**

Curing unit	Halogen / LED (700 mW/cm ²) & High power LED (more than 1200 mW/cm ²)
OPAQUE	20 seconds

4. Repair the restoration by applying a chairside composite material (e.g. G-aenial & Essentia) in layers. Light-cure each layer according to the manufacturer's instructions of the respective product

5. Finish and polish using standard techniques.

NOTES

• Please review this Instruction for Use carefully before using the product.

After reviewing the Instructions for Use, keep them in a convenient place and refer to them whenever the product is used.

• If the product is lost, contact your distributor or supplier shown on the package.

• This product must only be used by dentists or dental technicians.

• To prevent troubles with operating or storing, the product should be used according to the instructions specified in this Instruction for Use. Do not use this product for contraindications given in this Instruction for Use.

• Assume that the crown or inlay is sufficiently thick, using the methods for preparing and curing the composite.

• Wear glasses or take other appropriate measures when using the product.

• After dispensing, use immediately and replace caps on syringe or bottle.

• Avoid separation of the coating (Lustre Paint) from the orthoprosthetic.

• Sandblast the surface of orthoprostheses with alumina particles, 50 µm diameter and use the correct curing time.

• Avoid separation of the orthoprosthetic. Use a dental adhesive resin cement to lute jacket crowns, veneers, inlays or onlays. Sandblast the inner surface of the jacket crowns, veneers, inlays or onlays using low pressure (1.5 kg/cm²).

• The oxygen inhibited layer of the resin surface to be coated should be removed before applying Lustre Paint as it may hinder the proper curing of Lustre Paint.

• For curing Lustre Paint, violet LED (effective wavelength: 390-430 nm) is required.

• Place the light source as close as possible to the resin surface. Set a 2cm platform in the curing device and place the prosthesis on it with the coated surface facing the light. Reverse the prosthesis and repeat the curing light to fully cure the undercut or area in shadow.

CAUTION

1. In rare cases, this product may cause sensitivity in some people.

If such reactions are experienced, discontinue the use of the product and consult a physician.

2. Wear eye protection and plastic or rubber gloves during operation in order to prevent contact with water and air inhibited resin layers in order to prevent possible sensitivity.

3. LUSTRE PAINT DILUTING LIQUID, SEPARATOR, DIE-HARDNER, GC METAL PRIMER II / GC METAL PRIMER Z and GC CERAMIC PRIMER II are highly flammable liquid and vapour. Do not use near naked flame.

4. When using sources of ignition. Keep away from direct sunlight.

5. When using sources of ignition. The veneer surface, use a dust collector and wear a dust mask to avoid inhaling dust.

6. Place the light source as close as possible to the resin surface. Set a 2cm platform in the curing device and place the prosthesis on it with the coated surface facing the light. Reverse the prosthesis and repeat the curing light to fully cure the undercut or area in shadow.

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