# HANDY SURGERY

---

*Soft Line*

# HANDY SURGERY

Introduzione <i>Introduction</i>	2
Morpheus <i>Morpheus</i>	4
Prophy 900 <i>Prophy 900</i>	14
Easytray <i>Easytray</i>	18
Aspirol <i>Aspirol</i>	23
Aspserve <i>Aspserve</i>	28
Quadriled <i>Quadriled</i>	32
Easymove <i>Easymove</i>	36

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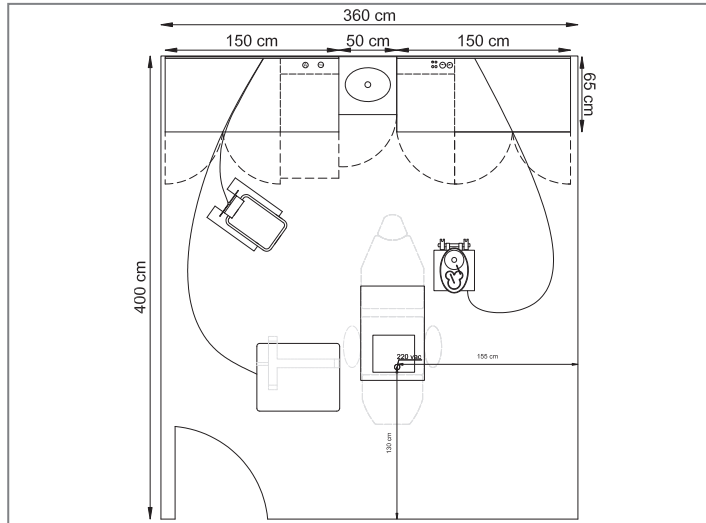
*Specialised in the production and supply of equipment for dental practices, Tecnomed Italia has created Handy Surgery Soft Line, a professional and high-quality line able to satisfy the different operational needs of professionals in the dental sector.*

*Soft Line ensures a functional and welcoming environment for patients, and a high-visual impact that contributes to improving the quality of the dental practice experience.*

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
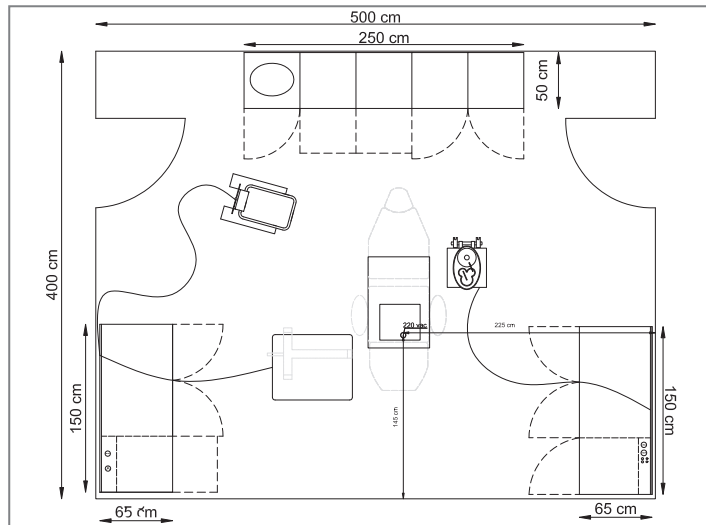
*Handy Surgery Soft Line is a versatile and ergonomic line that is entirely designed on castors: all the elements, including the dental equipment, can be moved around the practice according to individual requirements. With this new concept, creating the perfect practice that adapts to all kinds of spaces and needs becomes much easier thanks to the modular components. The excellent quality materials, coupled with the linear and elegant appearance of all the elements, contribute to creating a professional environment where meticulous attention has been paid to detail.*

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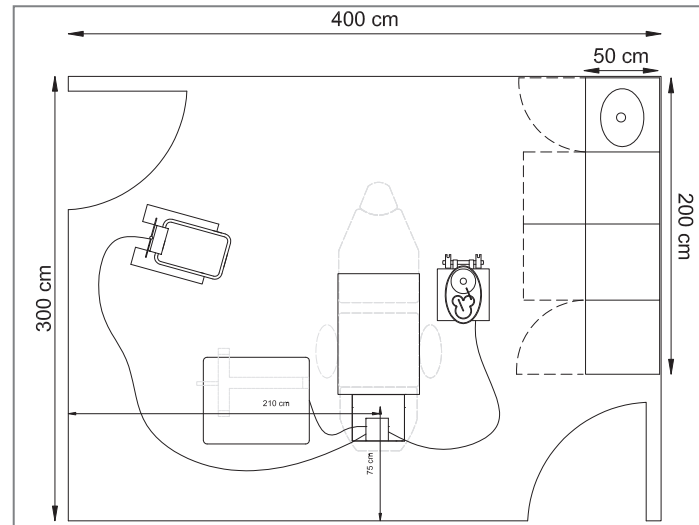*Morpheus is not just a simple dental chair: thanks to its four motors, it can perform multiple movements, quickly converting into a real operating table. Any unnecessary encumbrances have been eliminated to allow for smooth doctor-patient interaction, thus facilitating the usual surgical procedures.*

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*Two joysticks within easy reach of your feet let you manage the same movements as the push button panel.*

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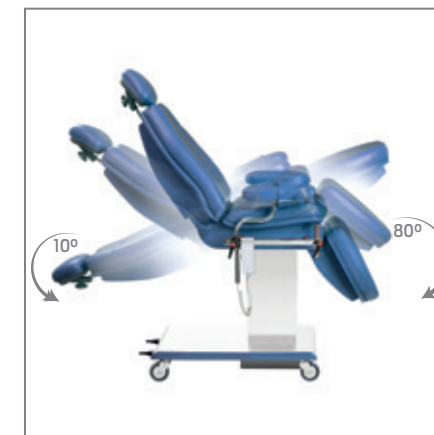
*Synchronised controls let you recline the chair from sitting to horizontal position.*

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*An extremely stable base lets you raise the chair from 55 to 95 cm of 200kg.*

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*Leg supports and a tilting seat let you place patients in the zero gravity and Trendelenburg positions.*

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*Tecnomed uses top quality materials to manufacture Morpheus: the structure of white enamelled stainless steel, perfectly smooth and hollow-free, makes the chair easy to sanitize and durable over time.*

*The high-quality antimicrobial artificial leather upholstery has all the features essential in surgery:*

- *Comfort*
- *Easy cleaning*
- *Water repellency*
- *Lightness*
- *Wear resistance*
- *Fire-proof*

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*To adapt even more effectively to different specialist applications, the Morpheus chair can be fitted with a wide range of accessories, including gynaecological thigh supports, a headrest with hole for aesthetic surgery and physiotherapy, an IV drip stand and an arm rest for blood tests. Each accessory is designed to integrate effectively with the chair without getting in the way of your movements, and to offer patients maximum comfort even throughout long treatments.*

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*Fully adjustable anatomic headrest with double-jointed height adjustment.*

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*The colour of Morpheus upholstery is available in an infinity of choices: among the wide range of colours available, you will certainly find the one that best suits your practice.*

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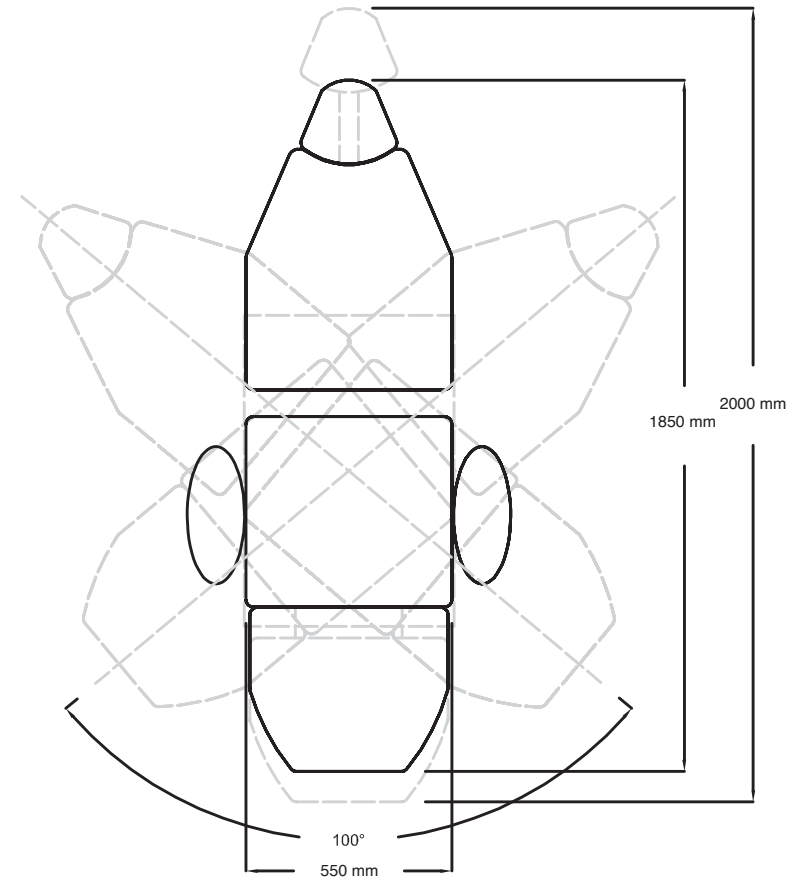
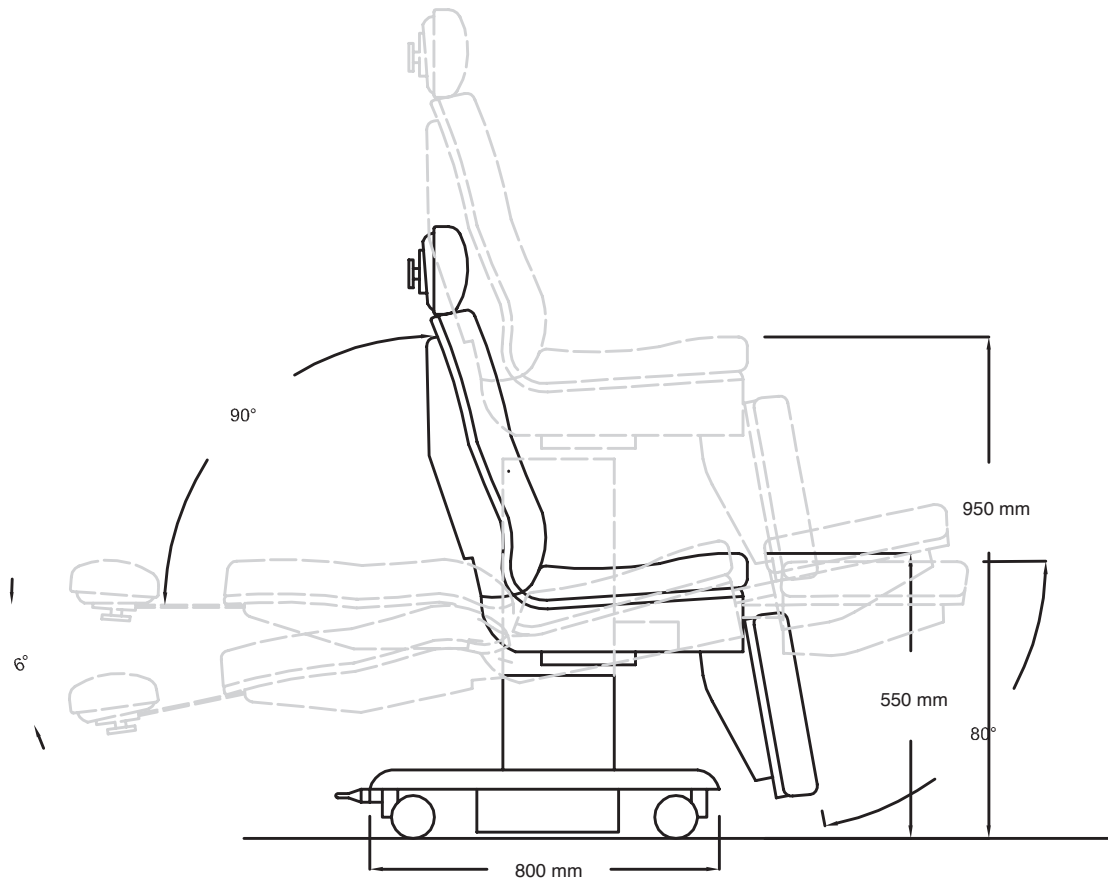
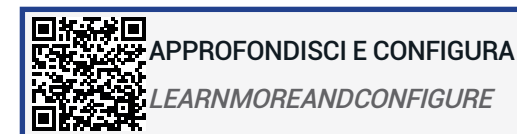
*An appropriate stool is an important investment for the comfort and health of the dental professional. Beauty and Round professional dentists' chairs are designed for complete comfort even after many hours of work. Their ergonomic structure helps you to maintain correct posture while distributing weight in a uniform manner. Their antibacterial and water-repellent imitation leather upholstery makes these chairs extremely easy to disinfect. The range offers a choice of thirty different colours for a perfect match with the finish and materials of your dental chair.*

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*Top quality imitation leather upholstery ensures a soft, anatomical seating position with uniform weight distribution and no painful compressions.*

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*Beauty and Round chairs feature adjustable pneumatic height adjustment and backrest depth adjustment.*

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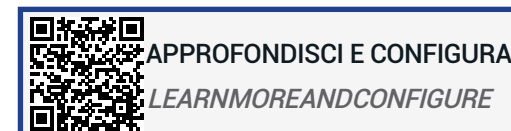
*The Round model also features backrest height and angle adjustment.*

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*Prophy 900 surgical cart has all the tools and features you need for surgery and is designed to ensure maximum operational efficiency. The setup of the available tools can be customized based on the specific clinical needs: Prophy 900 features a instrument table that can be set up for maximum five different instruments.*

*The quality of the materials used ensures utmost sturdiness and cutting-edge design: the 2cm thick polyurethane instrument table is covered with an epoxy paint resistant to disinfectants. Unlike plastics, the Prophy 900 coating does not turn yellow and does not crack.*

- **Maximum efficiency:** it guarantees smooth work processes, providing all the necessary tools.
- **Designed to ensure operational efficiency:** it is equipped with all the functions necessary to carry out the interventions.
- All **parameters** can be perfectly kept **under control** thanks to the wide LCD control panel with optional soft touch keys and progressive control pedal.
- **Indestructible:** 2 cm thick polyurethane instrument table.
- **Easy movement** thanks to the comfortable anti-static casters and the pedal shelf.

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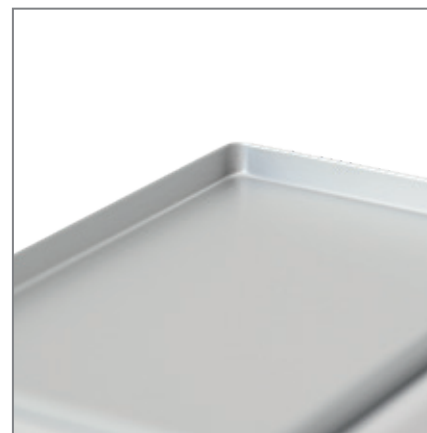
*The separate water supply guarantees total operating autonomy; there is also an independent spray adjustment for each instrument.*

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*Thanks to the 5 compartments of the instrument table, Prophy 900 can be set up with the most suitable equipment for every need.*

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*Prophy 900 features a large table top on which are inserted two tool trays, 175x280x15 mm, removable and autoclavable in order to ensure maximum hygiene, which is absolutely necessary for surgery.*

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### **LCD CONTROL PANEL**

*Prophy 900 can be equipped with an additional control panel with soft touch keys: thanks to the LCD screen, you can control all parameters.*

*The cart features a technology that automatically recognizes the selected instrument (micromotor, high speed handpiece or scaler) and allows you to set parameters and the operating mode.*

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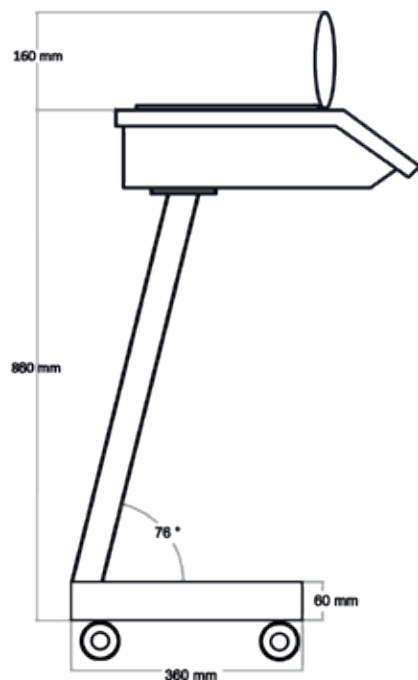
### **FEED-FEK PROGRESSIVE PEDAL**

*The Feed Fek progressive control pedal activates all instrument functions. Spray actuation can always be on by pressing the central button and activating the progressive lever. It can also be activated, if necessary, by pressing the lower lever only. It also features a button for reverse movement of the micromotor. The pedal shelf allows you to rest the pedal and move comfortably.*

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*In modern dental practices, the procedures and the techniques used to cover all types of pathologies are multiple: this involves the presence and use of various instruments, the latter being composed of handpieces, cords, pedals, etc.. All this can give rise to hygiene problems and also to confusion during surgery, along with a negative perception of the patient.*

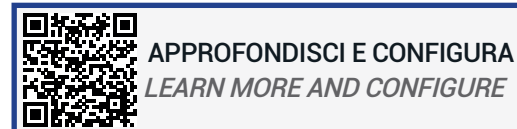
*The solution proposed by Tecnomed Italia is Easy Tray, the cart for equipment that has been designed to perfectly conceal the electrical connections and Made entirely of electro galvanized white enamelled sheet.*

*Easy Tray is versatile and modular, as it can be customized to infinity: starting from the "Nano" version, with three sterilizable stainless steel trays, the cart can be extended (thanks to the special extension) and configured according to your needs with different accessories: tray of two different sizes, front arm for instrument supports of your choice, articulated support for monitors, IV drip stand and much more.*

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*Easy Tray Nano order code: OF1003BIN.01*  
*Height 875 mm. Left and right handles. 4 swivel wheels (castors). 3 dental instrument tray 415x300x20 mm.*

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*Thanks to the new supports designed by Tecnomed Italia, Easy Tray (white model) can have a monitor or an All-in-one PC, camera, wireless medical keyboard and mouse incorporated: all the technologies to communicate with your practice network or with your patients are now together in a single trolley, transportable everywhere.*

*It is ideal to house complementary surgical equipment such as scalers, electrosurgery units, piezoelectric surgical devices, implant micromotors, endodontic micromotors, lasers, surgical Stryker equipment, whitening devices, multi-parameter monitors.*

*A single, integrated, height-adjustable infusion set stand can hold up to three bags of saline solution to ensure easy operation.*

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*Electrified column and cable channel with five removable and disinfectable trays.*

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*In the full optional version which includes the Tecnomed Italia equipments, the controls are integrated in the Prophy 900 foot control so that all instruments can be operated from the one pedal.*

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*Instrument holders autoclavable at 121°C on the front arm*

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*Articulates VESA 75/100 support for monitor or PC All in One and intraoral camera support*

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
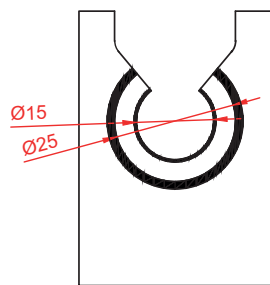
*The holders are mainly designed for the most popular equipment.  
Upon request, Tecnomed Italia manufactures tailored holders.*

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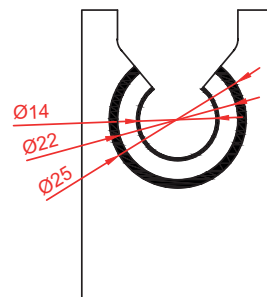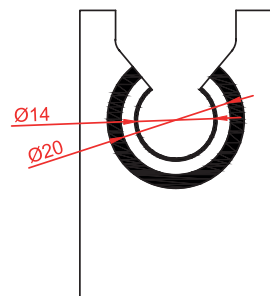
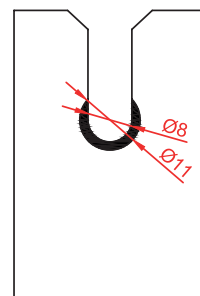
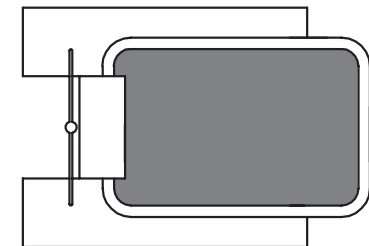
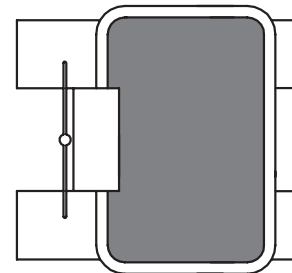
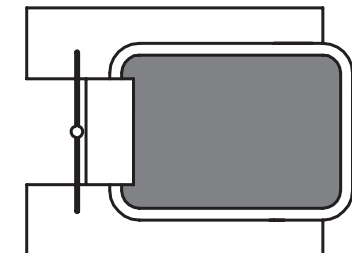
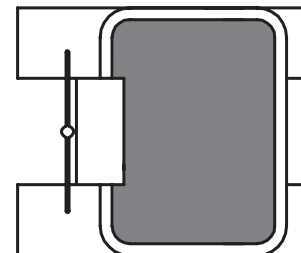
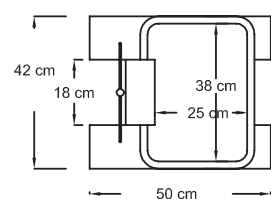
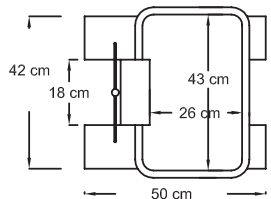
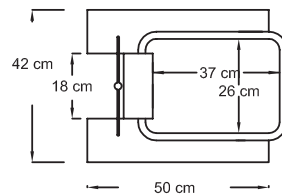
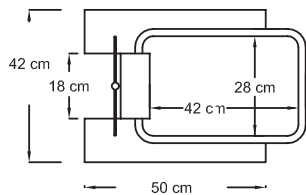
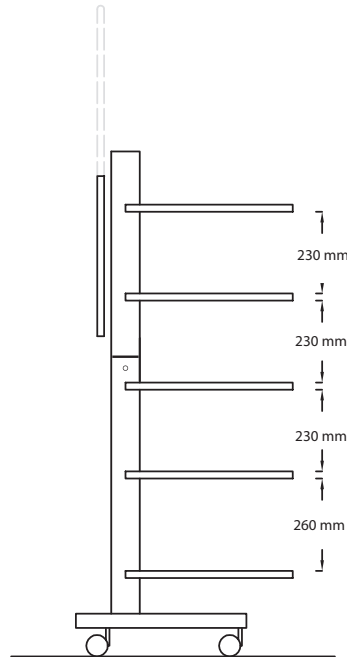
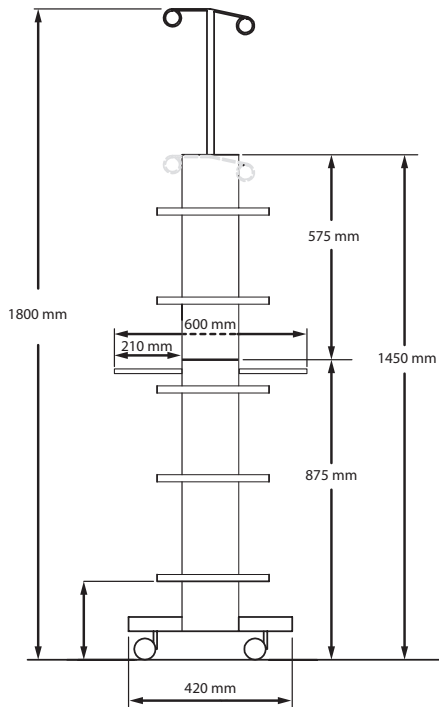
#### TECHNICAL SPECIFICATIONS

Voltage: 230 VAC  
 Maximum power consumption: 1150 W  
 Maximum output current: 5 A  
 Electrical power supply: 230VAC outlet  
 with 3 x 1.5 mm power cord  
 Weight: 25 kg

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*The water unit made of white enamelled stainless steel, equipped with glass, cuspidor and suction unit holders, has been designed to help the doctor if he/she is working without an assistant. It can be easily placed near or far from the operating area. In regular dental units, suction is enabled only when the cannulae are removed from the holder: Aspirol, instead, is equipped with a special power button that allows you to turn the suction unit off and on as needed; during use, this system is perfectly suited for rigid suction cannulae, such as Jankauer / Yankauer.*

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*The suction cannula switch-on button allows you to control the suction process as needed.*

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*Rinsing glass and cuspidor controls in stainless steel integrated in the handle.*

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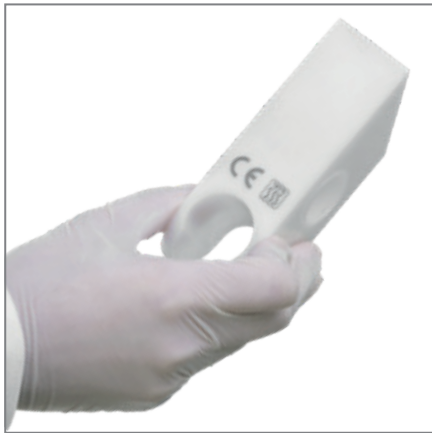
*The irrigation unit, the cuspidor and the cannula e terminals are removable and autoclave compatible to ensure a high level of hygiene in the practice.*

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*Particular attention has been paid to cannulae fixtures, which are usually fixed, but can also be removed and sterilized in an autoclave. Aspirol is completed by the automatic Sanivac system, which allows for perfect cleansing and disinfection of the suction tubes by means of a specific liquid taken from a dedicated tank.*

Supporti cannule estraibili ed autoclavabili a 121°C.  
*The cannula support are removable and autoclavable at 121°C.*

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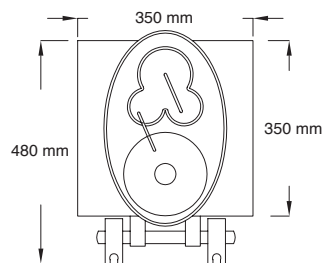
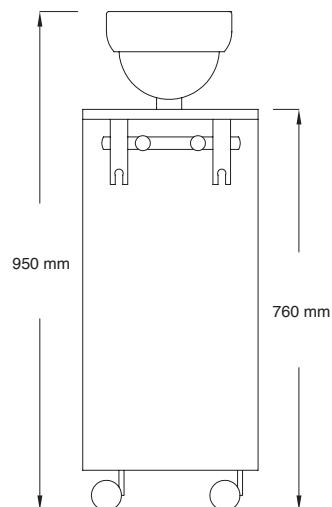
*The cuspidor (or spittoon) is now an outdated element in dentistry, especially for those operating units found within a surgical context. Even if the unit is equipped with a removable cuspidor, it is preferable to use the suction funnel system: initially used in the United States (where it has replaced the cuspidor), the sterilisable or disposable funnel connected to a suction tube offers numerous advantages.*

*Doing away with the cuspidor is a small change that contributes to maintaining a high-level of work quality, safety and hygiene in the dental practice or operating room.*

*In keeping with this philosophy of innovation, Tecnomed has designed the Aspiserve mobile unit: Aspiserve is the perfect solution for dental operating units within a surgical context, where the focus on reducing the overall bacterial load within the entire space is one of the top requirements.*

Filtro secreti con ampia superficie filtrante.  
*Secretis suction filter with large surface area.*

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*A practical drawer allows you to have the saliva ejector cannula at hand; all the equipment needed for suction can be placed inside the cabinet unit equipped with a shelf.*

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#### **HYGIENE BENEFITS**

*Patients are becoming more attentive and, if the funnel is placed in their hands in front of them, they avoid getting dirty and dirtying the area, resulting in less space to decontaminate after each visit.*

#### **TIME SAVINGS**

*Cleaning and sanitising become simpler, and not having to sanitise the water unit after each visit means optimal use of time.*

#### **IMPROVED PATIENT PERCEPTION**

*Following the same principle of sterilised and wrapped instruments and handpieces, each patient has an individual funnel-shaped element, in sterile packaging: the patient will therefore perceive a more sterile, safe and clean environment.*

#### **ERGONOMIC BENEFIT**

*The classic water unit occupies part of the work space, normally on the left hand side, dedicated to dental assistants or left-handed dentists. Even with rotating water units, one is forced to work in uncomfortable positions: by adopting the suction funnel system, space and movements are optimised.*

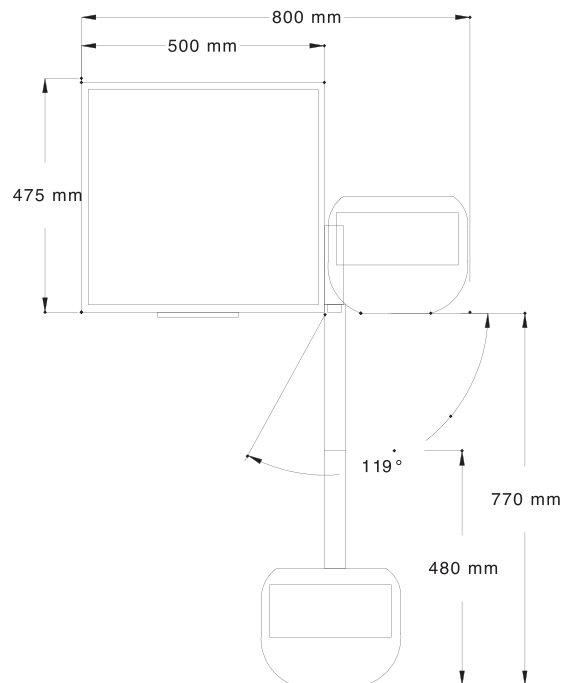
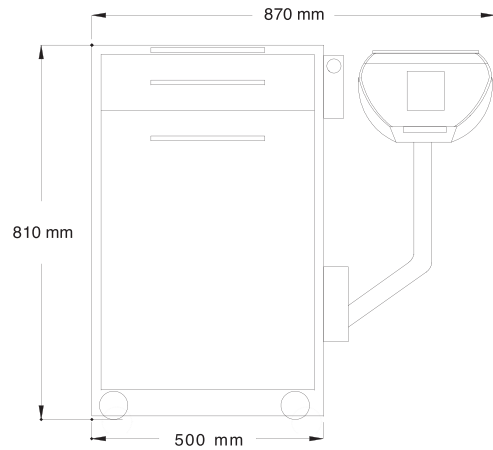
#### **LEGAL BENEFITS**

*From a legal point of view, it is not only hygiene regulations that are respected: today, more than ever, it is essential to respect amalgam recovery legislation; liquids and solids, including amalgam, are recovered exclusively by the system attached to the suction motor, in compliance with current regulations.*

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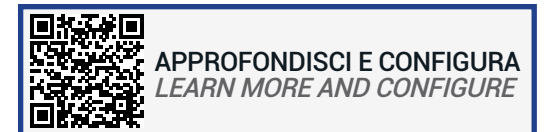
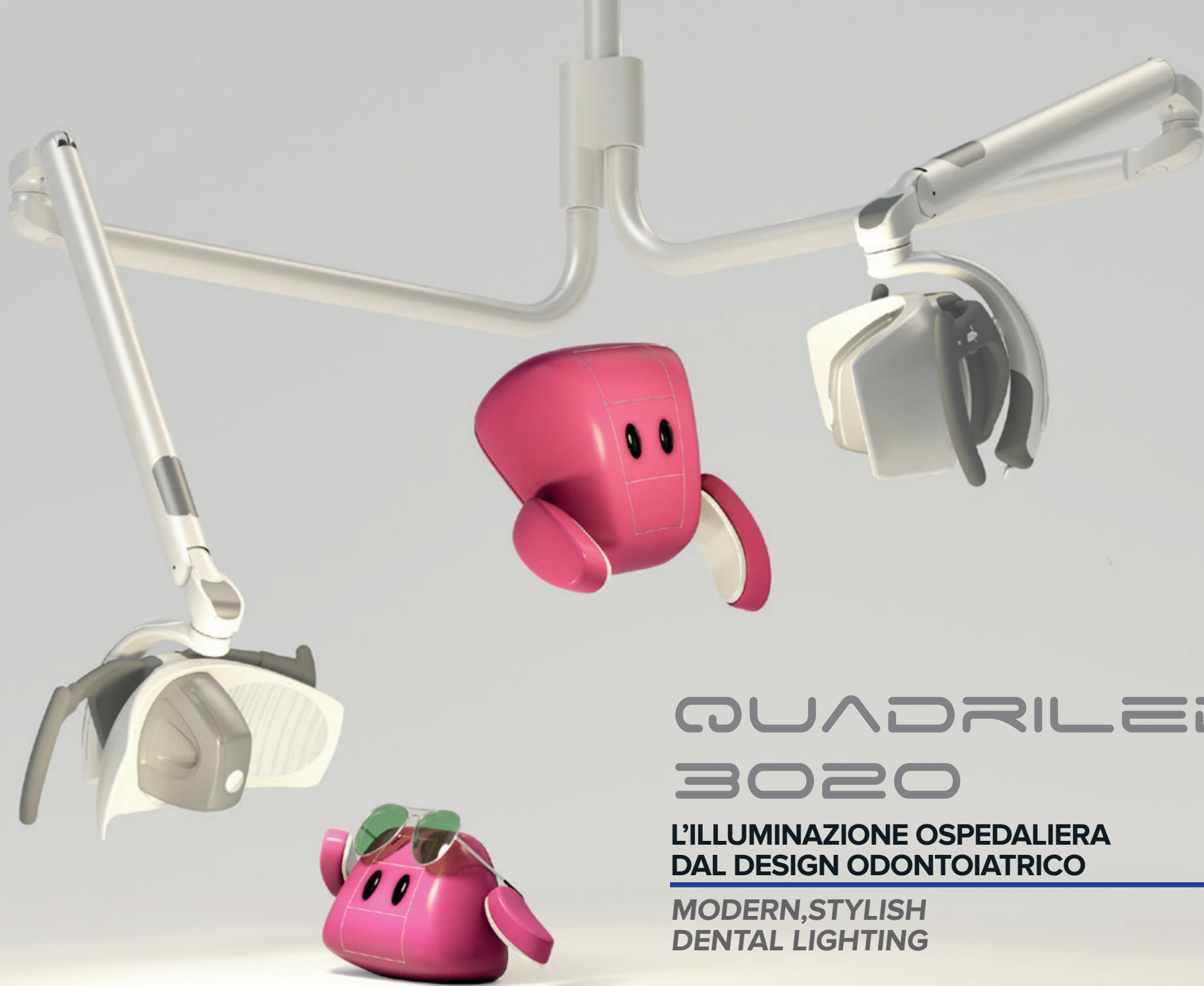
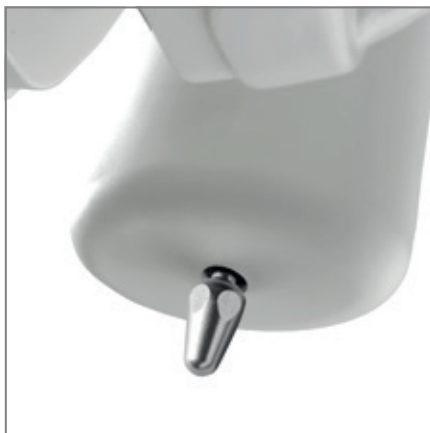
*Excellent, shadow-free visibility over the entire operating area, not just oral cavities. QuadriLED is an LED lighting unit with four intersecting light beams that produce a spot of 20x30 cm at an intensity of 100,000 lux.*


*The absence of shadows is ensured by the light beams emitted by the two heads: even if a light beam is crossed by an obstacle, the other still ensures shadow-free lighting, with a decrease in luminous intensity from 100,000 to 50,000 lux.*

*Thanks to rotation around a third axis, excellent luminosity and chromatic efficiency, QuadriLED provides perfect, uniform illumination of any operating area just like a scialytic operating light. Modern, highly professional looks combine with practicality and weight suited to dental studios. Installation does not require any reinforcement structure. QuadriLED can be installed on a wall or ceiling just like a conventional dental light.*

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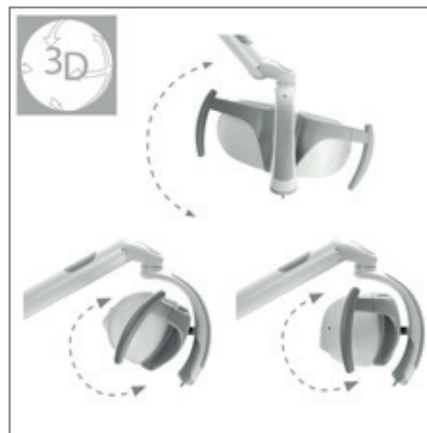*The new electronic luminous intensity control, activated by joystick or sensor, adjusts the light from maximum to minimum continuous power, keeping the colour temperature steady.*

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*Perfect balance combined with the unique patented 3-D rotation system of the head on optical axis ensures the best beam alignment accuracy, reducing the risk of glare.*

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*Surgical LED light with a total of 2064 facets that convey the reflected light.*

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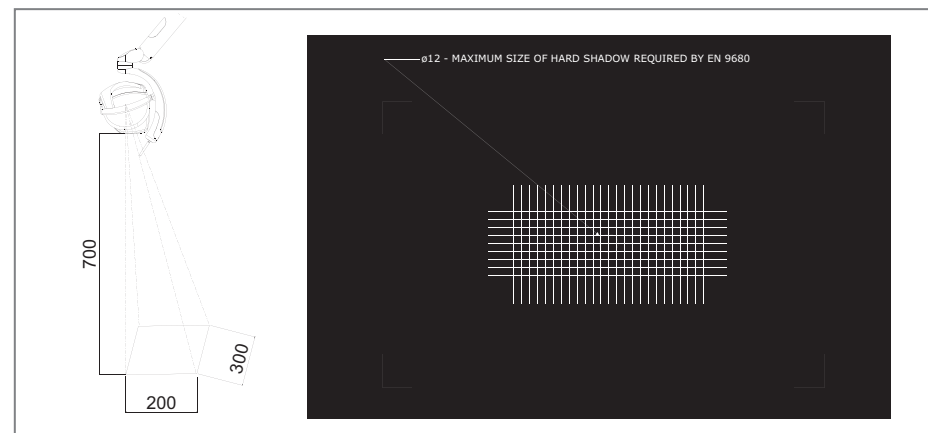
*QuadriLED features an efficient electronic control equipment, in particular:*

- *Power-on and control via micro joystick on the head*
- *Automatic saving of the luminous intensity used, after switching off*
- *Sound signal upon reaching of minimum and maximum luminous intensity*
- *Exclusive "one touch" control to get minimum luminous intensity with a simple click*

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*With the two heads, there are no shadow zones, only a drop in luminous intensity from 100,000 to 50,000 lux.*

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
*Luminous intensity can be adjusted up to 100,000 lux with 20x30 cm wide spot*

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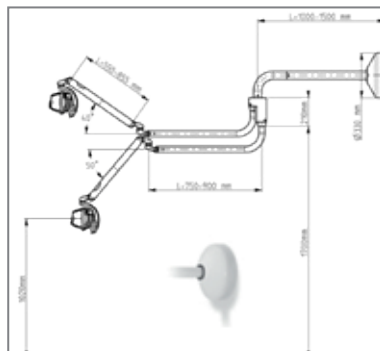
QuadriLED 3020, wall-mounted double head **CODE DE6.3020.1**

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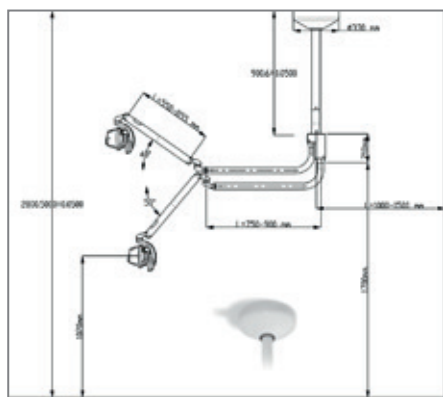QuadriLED 3020, ceiling-mounted double head **CODE DE6.3020**

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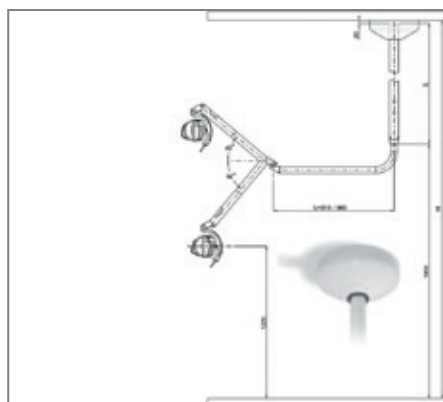
DualLED 3020, ceiling-mounted single head **CODE DE6.30.20S**

Altezza stanza (metri) Room height (metres)	Lunghezza palo da ordinare (cm) The length of the post you want to order (cm)
2,8	90
2,9	100
3	110
3,1	120
3,2	130
3,4	140

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Power supply voltage: 17-24V ~ (22-35 V) ±10% - 50/60 Hz

Power absorption: 23 VA

Protection against electrical hazards: Classe II

- Optical performances at 700 mm from the target

Dimensions of the spot of light: 300 mm x 200 mm (\*)

Lux: 6.000 to 100.000 lux (\*)

Color Temperature: 5.000 K (\*)

(\*) Typical value

The distance between the lamp and the target affect the optical performance.

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*Every professional has different operational needs, which require an environment that is functional but, at the same time, welcoming for patients that ensures they benefit from a visual impact that improves the quality of their dental practice experience. With the new Handy Surgery - Soft Line concept, dynamism and order are assured: the mobile elements allow maximum organisation and flexibility, actually adapting to each and every practice with a range of possible options. The furniture units are made of zinc-electroplated metal sheet (RAL9010 colour) that is uniform and robust thanks to a 10/10 thickness. The fronts of all the components and handles are painted with antibacterial epoxy powders.*

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*The washbasin module is equipped with Worktop in scratch-proof, water-repellent laminate and is supplied with wire ceramic sink. You can choose the position of the sink and the length of the top (1.5 or 2 metres) according to your operational needs.*

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*The wash basin is equipped with a medical lever mixers, to adjust the water temperature, and electric water supply foot pedal.*

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*The handles are in anodised aluminium and there is a choice of eight colours*

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*The drawers, mounted on guides with Teflon bearings, offer effortless sliding and are equipped with technopolymer trays, either with or without compartments, that are resistant to wear and ageing.*

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*With the countless variations and functionalities of Soft Line furniture, we design the perfect furniture solution alongside the medical professional, adapting it to the spaces available: the modularity and organisation of the space are also designed to meet the clinical needs and those of the patient.*

*These pieces have a linear and modest style: with a simple and compact line, it contributes to the creation of a clean, welcoming and aesthetically pleasing atmosphere.*

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*Everything the operators needs is provided in a practical way: each module has all the material organised and ready to be used*

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*Thanks to the use of castors, each piece of furniture becomes a genuine support element ready to be positioned according to the operational needs or personal preferences of the healthcare professional.*

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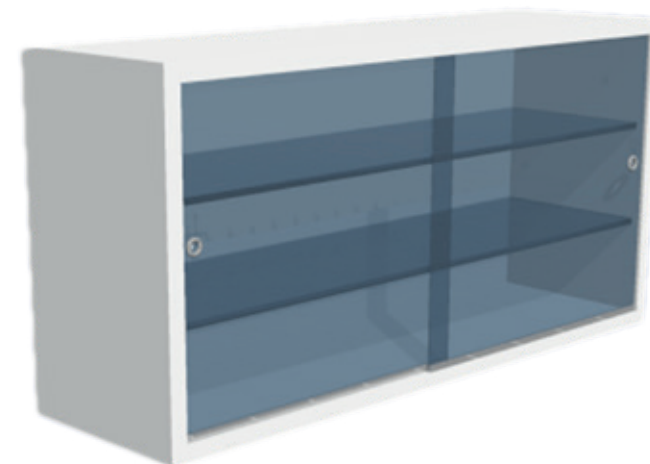
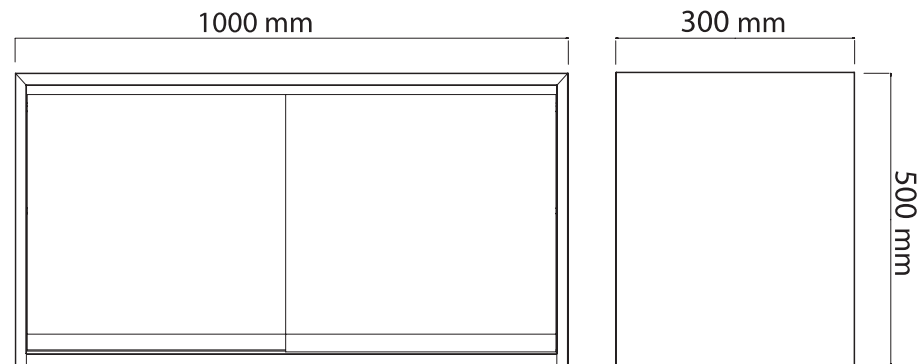
*To complete the furniture Tecnomed Italia's glass cabinets are indispensable: these cabinets offer practicality and functionality, and facilitate efficient and orderly organization. The glass cabinets available, are characterized by a solid and sturdy structure, and equipped with sliding frosted glass panels and two glass shelf.*

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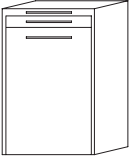
### MOBILE ASPISERVE ASPISERVE CABINET

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