## SOFT-LINER

### SOFT DENTURE RELINE MATERIAL

For use only by a dental professional in the recommended indications.

#### RECOMMENDED INDICATIONS

For the temporary rebasing (relining) of acrylic dentures. Tissue conditioning.

#### CONTRAINDICATIONS

Do not use on patients with known allergy to acrylic resins. In case of allergy, refer to a physician.

#### DIRECTIONS FOR USE

##### 1. PREPARATION

- a) Clean the contact surface thoroughly using pumice and water and abrade the area to be relined.

##### 2. MIXING

- a) The standard powder/liquid ratio is 2.2g/1.8g, (first graduation on powder measure to 4 graduations of the liquid syringe).

Note:

Consistency may be varied by changing powder/liquid ratio.

- b) Measure into the glass jar and mix for 30-60 seconds.

##### 3. RELINING

- a) Coat the mixture onto the denture as evenly as possible
- b) Insert the denture in the mouth, muscle trim and retain it in the centric occlusal position for 4-5 minutes.
- c) Remove and inspect for defects.
- d) Dip the denture into cold water, and trim away excess at the periphery with a heated sharp instrument.

#### STORAGE

Store in a cool dark place away from direct sunlight. (4 - 25°C) (39.2 - 77.0°F)  
(Shelf life: 3 years from date of manufacture)

#### PACKAGES

1-1 package : 100g powder, 100g (97mL) liquid, plus accessories.

#### NOTE

Close bottles immediately after use.

#### WARNING

1. In case of contact with eyes, flush immediately with water and seek medical attention.
2. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:

<http://www.gceurope.com>

They can also be obtained from your supplier.

#### CLEANING AND DISINFECTING:

**MULTI-USE DELIVERY SYSTEMS:** to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged.

**DO NOT IMMERSER.** Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional/national guidelines.

Last revised : 09/2016

MANUFACTURED by  
GC DENTAL PRODUCTS CORP.  
2-285 Toriimatsu-cho, Kasugai, Aichi 486-0844, Japan

DISTRIBUTED by  
GC CORPORATION  
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

EU : GC EUROPE N.V.  
Researchpark Haasrode-Leuven 1240, Interleuvenlaan 33,  
B-3001 Leuven, Belgium TEL: +32 16 74 10 00

GC ASIA DENTAL PTE. LTD.  
11 Tampines Concourse, #03-05, Singapore 528729  
TEL: +65 6546 7588

GC AUSTRALASIA DENTAL PTY. LTD.  
1753 Botany Rd, Banksmeadow, NSW 2019, Australia  
TEL: +61 2 9301 8200

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