


SONICflex™ Tip book.

KaVo SONICflex tips overview




KAVO

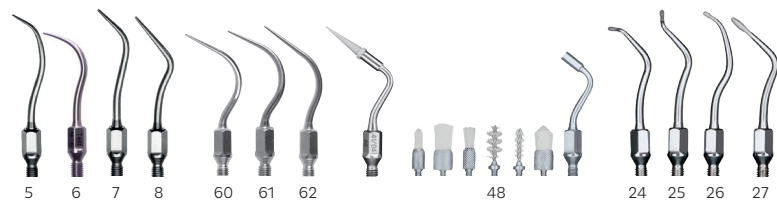

scaler | paro | implant | clean | root planer



micro | bevel | stripping, shaping | seal | caries

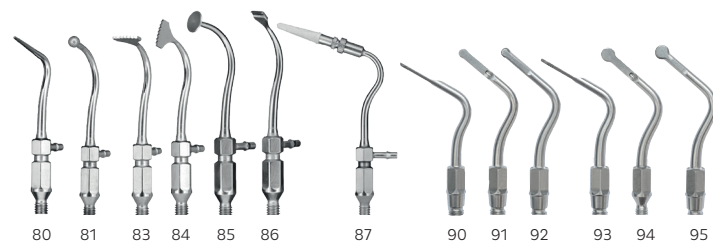

bone | crown extension



endo | retro



gold | ceram | CAD/CAM | CEM | crown

SONICflex – The great all-rounder for almost any indication.

Thanks to its versatility, the KaVo SONICflex will soon become one of your most frequently used instruments. With more than 50 tips, which are quick and easy to replace with just one turn, it adapts to your requirements across a vast range of applications.



SONICflex™

Handpiece overview.



Your daily advantage in performance.

Your KaVo SONICflex is the most powerful and quietest airscaler of its class. The handpiece with the widest range of uses.

It supports your efforts with adjustable performance levels, for every indication – strong and precise.

Superior technology – Main features and advantages for you:

- Three performance settings – optimum working conditions, for every indication
- Substantially quieter than comparable models offered by competitors (up to 15 dB) – which can directly translate into reduced stress for the dental team and patients
- No additional equipment is necessary; just connect it to the KaVo MULTIflex or Sirona coupling
- Optimum illumination: 25,000 LUX – for clear visualisation without eye-fatigue
- Legendary KaVo quality – the SONICflex 2008 L represents the third scaler generation from KaVo, the culmination of decades of knowledge and experience

PROPHYLAXIS

Important information regarding material numbers:

XX: Tip for SONICflex 2003/2000 (long thread)
XXA: Tip for SONICflex 2008 (short thread)

SONICflex Scaler tip no. 5 / 5A

Treatment: for removal of large areas of calculus
Form: Universal



Order number: Scaler tip no. 5
Scaler tip no. 5A



0.571.5171
1.005.8949

SONICflex Scaler tip no. 8 / 8A

Treatment: for removal of supragingival and subgingival calculus
Form: Perio, extra-long



Order number: Scaler tip no. 8
Scaler tip no. 8A



0.571.5371
1.006.1953

SONICflex Scaler tip no. 6 / 6A

Treatment: for removal of large areas of supragingival calculus
Form: Sickle



Order number: Scaler tip no. 6
Scaler tip no. 6A




0.571.5181
1.005.8950

SONICflex Scaler tip no. 48 / 48A

Treatment: removal of plaque, cleansing of fissures, cleansing of cavities and crown preparations
Form: various attachments, single-use brushes



Order number: Tip no. 48
Tip no. 48A
Clean brush no. 1
Clean brush no. 2
Clean brush no. 3
Clean brush no. 4
Clean brush no. 5
Clean brush no. 6
Brush set
Brush set A

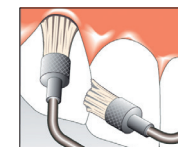


0.571.0401
1.006.1982
0.571.0402
0.571.0412
0.571.0422
0.571.0432
0.571.0472
0.571.0482
0.571.0001
1.006.2026

SONICflex Scaler tip no. 7 / 7A

Treatment: for removal of large areas of supragingival calculus
Form: Perio



Order number: Scaler tip no. 7
Scaler tip no. 7A

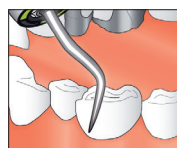


0.571.5191
1.005.8951

PARODONTOLOGY

SONICflex Paro tip no. 60 / 60A

Treatment: gentle pocket cleansing during periodontal treatment; effective cleansing in pockets up to 9 mm deep; for the anterior region, smooth surfaces

Form: long straight tip - rounded surface

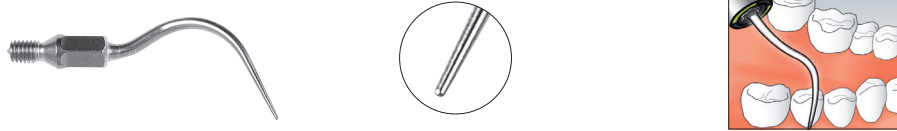

Order number: Paro tip no. 60
Paro tip no. 60A

0.571.7402
1.006.1934

SONICflex Paro tip no. 61 / 61A

Treatment: gentle pocket cleansing during periodontal treatment; effective cleansing in pockets up to 9 mm; for premolars and molars

Form: angled tip, **left** - rounded surface



Order number: Paro tip no. 61
Paro tip no. 61A

0.571.7412
1.006.1935

SONICflex Paro tip no. 62 / 62A

Treatment: gentle pocket cleansing during periodontal treatment; effective cleansing in pockets up to 9 mm; for premolars and molars

Form: angled tip, **right** - rounded surface



Order number: Paro tip no. 62
Paro tip no. 62A

0.571.7422
1.006.1936

Important information regarding
material numbers:

XX: Tip for SONICflex 2003/2000 (long thread)
XXA: Tip for SONICflex 2008 (short thread)

SONICflex Implant tip

Treatment: effective removal of subgingival plaque and calculus around implants

Form: conical tips, single-use plastic base (PEEK)



Order number: Implant set
Implant set A
Implant refill (30 pcs)

1.003.8167
1.006.2027
1.003.8168

PARODONTOLOGY

Important information regarding
material numbers:

XX: Tip for SONICflex 2003/2000 (long thread)
XXA: Tip for SONICflex 2008 (short thread)

SONICflex Root planer tip no. 24 / 24A

Treatment: for difficult-to-reach furcations, enables access to approximal furcations, even in cases of unbroken lines of dentition

Form: contra-angled button, small, **left**, diamond coating D25



Order number: Root planer tip no. 24
Root planer tip no. 24A

0.571.5621
1.006.1957

SONICflex Root planer tip no. 25 / 25A

Treatment: for difficult-to-reach furcations, enables access to approximal furcations, even in cases of unbroken lines of dentition

Form: contra-angled button, small, **right**, diamond coating D25



Order number: Root planer tip no. 25
Root planer tip no. 25A

0.571.5631
1.006.1959

SONICflex Root planer tip no. 26 / 26A

Treatment: effective cleansing of smooth surfaces and concave areas

Form: universal, button, **small**, diamond coating D25




Order number: Root planer tip no. 26
Root planer tip no. 26A

0.571.5641
1.006.1961

SONICflex Root planer tip no. 27 / 27A

Treatment: effective cleansing of smooth surfaces and concave areas

Form: extra-long perio, button, **large**, diamond coating D25

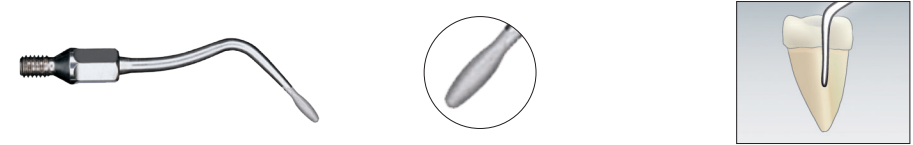

Order number: Root planer tip no. 27
Root planer tip no. 27A

0.571.5651
1.006.1963


MINIMALLY INVASIVE

Important information regarding
material numbers:

XX: Tip for SONICflex 2003/2000 (long thread)
XXA: Tip for SONICflex 2008 (short thread)

SONICflex Micro tip no. 30 / 30A

Treatment: minimally invasive treatment, for incipient caries
Form: small hemisphere, **mesial**, diamond coating D46

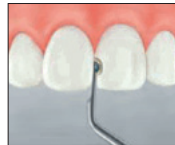


Order number: Micro tip no. 30
Micro tip no. 30A

0.571.6811
1.006.1969

SONICflex Micro tip no. 33 / 33A

Treatment: minimally invasive treatment, for incipient caries
Form: large hemisphere, **distal**, diamond coating D46

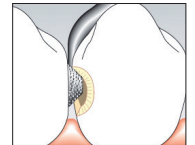


Order number: Micro tip no. 33
Micro tip no. 33A

0.571.6821
1.006.1975

SONICflex Micro tip no. 31 / 31A

Treatment: minimally invasive treatment, for incipient caries
Form: small hemisphere, **distal**, diamond coating D46

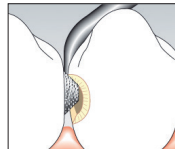


Order number: Micro tip no. 31
Micro tip no. 31A

0.571.6801
1.006.1971

SONICflex Bevel tip no. 58 / 58A

Treatment: bevelling of approximal cavity edges
Form: **mesial**, diamond coating D46

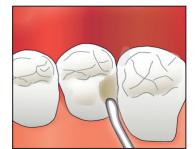


Order number: Bevel tip no. 58
Bevel tip no. 58A

0.571.7392
1.006.1988

SONICflex Micro tip no. 32 / 32A

Treatment: minimally invasive treatment, for incipient caries
Form: large hemisphere, **mesial**, diamond coating D46

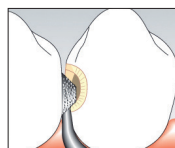

Order number: Micro tip no. 32
Micro tip no. 32A

0.571.6831
1.006.1973

SONICflex Bevel tip no. 59 / 59A

Treatment: bevelling of approximal cavity edges
Form: **distal**, diamond coating D46



Order number: Bevel tip no. 59
Bevel tip no. 59A

0.571.7382
1.006.1990


MINIMALLY INVASIVE

Important information regarding material numbers:

XX: Tip for SONICflex 2003/2000 (long thread)
XXA: Tip for SONICflex 2008 (short thread)

SONICflex Seal tip no. 45 / 45A

Treatment: minimally invasive preparation of fissures, no-pain when opening fissures
Form: conical, straight tip Ø 0.5 mm, diamond coating D46

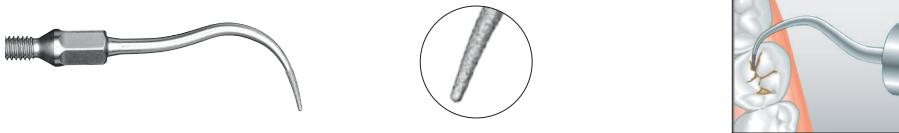

Order number: Seal tip no. 45
Seal tip no. 45A

1.000.8323
1.007.1506

SONICflex Cariex D tip no. 42 / 42A

Treatment: ultra-conservative, sonoabrasive caries excavation, fine
Form: ball, Ø 0.8 mm, diamond coating D64



Order number: Cariex D tip no. 42
Cariex D tip no. 42A

1.000.7105
1.006.1980

SONICflex Cariex D tip no. 43 / 43A

Treatment: ultra-conservative, sonoabrasive caries excavation, fine
Form: ball, Ø 1.2 mm, diamond coating D64



Order number: Cariex D tip no. 43
Cariex D tip no. 43A

1.000.7167
1.006.1981

SONICflex Cariex TC tip no. 71 / 71A

Treatment: controlled, layer-by-layer removal of carious tissues
Form: ball, Ø 1.0 mm, hard metal



Order number: Cariex TC tip no. 71
Cariex TC tip no. 71A

1.000.7362
1.006.2002

SONICflex Cariex TC tip no. 72 / 72A

Treatment: controlled, layer-by-layer removal of carious tissues
Form: ball, Ø 1.4 mm, hard metal



Order number: Cariex TC tip no. 72
Cariex TC tip no. 72A

1.000.7363
1.006.2004

RESTORATION


Important information regarding
material numbers:

XX: Tip for SONICflex 2003/2000 (long thread)
XXA: Tip for SONICflex 2008 (short thread)

SONICflex Prep gold tip no. 49 / 49A mesial

Treatment: precise preparation of various bevel-angles around margins, after "rough preparation" with rotary instrument; this enables improved dental restoration

Form: **mesial**, tips with defined bevel-angles: lateral - 60°, cervical - 30°, diamond coating D46

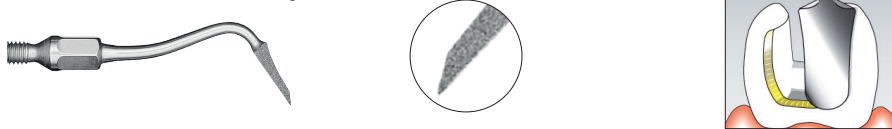


Order number: Prep gold tip no. 49
Prep gold tip no. 49A

0.571.7212
1.006.1983

SONICflex Prep gold tip no. 50 / 50A distal

Treatment: precise preparation of various bevel-angles around margins, after "rough preparation" with rotary instrument; this enables improved dental restoration

Form: **distal**, tips with defined bevel-edges: lateral - 60°, cervical - 30°, diamond coating D46

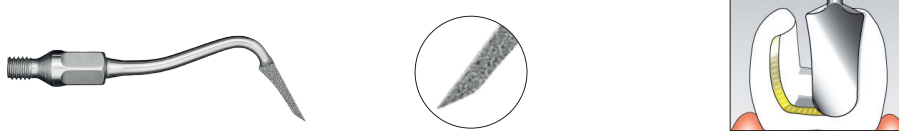


Order number: Prep gold tip no. 50
Prep gold tip no. 50A

0.571.7222
1.006.1984

SONICflex Prep ceram tip no. 51 / 51A

Treatment: precise preparation of various bevel-angles around margins, with gradual angle transitions from cervical to lateral areas

Form: **mesial**, tips with defined bevel-angles: lateral - 60°, cervical - 75°, diamond coating D46

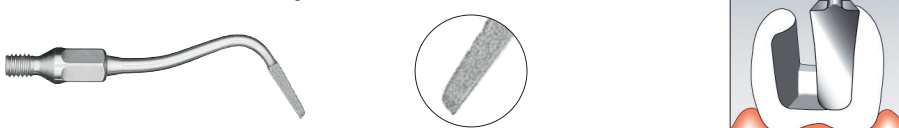


Order number: Prep ceram tip no. 51
Prep ceram tip no. 51A

0.571.7262
1.006.1985

SONICflex Prep ceram tip no. 52 / 52A

Treatment: precise preparation of various bevel-angles around margins, with gradual angle transitions from cervical to lateral areas

Form: **distal**, tips with defined bevel-edges: lateral - 60°, cervical - 75°, diamond coating D46

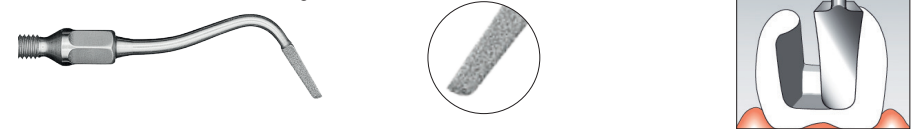

Order number: Prep ceram tip no. 52
Prep ceram tip no. 52A


0.571.7272
1.006.1986

RESTORATION

SONICflex Prep CAD/CAM tip no. 34 / 34A

Treatment: perfect adaptation of cavity preparations, to the specific requirements of CAD/CAM systems

Form: **mesial**, lateral angle 30°, cervical angle 90°, diamond coating D46

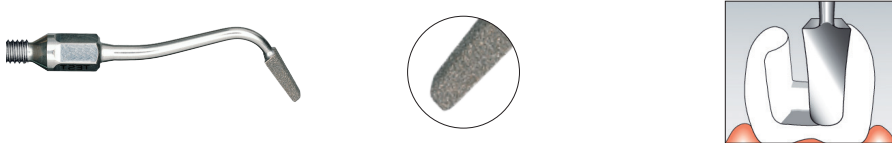


Order number: Prep CAD/CAM tip no. 34
Prep CAD/CAM tip no. 34A

1.002.1984
1.006.1977

SONICflex Prep CAD/CAM tip no. 35 / 35A

Treatment: perfect adaptation of cavity preparations, to the specific requirements of CAD/CAM systems

Form: **distal**, lateral angle 30°, cervical angle 90°, diamond coating D46

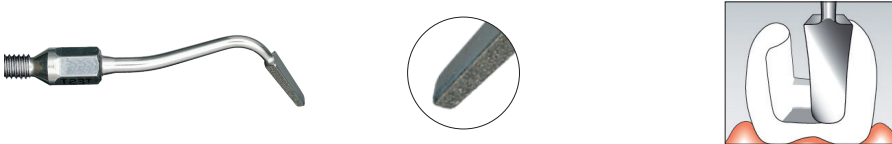

Order number: Prep CAD/CAM tip no. 35
Prep CAD/CAM tip no. 35A

1.002.1986
1.006.1979

Important information regarding
material numbers:

XX: Tip for SONICflex 2003/2000 (long thread)
XXA: Tip for SONICflex 2008 (short thread)

SONICflex Stripping tip no. 73 / 73A

Treatment: separation before commencing the crown stump preparation, rounding of sharp transitions on proximal preparation margins in the scope of cavity preparation, anatomical design of proximal surfaces of composite fillings, proximal enamel reduction in the scope of orthodontics

Form: flat, mesial diamond coating D25



Order number: Stripping tip no. 73
Stripping tip no. 73A

1.011.2993
1.011.2994

SONICflex Stripping tip no. 74 / 74A

Treatment: separation before commencing the crown stump preparation, rounding of sharp transitions on proximal preparation margins in the scope of cavity preparation, anatomical design of proximal surfaces of composite fillings, proximal enamel reduction in the scope of orthodontics

Form: flat, distal diamond coating D25



Order number: Stripping tip no. 74
Stripping tip no. 74A

1.011.2995
1.011.2996

RESTORATION

SONICflex Shaping tip no. 75 / 75A

Treatment: separation before commencing the crown stump preparation, rounding of sharp transitions on proximal preparation margins in the scope of cavity preparation, anatomical design of proximal surfaces of composite fillings, proximal enamel reduction in the scope of orthodontics

Form: curved, mesial diamond coating D25



Order number: Shaping tip no. 75
Shaping tip no. 75A

1.011.2997
1.011.2998

SONICflex Shaping tip no. 76 / 76A

Treatment: separation before commencing the crown stump preparation, rounding of sharp transitions on proximal preparation margins in the scope of cavity preparation, anatomical design of proximal surfaces of composite fillings, proximal enamel reduction in the scope of orthodontics

Form: curved, distal diamond coating D25



Order number: Shaping tip no. 76
Shaping tip no. 76A

1.011.2999
1.011.3000


Important information regarding material numbers:

XX: Tip for SONICflex 2003/2000 (long thread)
XXA: Tip for SONICflex 2008 (short thread)

SONICflex CEM tip no. 12 / 12A

Treatment: gentle and secure placement of inlays

Form: rounded plastic cover

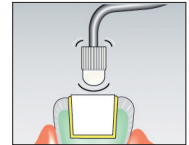



Order number: CEM tip no. 12
CEM tip no. 12A
CEM attachment

0.571.5431
1.006.2021
0.571.7142

SONICflex Crown prep tip no. 97 / 97A

Treatment: crown preparation - the use of rotary, crown preparation instruments, in combination with sonically-vibrated tips of similar form, is the perfect protocol for successful crown preparation

Form: radius, diamond coating D46

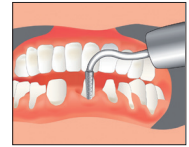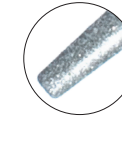



Order number: Crown prep tip no. 97
Crown prep tip no. 97A

1.008.6383
1.008.6384

SONICflex Crown prep tip no. 98 / 98A

Treatment: crown preparation - the use of rotary, crown preparation instruments, in combination with sonically-vibrated tips of similar form, is the perfect protocol for successful crown preparation

Form: flat, diamond coating D46

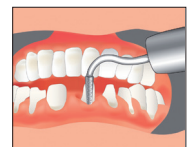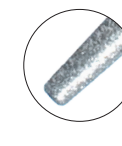

Order number: Crown prep tip no. 98
Crown prep tip no. 98A

1.008.6385
1.008.6386

ROOT TIP RESECTION

Important information regarding material numbers:

XX: Tip for SONICflex 2003/2000 (long thread)
XXA: Tip for SONICflex 2008 (short thread)

SONICflex Retro tip no. 56 / 56A

Treatment: searching for/locating of root canal; significant work simplification during retrograde cavity preparation
Form: extremely narrow cylindrical tip, **left**, diamond coating D46



Order number: Retro tip no. 56
Retro tip no. 56A

0.571.7322
1.006.2040

SONICflex Retro tip no. 57 / 57A

Treatment: searching for/locating of root canal; significant work simplification during retrograde cavity preparation
Form: narrow cylindrical tip, **right**, diamond coating D46



Order number: Retro tip no. 57
Retro tip no. 57A

0.571.7332
1.006.2041

SONICflex Retro tip no. 16 / 16A

Treatment: preparation of root canal, significantly simplified access, even with highly-convoluted roots
Form: **left**, small cylinder, diamond coating D64



Order number: Retro tip no. 16
Retro tip no. 16A

0.571.5541
1.006.2036

SONICflex Retro tip no. 17 / 17A

Treatment: preparation of root canal, significantly simplified access, even with highly-convoluted roots
Form: **right**, small cylinder, diamond coating D64



Order number: Retro tip no. 17
Retro tip no. 17A

0.571.5581
1.006.2035

SONICflex Retro tip no. 20 / 20A and no. 21 / 21A

Treatment: preparation of the undercut; correct, internal circumferential root canal preparation, with minimal bone defects
Form: pear-shaped, **small, left**, diamond coating D46



Order number: Retro tip no. 20
Retro tip no. 20A

0.571.5521
1.006.2037

Form: pear-shaped, **small, right**, diamond coating D46



Order number: Retro tip no. 21
Retro tip no. 21A

0.571.5561
1.006.2038

SONICflex Retro tip no. 55 / 55A

Treatment: for roots, with apices highly-inclined towards buccal cavity
Form: diamond coating D46



Order number: Retro tip no. 55
Retro tip no. 55A

0.571.7342
1.006.2039

ENDODONTICS

SONICflex Endo tip no. 66 / 66A


Treatment: shaping of access cavity, removal of protuberances
Form: button, **large**, diamond coating D46



Order number: Endo tip no. 66 1.000.5825
Endo tip no. 66A 1.006.1992

SONICflex Endo tip no. 67 / 67A

Treatment: expansion of larger channels
Form: conical **125°**, diamond coating D46

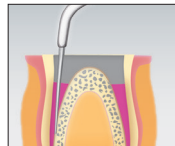

Order number: Endo tip no. 67 1.000.5822
Endo tip no. 67A 1.006.1994

SONICflex Endo tip no. 68 / 68A

Treatment: expansion of larger channels
Form: conical **125°**, diamond coating D46




Order number: Endo tip no. 68 1.000.5823
Endo tip no. 68A 1.006.1996

Important information regarding material numbers:

XX: Tip for SONICflex 2003/2000 (long thread)
XXA: Tip for SONICflex 2008 (short thread)

SONICflex Endo tip no. 69 / 69A


Treatment: removal of residual root canal filling materials
Form: button, **small**, diamond coating D46

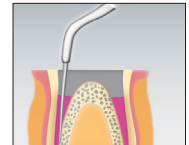

Order number: Endo tip no. 69 1.000.5827
Endo tip no. 69A 1.006.1998

SONICflex Endo tip no. 70 / 70A



Treatment: for particularly long and wide canals
Form: conical **117°**, diamond coating D25, anterior area is not diamond coated

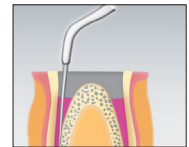

Order number: Endo tip no. 70 1.000.5821
Endo tip no. 70A 1.006.2000

SONICflex Endo clean no. 96 / 96A

Treatment: for flushing of root canal, for removal of debris and smear layer, as well as enhancing the performance of antibacterial rinsing solution
Form: 105° resp. 75° angle

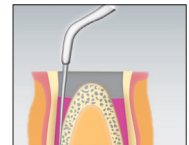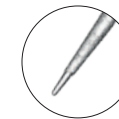

Order number:
1x Holder No. 96/96A and 6x endo clean needles 015, 020 and 025:


Set: 1.006.2045 | Set A: 1.007.1142

Endo clean holder no. 96:

1.008.5163 | A: 1.008.5164

Endo clean needles (6 pcs each) for all SONICflex:

015 white: 1.006.2042 | 020 yellow: 1.006.2043 | 025 red: 1.006.2044

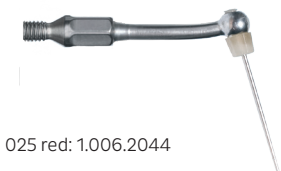

SURGERY

Important information regarding material numbers:

XX: Tip for SONICflex 2003/2000 (long thread)
XXA: Tip for SONICflex 2008 (short thread)


SONICflex Crown extension tip 020 no. 90 / 90A

Treatment: surgical crown extension without flap formation for restoration of the biological breadth in the presence of:
 non-symmetrical arcade profile, gummy smile, violation of biological breadth

Form: small, straight, diamond-coated outer surface D46



Order number: Crown extension tip 020 no. 90
 Crown extension tip 020 no. 90A

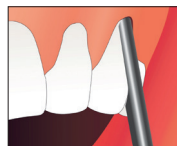

1.011.3001
 1.011.3002

SONICflex Crown extension tip 020 no. 92 / 92A

Treatment: surgical crown extension without flap formation for restoration of the biological breadth in the presence of:
 non-symmetrical arcade profile, gummy smile, violation of biological breadth

Form: small, oblique, diamond-coated outer surface D46



Order number: Crown extension tip 020 no. 92
 Crown extension tip 020 no. 92A



1.011.3005
 1.011.3006


SONICflex Crown extension tip 020 no. 91 / 91A

Treatment: surgical crown extension without flap formation for restoration of the biological breadth in the presence of:
 non-symmetrical arcade profile, gummy smile, violation of biological breadth

Form: small, oblique, diamond-coated outer surface D46



Order number: Crown extension tip 020 no. 91
 Crown extension tip 020 no. 91A

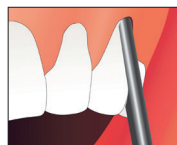

1.011.3003
 1.011.3004


SONICflex Crown extension tip 030 no. 93 / 93A

Treatment: surgical crown extension without flap formation for restoration of the biological breadth in the presence of:
 non-symmetrical arcade profile, gummy smile, violation of biological breadth

Form: large, straight, diamond-coated outer surface D46



Order number: Crown extension tip 030 no. 93
 Crown extension tip 030 no. 93A

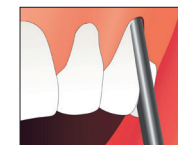


1.011.3007
 1.011.3008

SURGERY

SONICflex Crown extension tip 030 no. 94 / 94A

Treatment: surgical crown extension without flap formation for restoration of the biological breadth in the presence of:
non-symmetrical arcade profile, gummy smile,
violation of biological breadth

Form: large, oblique, diamond-coated outer surface D46

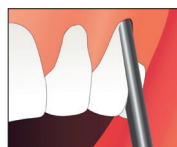


Order number: Crown extension tip 030 no. 94
Crown extension tip 030 no. 94A

1.011.3009
1.011.3010

SONICflex Crown extension tip 030 no. 95 / 95A

Treatment: surgical crown extension without flap formation for restoration of the biological breadth in the presence of:
non-symmetrical arcade profile, gummy smile,
violation of biological breadth

Form: large, oblique, diamond-coated outer surface D46

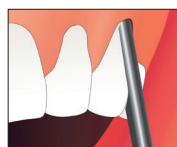

Order number: Crown extension tip 030 no. 95
Crown extension tip 030 no. 95A

1.011.3011
1.011.3012


Important information regarding
material numbers:

XX: Tip for SONICflex 2003/2000 (long thread)
XXA: Tip for SONICflex 2008 (short thread)

SONICflex Bone tip no. 80 / 80A

Treatment: sectioning of Sharpey's fibres, extension of alveolae, removal of root residues,
gentle extraction before implantation, bone splitting

Form: square, cutting

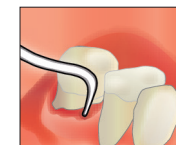



Order number: Bone tip no. 80
Bone tip no. 80A

1.004.3875
1.006.2006

SONICflex Bone tip no. 81 / 81A

Treatment: preparation of bone window for sinus lift, with very massive bones, if applicable,
rotating and sectioning of substantia corticalis, plus removal of remaining bone
using the diamond-coated tip

Form: ball, large, diamond coating D46

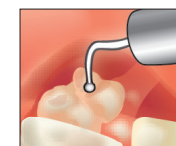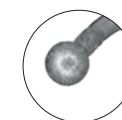

Order number: Bone tip no. 81
Bone tip no. 81A

1.004.3876
1.006.2008

SONICflex Bone tip no. 83 / 83A

Treatment: removal of bone blocks, osteotomies (distraction, bone splitting), if applicable,
dealing with extremely thick, substantia corticalis

Form: saw sagittal



Order number: Bone tip no. 83
Bone tip no. 83A


1.004.3878
1.006.2012

SURGERY

SONICflex Bone tip no. 84 / 84A

Treatment: removal of bone blocks, osteotomies (distraction, bone splitting); if applicable, dealing with extremely thick, substantia corticalis

Form: saw axial

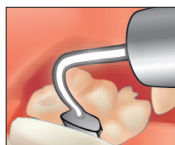

Order number: Bone tip no. 84
Bone tip no. 84A

1.004.3879
1.006.2014

SONICflex Bone tip no. 85 / 85A

Treatment: the disc-shaped form makes it ideal for safe repositioning of the Schneider membrane during a sinus lift

Form: elephant foot




Order number: Bone tip no. 85
Bone tip no. 85A

1.006.0645
1.007.1624

SONICflex Bone tip no. 86 / 86A

Treatment: ideal for easy and quick harvesting of greater quantities of bone, especially useful for bone augmentation

Form: bone harvester

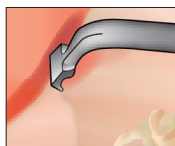

Order number: Bone tip no. 86
Bone tip no. 86A

1.006.0741
1.007.1625

Important information regarding material numbers:


XX: Tip for SONICflex 2003/2000 (long thread)

XXA: Tip for SONICflex 2008 (short thread)

SONICflex Bone tip no. 87 / 87A

Treatment: ideal for precise, extremely narrow incisions, when cutting ligaments

Form: saw axial, replaceable saw blades
saw blade thickness: 0.15 mm

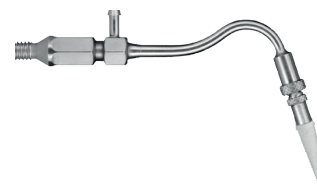

Order number: Bone tip no. 87
Bone tip no. 87A
Saw blades (6 pcs)

1.006.1406
1.007.1627
1.006.1405

Accessories:



Torque wrench

1.000.4887



Steri cassette for up to 6 tips

0.411.9101



Key

0.411.0892



Adapter for supply
of liquid medicaments

1.000.6296
A: 1.006.5966

Dental Excellence in every area.



Practice equipment

KaVo treatment units and lights, dental chairs, patient communication systems, dental microscope and additional operatory accessories.



Instruments

Dental straight and contra-angle handpieces, turbines, air polishing systems and small equipment for all application areas including diagnosis, prophylaxis, restorative, surgery, endodontics and instrument care.



Imaging

Intraoral X-ray equipment, sensors and imaging plate systems, panoramic and cephalometric in combination with CBCT, as well as dedicated CBCT devices for every indication in dentistry.



CAD/CAM

Dental CAD/CAM solutions for premium aesthetic, natural-looking and long-lasting restorative work, suitable for dentists and dental technicians.

Mat.-No. 10087148 11/19 en
We reserve the right to make technical modifications.
Slight colour differences are due to the printing process. © Copyright KaVo Dental GmbH.