SR Ivolen®

Instructions for Use
Verarbeitungsanleitung
Mode d'emploi
Istruzioni d'uso
Instrucciones de uso
Instruções de Uso

Bruksanvisning
Brugsanvisning
Käyttöohjeet
Bruksanvisning
Productinformatie
Οδηγίες Χρήσεως

BPS





For dental use only.

Caution: U.S. Federal Law restricts this device to sale by or on the order of a licensed dentist.

Made in Liechtenstein

Ivoclar Vivadent AG, FL-9494 Schaan / Liechtenstein

The logo graphic for Ivoclar Vivadent, featuring a series of dots arranged in a semi-circular pattern above the text.
ivoclar
vivadent
technical

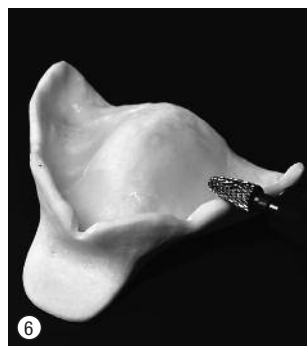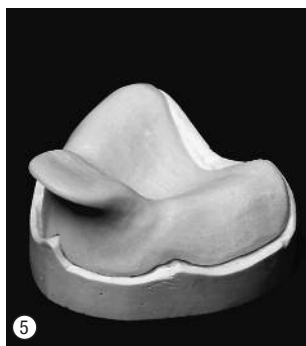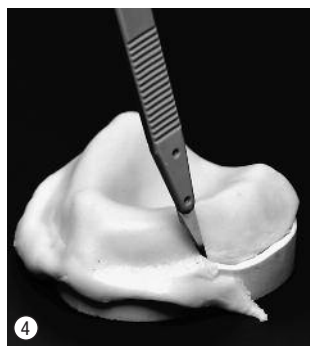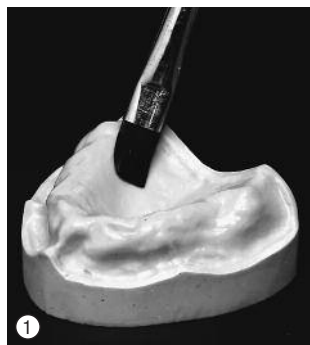Instructions for Use

Product Description

The cold-curing SR Ivolen is especially suitable for the fabrication of individualized trays, base plates and registration base plates.

In conjunction with the corresponding adhesive, SR Ivolen can be used with most impression materials.

Colour

yellow

Composition

Powder

Polymethyl methacrylate, copolymer, calcium carbonate, catalyst and pigment

Liquid

Methyl methacrylate and paraffin oil, catalyst

Indication

- Personalized impression trays
- Individualized base plates for:
 - wax rims
 - intra-oral registration devices (e.g. Gnathometer M)
 - Tooth setup with wax
 - Fabrication of incisal plates for personalized anterior guidance (e.g. in Stratos®)

Contraindication

- Material must not be processed in the oral cavity.
- Material is unsuitable for alginates and hydrocolloids.
- If the patient is known to be allergic to any of the ingredients in SR Ivolen.

Application

- ① After blocking out undercuts, soak the cast and isolate with Ivoclar Vivadent Separating Fluid.
- ② Mix the polymer (powder) and monomer (liquid) according to the Instructions: 1 measuring spoon of SR Ivolen polymer to 6 ml of SR Ivolen monomer. Mix in a mixing cup and spatulate well until the dough no longer sticks to the mixing cup. Knead the dough for a few more seconds. Wear gloves for kneading.
- ③ SR Ivolen can be immediately applied and shaped; the material is not sticky. Alternatively, the material can be pressed between 2 polyethylene sheets with a compression roller to form a layer of uniform thickness.
- ④ Adapt SR Ivolen to the cast; shape with a suitable instrument and cut off excess.

- ⑤ You may use excess to form tray handles.
- ⑥ After the material has self-cured for approx. 7 minutes, trim the margins with burs or similar instruments. If desired, you may also polish SR Ivolen.

Warnings

- The monomer contains methyl methacrylate (MMA)
- MMA is irritating and easily flammable (flash point: +10 °C/50 °F)
- MMA is irritating to the eyes, skin and respiratory system
- May cause sensitization upon skin contact
- Avoid contact of the skin with monomer or uncured material. Commercial medical gloves do not provide protection against the sensitizing effect of methacrylates.
- Do not inhale vapours
- Keep away from sources of ignition – no smoking.
- Do not empty into drains.
- Take precautionary measures against static discharges

Storage

- Storage temperature: 2–28 °C / 36–82 °F. Protect material from light.
- Do not use material after the indicated date of expiration.
- Keep out of the reach of children.

Date information prepared: 11/2010

Manufacturer:

Ivoclar Vivadent AG, FL-9494 Schaan / Liechtenstein

The material has been developed solely for use in dentistry. Processing should be carried out according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions constitute no warranty of attributes.

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Ivoclar Vivadent Limited
Ivoclar Vivadent, Inc.