TEMPSMART™ DC

DUAL-CURED COMPOSITE FOR TEMPORARY CROWN AND BRIDGE

For use only by a dental professional for the following indications.

INDICATIONS FOR USE

- Fabrication of temporary crowns, bridges, inlays, onlays and veneers
- Fabrication of long-term temporary restorations

CONTRAINDICATIONS

Avoid use of this product in patients with known allergies to methacrylate monomer or methacrylate polymer.

DIRECTIONS FOR USE

Fabrication of temporary restorations

- Take the impression using alginate or silicone impression material. Alternatively, temporary can be made using a laboratory-prepared thermoforming matrix.
- Block out preparation undercuts.
- Block out preparation undercuts.
- Clean tooth preparation with water spray and lightly dry.
- Lightly lubricate with separating medium, such as petroleum jelly or prepared teeth or resin core.

NOTE:
Prior to first use of a new syringe or after a long interval between uses, extrude paste from the syringe onto a mixing pad.

1) Remove cap of syringe by rotating 1/4 turn counter-clockwise. Attach a mixing tip to the V-shaped notch on the mixing pad. Place the tip with the notch facing between the fingers. Push firmly to attach the mixing tip. Then rotate the colored cap of mixing tip 1/4 turn clockwise.

2) Discard first centimeter of extruded material before use.

3) Extrude composite directly into impression.

NOTE:
Use mixing tip can be left on syringe to serve as a storage cap until next use. For each application, place a new mixing tip.

4) Fabricating a temporary restoration

Fabricate a temporary crown and bridge material according to timing shown below. Time indicated starts after mixing paste.

Option 1: Fabricating in the mouth

0'00"-0'45" Place TEMPSMART DC into impression and reposition in mouth.

Due to high polymerization temperature created, do not irradiate temporary restoration for more than 2 seconds. Do not final cure on tooth.

1) Above processing times are applicable for products stored and processed at a temperature of 23°C/74°F and at 50% relative humidity. Higher temperatures shorten these times, while lower temperatures extend them. The size of the restoration also has impact on setting times: as size of restoration increases, setting takes longer.

Do not breathe in polishing dust. Use personal protective devices such as facemask and safety glasses.

(1) When using a resin material such as TEMPSMART DC, G-aenial Flo X, G-aenial Universal Injectable or UNIFAST Trad, or grind to obtain proper occlusion.

No use eugenol cement as it will prevent polymerization of resin.

Completely remove saliva contaminated part using a carbide bur.

A1, A2, A3, A5, B1 and BW

Recommended for optimal performance, store in a cool and dark place (4-25°C / 39-77°F) away from high temperatures or direct sunlight.

MULTI-USE DELIVERY SYSTEMS: to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged.

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