**VOCO****Traypurol****Concentrated cleaning agent for impression trays and instruments**

Traypurol removes residues of alginates, zinc oxide eugenol, carboxylate and phosphate cement from impression trays and instruments without the use of an ultrasonic device.

Traypurol is biodegradable according to OECD guidelines and provides gentle cleaning action.

Instructions for use:

Use **Traypurol** in a 5 to 10 % solution (dilute approx. 50 - 100 ml **Traypurol** in 1 l of water). For persistent residues, **Traypurol** can also be used in a more concentrated solution. Warm water accelerates the cleaning process.

Pre-clean the trays and instruments and immerse them in the **Traypurol** solution. The cleaning process will be enhanced if trays are placed upright in the solution.

Residues are softened in the **Traypurol** solution within a few hours and can then be easily rinsed off under running water or removed with a brush.

If possible leave impression trays and instruments soaking in the solution overnight.

Traypurol is suitable for all non-rust metals and for plastics. Prepare a fresh solution every 2 to 3 days. **Traypurol** is also very suitable for use in ultrasonic devices.

Protective gloves and glasses should be worn when using this product.

In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.

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VOCO GmbH