GENERAL PURPOSE ACRYLIC RESIN

GC UNIFAST *Trad* is a self curing, fast setting acrylic resin for use in a variety of dental applications. For use in either pressure or paint-on techniques.

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

1. Repair of dentures.
2. Construction of temporary crowns.
3. Impression trays for individual teeth.
4. Orthodontic plates, splints, check bite and other general purposes.

CONTRAINDICATIONS

In susceptible individuals, sensitization to the product cannot be excluded. Use of the product should be discontinued if allergic reactions are observed.

DIRECTIONS FOR USE


A. PRESSURE TECHNIQUE

The recommended powder / liquid ratio is 1g of powder (1st graduation of powder measure) to 0.5mL (1st graduation of liquid measure) of liquid.

1. Dispense the required amount of liquid into the rubber cup as follows :
 - a) Remove the protective cap and screw on the dispensing nozzle (fig. 1)
 - b) Pour into the measure provided (fig. 2) until the bottom of the meniscus is level with the required graduation (fig. 3).
 - c) Replace the protective cap to prevent evaporation.
2. Add the powder and mix thoroughly for 10-15 seconds.
3. The mixture reaches a dough-state 20-30 seconds after mixing. All manipulation should be finished before 2 minutes after mixing when setting starts.

B. PAINT-ON TECHNIQUE

Use standard techniques.

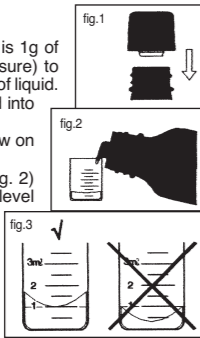STORAGE CONDITIONS

Keep dry. Excessive heat and freezing temperatures may impact product performance. For optimal performance store in a cool, dark place (2°C to 37°C / 36°F to 98°F). (Shelf life: 3 years from date of manufacture).

NOTES

1. Do not use UNIFAST Trad powder or liquid with components of other brands of acrylic resin.
2. Remove residual resin with a dry gauze after each application.
3. Mop up liquid spills with a dry cloth.

PACKAGES

1. Bottles of powder, 100g, 250g, 1kg.
2. Bottles of liquid, 100g (104mL), 250g

SHADES

1. Crown colours : Ivory.
2. Gingival colours : Pink # 3, Live pink # 8, Pink X.

WARNING

1. The liquid is HIGHLY FLAMMABLE. Avoid sources of ignition.
2. In case of contact with eyes, flush immediately with water and seek medical attention.

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