VivaStyle®

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ivoclar®
vivadent®
clinical

Application

- After a dental examination has been carried out and the duration of the treatment has been determined, VivaStyle 10% or VivaStyle 16% is administered by the patient at home as instructed by the dental professional.
- The first application of any tooth whitening cycle should be performed by dentists or under their supervision, provided the same level of safety is guaranteed. Subsequently, the gel must be placed at the disposal of the patient to complete the bleaching cycle at home.
- Before VivaStyle 10% or VivaStyle 16% is used, the dental professional should carefully examine the patient's medical history, inform the patient about the application procedure and clean the patient's teeth professionally. It is advisable for the dental professional to monitor the whitening process.

Step-by-step instructions

1. Prior to the application of the VivaStyle gel, thoroughly brush and floss your teeth.
2. Remove the cap from the tip of the syringe. Place a small amount of gel toward the facial side of each tooth in the tray. Use the calibration markings on the syringe as a guide when dispensing the material. Do not overfill the gel in the trays. Place the cap back on the application tip.
3. Place the gel-filled trays over your teeth. Do not squeeze the tray when you insert it.
4. Use a toothbrush, cotton bud or index finger to remove excess gel. Do not swallow.
5. Wear the tray as prescribed by your dentist, e.g. for one hour a day.
6. Remove the tray after the prescribed time.
7. Rinse the mouth thoroughly to remove residual gel. Spit everything out – do not swallow.
8. Clean the tray with a toothbrush under running water. Always store the tray in the accompanying tray case.
9. Repeat the treatment with the VivaStyle gel every day if possible or according to the schedule prescribed by your dentist.

Notes

- Overfilling the tray does not result in faster or better results. Instead, it may irritate the gums.

- Do not eat, drink or smoke while wearing the tray.
- Until you have finished the full treatment
 - avoid consuming citrus fruits and fruit juices, as they can promote hypersensitivity;
 - avoid, or at least try to minimize, the consumption of coffee, tea, red wine and colas as well smoking to prevent the risk of staining the teeth.
- The gel only whitens natural teeth. After the treatment, the appearance of existing restorations in the front teeth may no longer match that of the whitened teeth. Therefore, these restorations may have to be replaced.
- The treatment with VivaStyle 10% or VivaStyle 16% should be completed at least two weeks prior to the placement of tooth-coloured restorations.
- The teeth cannot be made to look whiter than their natural colour.

If you have any questions about the treatment or if you experience uncomfortable sensitivity, please contact your dental office.

Warning

Avoid contact of the material with the skin, mucous membrane and eyes. After accidental contact, rinse the affected area with copious amounts of water. If contact with the eyes occurs, immediately rinse with plenty of water and seek medical advice.

Storage

- Protect VivaStyle syringes from heat and sunlight
- Do not use after the date of expiration
- Store at 2–28 °C / 36–82 °F.

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English

Description

VivaStyle is a ready-for-use whitening gel for lightening discoloured teeth. The product is administered by the user. The gel is applied by means of a custom-fitted tray.

Composition

Glycerin, Aqua, Urea Peroxide, Carbomer, Potassium Nitrate, Sodium, Hydroxide, Aroma, Disodium EDTA, VivaStyle 16% only: Sodium Saccharin.

Active ingredients: 10% or 16% carbamide peroxide, which is equivalent to 3.6% and 5.8% hydrogen peroxide respectively.

One syringe contains 3.7 g gel. This corresponds to 3 ml.

Indication

- Whitening of discoloured vital teeth
- Whitening of discoloured non-vital teeth

Contraindication

VivaStyle Gel should not be used

- during pregnancy or lactation;
- by children;
- by heavy smokers;
- by people who consume large quantities of alcohol;
- by people who are known to be allergic to any of the material's ingredients;
- in the presence of cervically exposed teeth;
- in the presence of untreated primary or secondary caries;
- in the presence of restorations with marginal gaps.

Applicable to EU countries:

- VivaStyle is not intended for use by patients under the age of 18
- For sale to dentists only

Side effects

Temporary sensitivity of the oral mucous membrane or the teeth may occur. These side effects usually subside within a short time after the treatment is finished or discontinued.

Application

- A dental professional should examine the patient's medical history, establish the indication and determine the duration of the treatment. It is advisable for the dental professional to monitor the whitening results. VivaStyle 10% or VivaStyle 16% is used by the patient at home as instructed by the dental professional.
- The first application of any tooth whitening cycle should be performed by dentists or under their supervision, provided the same level of safety is guaranteed. Subsequently, the gel must be placed at the disposal of the patient to complete the bleaching cycle at home.
- VivaStyle should be used before new tooth-coloured restorations are placed, since restorative materials cannot be whitened. The treatment with VivaStyle should be completed two weeks prior to the placement of tooth-coloured restoration. Existing restorations may not match the appearance of the tooth structure after the whitening process and therefore may have to be replaced.

Step-by-step instructions

Pretreatment

Before the VivaStyle tooth whitening gel is used, the dental professional should carefully examine the patient's medical history and clean the patient's teeth professionally.

Tray fabrication

A properly fabricated tray will enhance the effectiveness of the whitening process. Well defined trays will improve patient comfort and reduce the chance of irritating the mucous membrane.

1. Take an impression and fabricate the stone model.
2. Heat the tray material on a vacuum former until it sags 1 to 2 cm. Adapt it to the model and place it under vacuum for 30 seconds.
3. After cooling, remove the tray from the model and remove excess material.
4. Carefully trim the tray about 0.5–1 mm short of the gingival margins, using either fine scissors or a blade.
5. Place the tray back on the stone model to assess the fit. Final shaping and defining of the periphery may be accomplished with a coarse acrylic polisher.

Instructing the patient

Careful instruction of the patient is fundamental to the safe and effective application of VivaStyle tooth whitening gel at home.

1. Review each individual step of the application procedure in detail with your patient, using the enclosed information on VivaStyle.
2. Make sure that the tray fits accurately.
3. Using a VivaStyle gel syringe, show the patient how to fill the tray correctly. Important: Do not overfill the trays and only apply gel on the surfaces where it is needed. The calibration markings on the syringe should be used as a guide when extruding the material.
4. Show the patient how to insert the tray into the mouth. Explain the proper procedure for removing any excess tooth whitening gel.
5. Determine how long the patient should wear the tray. One hour a day is usually sufficient. Depending on the individual requirements of the patient, a longer wear time may be prescribed. The overall application time should not exceed four to six weeks.
6. Explain the proper removal and cleaning procedures for the tray. The tray should be stored in the accompanying case.

Notes

- The patient should be provided with the correct number of syringes needed for the treatment.
- Patients should be informed about the following points in particular:
 - They should avoid the consumption of citrus fruit and fruit juices during the treatment, as they can promote sensitivity.
 - They should try to avoid, or at least minimize, the consumption of coffee, tea, red wine and colas as well as smoking during the treatment to prevent the risk of staining the teeth.
 - Overfilling the tray with tooth whitening gel does not result in faster or better results. Instead, it may promote sensitivity.
 - Excess gel should be carefully removed to prevent irritation of the gums.
- Patients should call their dentist if they have any questions about the treatment or if unpleasant sensitivity occurs.

Warning

Avoid contact of the gel with the skin, mucous membrane and eyes. After accidental contact, rinse the affected area with copious amounts of water. If contact with the eyes occurs, immediately rinse with plenty of water and seek medical advice.

Storage

- Protect VivaStyle syringes from heat and sunlight
- Do not use after the date of expiration
- Store at 2 – 28 °C / 36 – 82 °F.

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Ivoclar Vivadent Limited
Ivoclar Vivadent, Inc.