Insert

Instructions for use

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EN Instructions for use.

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DE

EINFÜGUNG

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Behandlungen: Prophylaxe, Endodontie, Parodontologie, konservierende und restaurative Zahn-heilkunde.

Verwendung:

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EN

INSERT

Material: High-quality stainless steel.

Treatments: Prophylaxis, endodontics, periodontics, conservative and restorative dentistry.

Use:

1. Prophylaxis: use the recommended tips and the corresponding potency for scaling in the ultrasonic scaler.
2. Perio: use the recommended tips and the corresponding potency for perio in the ultrasonic scaler.
3. Endo: use the recommended tips and the corresponding potency for the endo function in the ultrasonic scaler.

Storage indications:

The tips must be stored in a clean, dry and well ventilated area, situated indoors and away from corrosive gases.

Contraindications:

- Do not use on patients with haemophilia.
- Use by doctors with pacemakers and on patients with pacemakers is strictly prohibited.
- Use carefully on heart disease patients, pregnant women and children.

Precautions:

- The tip must be firmly screwed on to the hand-piece.
- The tip must be tightened when working with sprays.
- The tip must be cleaned and disinfected before use.
- Replace the tips when the outer layers are damaged or worn.
- Do not bend or fold the end of the tip.
- The golden tips must be rinsed immediately with water after being soaked in H₂O₂ NaClO, otherwise they will deteriorate.
- Always check the compatibility of the tips with your ultrasonic scaler.

Connection: Use the tightening wrench (torque wrench) to screw in the tips.

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Guilin Yikeshi Machine Co., Ltd,