Temporary restorative material

Instructions for use

- DE** Gebrauchsanweisung.
- EN** Instructions for use.
- ES** Instrucciones de uso.
- FR** Mode d'emploi.
- IT** Istruzioni per l'uso.
- PL** Instrukcja postępowania.
- PT** Instruções de uso.

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TEMPORÄRES RESTAURATIONS MATERIAL

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INSTRUCTIONS FOR USE **EN**

TEMPORARY RESTORATIVE MATERIAL

Proclinic temporary restorative material is a zinc oxide based reinforced cement for temporary restorations and bases.

WAY OF USE

Spatulate over a glass surface adding parts of the powder to the liquid until all the powder is added. Place the cement into a dried and retentive cavity. If the pulp is very closed, a base liner of calcium hydroxide or better calcium hydroxiapatite should be used. Mix two spoons of powder, with a drop of liquid. Do not press the powder and remove the excess of powder with a spatula. The spoon is included in the kit. • If the mix is done on different proportions, the physical properties can vary.

MIX

Mix over a glass surface by a spatula, adding to the liquid small parts of the powder until all is added. Spatulate until a uniform mass is obtained. The time of mixing should be not longer than 1 min. Place the cement into a dried and retentive prepared cavity. If the pulp is very closed or exposed, a calcium hydroxide can be placed. If the occlusion should be controlled, use a matrix in the usual way. Working time: 2,5 min. Setting time: 5,5 min.

PRECAUTIONS

The Zinc oxide cement can interfere in curing resins. In case of contact with eyes rinse and get medical attention. Keep the bottles tight closed and separated from sunlight.

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