

RELIANCE ETCHANT

INDICATIONS FOR USE: Reliance Etchants are intended for use on enamel to prepare the surface for bonding.

PRODUCT FEATURES

Etchant is 37% phosphoric acid (H_3PO_4) polymer-thickened for use on enamel to prepare the surface for bonding. The etchant will effectively etch and demineralize enamel producing the necessary retentive surface for successful bonding. Our specially formulated etchant leaves no silica residue.

INSTRUCTIONS FOR USE

1. Isolate the teeth for etching.
2. With a cotton pledget or brush, dab the etching agent onto the entire area to be bonded. Do not rub the etching agent onto the enamel. Allow 30 seconds for etching (60 seconds for deciduous teeth). Avoid placing etching agent on soft tissue.
3. Rinse thoroughly (at least 5 seconds per tooth) to completely remove all traces of etchant.



CAUTION: Etching liquid or gel (phosphoric acid) is a severe eye and tissue irritant. If accidentally instilled into the eye, flush with copious amounts of water and seek medical attention immediately. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes. Injury may result if etchant is allowed to remain on the skin or mucosa for extended periods of time.



TEMPERATURE LIMITATION: Store at room temperature (15-30°C / not to exceed 86 °F.) or can be refrigerated (2-8°C /36-46°F.)

TO REORDER: 23 Gram Liquid: Ref# EL24, 23 Gram Cherry: Ref# EL24C, 9 Gram Liquid: Ref# EL9, 18 Gram Gel: EG18, 9 Gram Gel: Ref# EG9

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, either direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.


Rx Only: U.S. Federal law restricts this device to sale by or on the order of a dental professional.



Reliance Orthodontic Products, Inc.

1540 West Thorndale Ave., Itasca, IL 60143 U.S.A.
1-800-323-4348 · www.Relianceorthodontics.com

Australian Sponsor: Emergo
Australia
201 Sussex St. Darling Park
Tower II, Level 20,
Sydney, NSW 2000 Australia

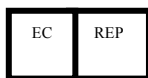

EMERGO EUROPE
Molenstraat 15
2513 BH, The Hague
The Netherlands



Scan for more information

